English
<b>Phosphoric Acid Gel Etchant 37.5%</b>
Instructions
<ol style="list-style-type: none"><li>Apply Kerr Gel Etchant* to enamel and dentin surfaces for 15 seconds.</li> <li>Rinse thoroughly for 15 seconds with water. It is important not to splash etch into eyes, onto soft tissues or skin.</li> <li>Dry with clean, oil-free, air without desiccating the dentin. Do not allow etched surface to become contaminated with saliva or blood during the bonding procedure.</li> <li>Proceed with placement of bonding agent.</li></ol>

- Apply Kerr Gel Etchant\* to enamel and dentin surfaces for 15 seconds.
- Rinse thoroughly for 15 seconds with water. It is important not to splash etch into eyes, onto soft tissues or skin.
- Dry with clean, oil-free, air without desiccating the dentin. Do not allow etched surface to become contaminated with saliva or blood during the bonding procedure.
- Proceed with placement of bonding agent.

**NOTE:** Never use excessive force on syringe plunger to dispense etch which may lead to over dispensing of the material. If resistance is felt replace current tip with a new tip and check flow extraorally.

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**\* Caution:** Contains phosphoric acid. Avoid contact with skin, eyes and soft tissue. In case of contact with skin or eyes, flush immediately with water. Get medical attention for eyes. Do not take internally.

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Kerr’s technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr’s product. Such advice does not expand Kerr’s limited warranty or relieve the dentist of testing Kerr’s products to determine their suitability for the intended uses and procedures. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr’s product.

In the event of a defect in material or workmanship, Kerr’s liability is limited, at Kerr’s option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages. EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY OR FITNESS FOR A PARTICULAR PURPOSE

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Japan, Inc.