3M ESPE
VPS Tray Adhesive
(en) Vinyl Polysiloxane Adhesive
(de) Vinyl Polysiloxan Adhäsiv
(fr) Adhésif de vinyl polysiloxane
(it) Adesivo vinilpolisilossano
(es) Adhesivo de vinil polisiloxano
(pt) Adesivo de polissiloxano de vinilo
(nl) Vinyl polysiloxane adhesief
(el) Συγκολλητικός παράγοντας για βινυλπολυσιλοξάνη
(sv) Vinylpolysiloxanadhesiv
(fi) Vinyyli-polysiloksaani adhesiivi
(da) Vinylpolysiloxan-adhæsiv
(no) Vinylpolysiloksan adhesiv

Instructions for Use	Gebbruksinformatie
Gebrauchsinformation	Οδηγίες χρήσης
Mode d’emploi	Bruksanvisning
Informazioni per l’uso	Käyttöinfo
Informaciones de uso	Bruksanvisning
Instruções de uso	Bruksinformatjon

	
3M ESPE AG	CE
Dental Products	
D-82229 Seefeld – Germany	

3M ESPE Dental Products, St. Paul, MN 55144-1000 U.S.A.

3M ESPE Customer Care/MSDS Information: U.S.A. 1-800-634-2249 and Canada 1-888-363-3685.

3M and ESPE are trademarks of 3M or 3M ESPE AG. © 3M 2011. All rights reserved.

ENGLISH

Product Description
VPS Tray Adhesive, manufactured by 3M ESPE, creates tight adhesion between impression trays and addition type silicon impression materials from 3M ESPE.

☞ These Instructions for Use should not be discarded for the duration of product use.

Indications

- Impression trays made of metal, methacrylate, or polypropylene.
- Vinyl Polysiloxane impression materials (not suitable for use with polyether impression materials).

Precautionary Measures

VPS Tray Adhesive is highly flammable. Avoid electrostatic charge. Keep away from heat, sparks, open flame, pilot lights, and other ignition sources.

3M ESPE MSDSs can be obtained at www.mmm.com or from your local subsidiary.

Application

- After checking the impression tray for fit in the patient’s mouth, clean the tray to prevent any contamination of the adhesive brush.
- Use the brush to apply a thin layer of the adhesive to the tray.
- Allow to dry completely; 5–15 minutes are ideal.
- Carefully close the bottle tightly after use. If present, remove any adhesive remnants from the threads of the bottle and screw cap with a cloth soaked in acetone before closing the bottle.

Cleaning

- Use ethanol to remove any impression or adhesive residues clinging to the metal impression tray.

Notes

- Do not pour together small amounts of adhesive remaining in a number of bottles. Even a slight increase in temperature can cause overfilled bottles to burst.
- VPS Tray Adhesive does not always provide adequate adhesion to impression trays made of other plastics or base plates.
- Avoid application of the adhesive to textiles, plastic or varnished surfaces, or similar areas at all times because its removal without any residue cannot be guaranteed.
- Use VPS Tray Adhesive only in well-ventilated rooms.
- Rinse with water if the adhesive comes into contact with the skin.

Storage and Stability
Store the product at 15–25°C/59–77°F. Do not use after the expiration date.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE’s sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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