
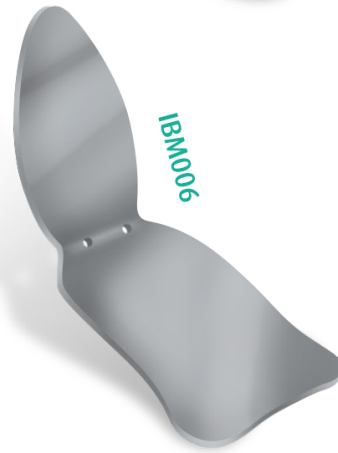
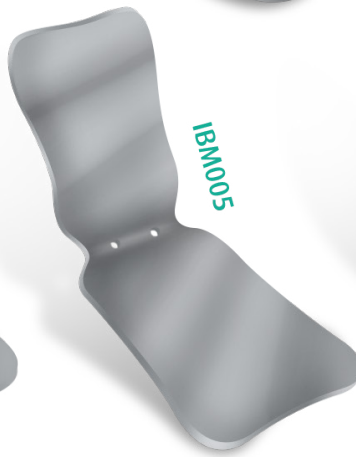
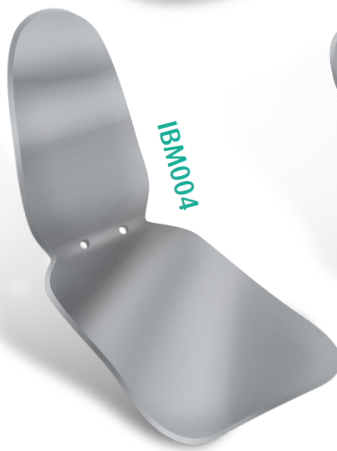
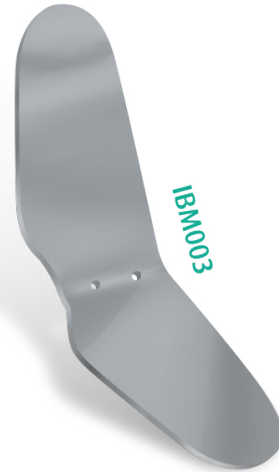
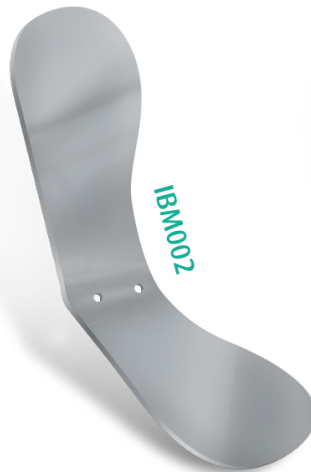


Metallic Mirror

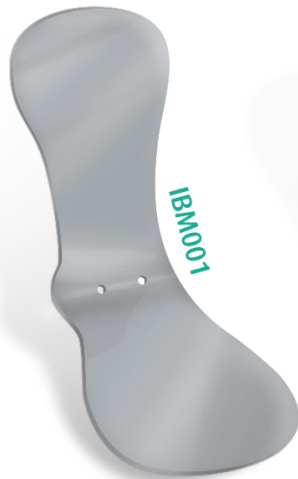

- Perfect image reflection
- Round edges
- Perfect fit in the mouth
- Maximum comfort

METALLIC MIRROR: It allows perfect image reflection during intraoral dentistry image capturing. Its round edges give comfort to the patients, the opposite side not used to reflect the image works as a handle preventing the professional hands to appear in the pictures.

INDICATION: Intraoral photograph capturing.

Mirror IBM 001 Adult Occlusal and child

Mirror IBM 002 Lingual

Mirror IBM 003 Lateral

Mirror IBM 004 Adult Occlusal and lateral

Mirror IBM 005 Adult Occlusal and child

Mirror IBM 006 Child Occlusal and Lateral

USE INSTRUCTIONS: Select the METALLIC MIRROR according to the usage indication. Take it to the patient's mouth and keep it positioned to capture the photograph.

CLEANING: The organic matter removal must be performed in ultrasonic cleaner, using enzymatic soap (minimum three enzymes) in dilution recommended by the manufacturer. After washing it in running water, observe presence of residues, and repeat the process if necessary. If any residues is left, remove it manually.

DISINFECTION: 10 minutes in 0.2% peracetic acid solution.

STERILIZATION: Should be done by autoclaving at a temperature of 121°C up to 135°C (249°F up to 275°F), observing the manufacturer's autoclaving instructions manual. Repeat the process for each use.

STORAGE: Keep the product in a designated and closed cabinet protected from dust and insects.

COMPOSITION: Chromed brass.

PRECAUTION: Sterilize the product before use. Do not touch the product at autoclave's inner wall. Do not use steel wool or abrasives to clean the mirror, it may result in scratches and damage the mirrors. In order to avoid blurriness it is recommended to use compressed air before or during the procedure.

EXPIRY DATE: 05 years from manufacturing date in a non-violated package. Products durability depends on how it is handled. Mishandling it and incorrect sterilization may reduce its useful life. Check the products condition before using it and replace it when observing any damages.

Code

Description

Presentation

ANVISA

106210101008

Metallic Mirror Kit IBM001, IBM002 and IBM003

1 Kit

180213420017

106210101009

Metallic Mirror IBM001

1 unit

106210101010

Metallic Mirror IBM002

1 unit

106210101011

Metallic Mirror IBM003

1 unit

106210101012

Metallic Mirror IBM004

1 unit

106210101013

Metallic Mirror IBM005

1 unit

106210101015

Metallic Mirror IBM006

1 unit

 **Indusbello Ind. e Com. de Prod. Méd. e Odontológicos. Ltda.**

CNPJ 74.017.708/0001-91 | Aut. Func./MS 15341379-MW34 +55 (43) 3378-8300

R. Tereza de Souza, 171 | Pq. Ind. Kiugo Takata CEP 86042-390 Londrina-Paraná

Resp. Técnico: Willian Ramos - CRQ IX Região 09903397

Dr. Rubens Henrique Pinheiro de Souza - CRO-PR 6645

 Made in Brazil