ENGLISH —

Product Description
3M Dual Arch Impression Tray is a disposable impression tray made of plastic that can be used for small impression cases (1-2 units). The trays are used anterior and posterior. Instructions for Use should not be discarded for the duration of product use.

Suitable impression materials:

- Polyether or VPS impression materials: medium consistency used in the monophase technique and medium or heavy consistency as tray material used with a light consistency as a wash material in a 1-step technique.

Putty consistencies are not suitable for this tray.

- Crown/Bridge Impression
- Bite Registration
- Master and Counter Impression

Impression-taking

- Select a suitable tray according to the impression need - anterior or posterior.
- Place unloaded 3M Dual Arch Impression Tray into proper position. When using the posterior, or sideless Dual Arch Impression Tray, make sure the crossbar of the tray is distal to the last maxillary tooth and the tray is centered buccally/lingually.
- Have the patient "close" on the unloaded tray and check the opposite side of the patient's mouth to make sure that the patient has closed into the correct occlusion.

- If the impression tray is tested for fit in the patient's mouth, clean the tray prior to using adhesive to prevent any contamination of the adhesive brush.

- Point impression adhesion on the tray walls only, not on webbing. Load both sides of 3M Dual Arch Impression Tray with impression material. **Note:** Dual Arch Impression Tray is suitable for a firm setting material. It may not be used in a two-step technique.

- Remove retraction cord and/or rinse off retraction paste if used. Syringe wash impression material around the prepared tooth and make sure to fill any edentulous areas.

- With syringed impression material in place and still unset, place the 3M Dual Arch Impression Tray into proper position. Make sure crossbar is correctly placed (distal to the most posterior tooth).

- Have patient close properly (verifying the teeth on opposite side are touching as normal bite).
- When set, have patient open mouth and remove the impression tray from mouth. Inspect the final impression. No grey plastic pressure areas should be visible.

Disinfection of the Impression
Place the impression in a standard disinfectant solution. Follow the disinfectant solution manufacturer instructions for wet contact time on the setting material. After disinfection, rinse the impression under running tap water for approximately 15 seconds, then dry.

Disposal
The impression tray is made of high-density polyethylene and can be recycled in accordance with local regulations after the impression material has been removed. Do not discard the tray along with the impression material in the usual practice waste.

Customer Information
No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Waranty
3M warrants this product to be free from defects in material and manufacture. 3M MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for use. If the product is found to be defective within the warranty period, your exclusive remedy and 3M's sole obligation shall be repair or replacement of the 3M product.

Limitation of Liability
Except where prohibited by law, 3M will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

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