3M ESPE

Impression™ 4 Preliminary

EN VPS Preliminary Impression Material

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Application

- Load the impression tray with one of the impression materials listed above, subsequently place the filled tray in the mouth parallel to the long axis of the teeth, and hold it in place without applying any pressure.
- Remove the impression from the mouth once the setting has elapsed.

After Impression Taking

► Thoroughly examine and explore the sulcus of the teeth and surrounding dentition. Remove any residual impression material from the mouth.

Higiene

► Place the impression in a standard disinfectant solution for the period of time recommended by the manufacturer. Excessive disinfection may damage the impression.

► After disinfection, rinse the impression under running water for approx. 15 sec.

Temporary Preparation

► Impressions for temporaries may be cut with scissors or a scalpel.
► Any commercial temporary crown and bridge materials may be used, e.g., 3M ESPE Protemp materials. Materials for temporaries set slightly more rapidly than with alginate materials.

Model Preparation

► For optimal results, wait 30 minutes before pouring the impression. There are no other restrictions on time for impressions stored at room temperature.

► Unlike alginate, the impressions may be stored repeatedly.
► Any commercial dental stone or temporary crown and bridge materials may be used.

► To avoid introducing bubbles, briefly rinse with water shortly before pouring and blow-dry. The hydrophilic properties of Impression 4 Preliminary impression material make it easy to pour. The use of silicone surfactants is not required.

Cleaning

► Prior to setting, the paste can be removed from surfaces with ethanol.

► Avoid exposing the material to clothing since the material cannot be removed by chemical cleaning.

Notes

► Keep the filled mixing tip permanently immersed in the paste to avoid the need for use. Removed and subsequent recharging of a used mixing tip may lead to cross-contamination by residual paste and ensuing plugging.

► 3M ESPE Preliminary impression materials must not be exposed to condensation-cured silicones or polyether materials. Even traces of these materials interfere with the setting of the impression.

► Disposable latex gloves may impair the setting of silicone impression materials. Vinyl gloves are more suitable.

► Impression 4 Preliminary impressions must not be exposed to solvents or solvent-containing liquids as this would lead to swelling and inaccuracy.

Storage and Shelf-life

► Store the product at 15–25° C/59–77°F.

► Do not use after the expiration date.

► Store set impressions at temperatures below 30° C/86°F.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warnarty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. Use is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Product	ISO 4823 Consistency	Paste Color	Mixing Ratio (Vol.)	Mixing Device
Imprint 4 Preliminary Penta	Type 2 medium-bodied	light pink	5:1	All Pentamix™ mixing device versions
Imprint 4 Preliminary Penta Super Quick	Type 2 medium-bodied	light pink	5:1	All Pentamix™ mixing device versions
Imprint 4 Preliminary Super Quick	Type 2 medium-bodied	light pink	1:1	Garant™ Dispenser

*These instructions for use should not be discarded for the duration of product use. For details on all additionally mentioned products, please refer to the corresponding instructions for Use.

Indications

- Impressions for the production of temporary restorations.
- All types of preliminary impressions
- Impressions of the opposing jaw
- Impressions for orthodontic models

Precautionary Measures

For Patients

This product contains substances that may cause an allergic reaction by skin contact in certain individuals. Avoid use of this product in patients with known mint oil allergies. If prolonged contact with oral soft tissue occurs, flush with large amounts of water. If allergic reaction occurs, seek medical attention as needed, remove the product if necessary and discontinue future use of the product.

For Dental Personnel

This product contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular, avoid exposure to uncured product. If allergic reaction occurs, seek medical attention as needed. Use of protective gloves and a no-touch technique is recommended. Mint oil may penetrate commonly used gloves. If product contacts glove, remove and discard glove, wash hands immediately with soap and water and then re-glove. If allergic reaction occurs, seek medical attention as needed.

3M MSDSs can be obtained from www.mmm.com, or contact your local subsidiary.

Preparation

Preparing the Impression Tray:

Rigid non-perforated or perforated trays are most suitable, e.g., 3M ESPE Impression Tray.
► When using 3M ESPE Impression Trays, no tray adhesive is required.
► With other impression trays, apply VPS Tray Adhesive for sufficient adhesion and allow it to dry completely; the ideal drying time is 5–15 minutes.

Preparing the Penta Cartridge:

► Insert the impression material to be used in the designated 3M ESPE Penta cartridge only.

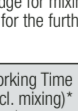
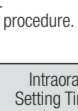
► Place the cartridge in the Pentamix mixing device and attach a new Penta Mixing Tip – Red.

– If a new mixing tip is installed when the cartridge is inserted, check whether the drive shaft is engaged before you start mixing.
► When newly filled cartridges, start mixing and extrude and discard the first unevenly mixed paste prior to the action of use for impression taking. Do not use the paste to take an impression until the color of the paste is homogeneous.

► Please follow the section "Impression Taking" for the further procedure.

Preparing the Garant Dispenser:

► Place the cartridge of the impression material to be used in a Garant dispenser.
► When using new cartridges, remove the cap and check that both cartridge openings are not clogged and extrude a small amount of paste without the mixing tip until base paste and catalyst are evenly extruded.
► With cartridges that have already been used, remove the mixing tip and check that both cartridge openings are not clogged.
► Attach a Garant mixing Tip "Green" to the cartridge for mixing.
► Please follow the section "Impression Taking" for the further procedure.

Symbols on product	Working Time (incl. mixing) min:sec	Intraoral Setting Time min:sec
		
Imprint 4 Preliminary Penta	1:30	2:00
Imprint 4 Preliminary Penta Super Quick	1:00	1:30
Imprint 4 Preliminary Super Quick	1:30	2:00
Imprint 4 Preliminary Super Quick	1:00	1:30

* These are the processing times at 23° C/74° F. Higher temperatures shorten, while lower temperatures prolong these times.

Impression Taking

When applying any kind of paste be sure:

► The impression tray is prepared with adhesive.
► That the base paste and catalyst paste flow evenly into the mixing tip and extrude homogeneously in contact and free of streaks; discard any unevenly mixed paste;

► To keep the mixing tip permanently immersed in the paste to avoid the formation of air bubbles;

► Not to exceed the processing time to avoid incomplete and inaccurate impressions.

► Not to extrude set material by force. This may damage the cartridge and mixing tip resulting in the formation of leaks.

Application

► Load the impression tray with one of the impression materials listed above, subsequently place the filled tray in the mouth parallel to the long axes of the teeth, and hold it in place without applying any pressure.

After Impression Taking

► Thoroughly examine and explore the sulcus of the teeth and surrounding dentition. Remove any residual impression material from the mouth.

Higiene

► Place the impression in a standard disinfectant solution for the period of time recommended by the manufacturer. Excessive disinfection may damage the impression.

► After disinfection, rinse the impression under running water for approx. 15 sec.

Temporary Preparation

► Impressions for temporaries may be cut with scissors or a scalpel.
► Any commercial temporary crown and bridge materials may be used, e.g., 3M ESPE Protemp materials. Materials for temporaries set slightly more rapidly than with alginate materials.

Model Preparation

► For optimal results, wait 30 minutes before pouring the impression. There are no other restrictions on time for impressions stored at room temperature.

► Unlike alginate, the impressions may be stored repeatedly.
► Any commercial dental stone or temporary crown and bridge materials may be used.

► To avoid introducing bubbles, briefly rinse with water shortly before pouring and blow-dry. The hydrophilic properties of Impression 4 Preliminary impression material make it easy to pour. The use of silicone surfactants is not required.

Cleaning

► Prior to setting, the paste can be removed from surfaces with ethanol.

► Avoid exposing the material to clothing since the material cannot be removed by chemical cleaning.

Notes

► Keep the filled mixing tip on the cartridge to serve as a closure until the next use. Removed and subsequent recharging of a used mixing tip may lead to cross-contamination by residual paste and ensuing plugging.

► 3M ESPE Preliminary impression materials must not be exposed to condensation-cured silicones or polyether materials. Even traces of these materials interfere with the setting of the impression.

► Disposable latex gloves may impair the setting of silicone impression materials. Vinyl gloves are more suitable.

► Impression 4 Preliminary impressions must not be exposed to solvents or solvent-containing liquids as this would lead to swelling and inaccuracy.

Storage and Shelf-life

► Store the product at 15–25° C/59–77°F.

► Do not use after the expiration date.

► Store set impressions at temperatures below 30° C/86°F.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warnarty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. Use is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Indications

► Impressions for the production of temporary restorations.

► All types of preliminary impressions

► Impressions of the opposing jaw

► Impressions for orthodontic models

Precautionary Measures

For Patients

This product contains substances that may cause an allergic reaction by skin contact in certain individuals. Avoid use of this product in patients with known mint oil allergies. If prolonged contact with oral soft tissue occurs, flush with large amounts of water. If allergic reaction occurs, seek medical attention as needed, remove the product if necessary and discontinue future use of the product.

For Dental Personnel

This product contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular, avoid exposure to uncured product. If allergic reaction occurs, seek medical attention as needed. Use of protective gloves and a no-touch technique is recommended. Mint oil may penetrate commonly used gloves. If product contacts glove, remove and discard glove, wash hands immediately with soap and water and then re-glove. If allergic reaction occurs, seek medical attention as needed.

3M MSDSs can be obtained from www.mmm.com, or contact your local subsidiary.

Preparation

Preparing the Impression Tray:

Rigid non-perforated or perforated trays are most suitable, e.g., 3M ESPE Impression Tray.
► When using 3M ESPE Impression Trays, no tray adhesive is required.
► With other impression trays, apply VPS Tray Adhesive for sufficient adhesion and allow it to dry completely; the ideal drying time is 5–15 minutes.

► Please follow the section "Impression Taking" for the further procedure.

► Prepare the Garant dispenser:

► Place the cartridge of the impression material to be used in a Garant dispenser.

► When using new cartridges, remove the cap and check that both cartridge openings are not clogged and extrude a small amount of paste without the mixing tip until base paste and catalyst are evenly extruded.

► With cartridges that have already been used, remove the mixing tip and check that both cartridge openings are not clogged.

► Attach a Garant mixing Tip "Green" to the cartridge for mixing.

► Please follow the section "Impression Taking" for the further procedure.

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