


## Steps for using Imprint<sup>™</sup> 4 VPS Impression Material in a 1-Step Impression Technique

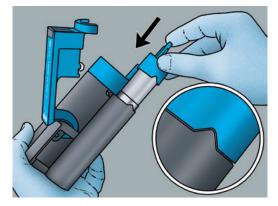


Place the Penta<sup>™</sup> foil bags into the designated Penta<sup>™</sup> cartridge.

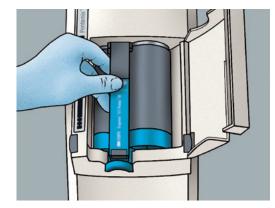


Insert the loaded cartridge into the Pentamix<sup>™</sup> Automatic Mixing Unit.

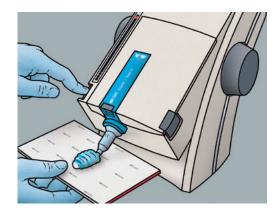


When using new foil bags, start mixing and extrude a small amount until paste is homogeneous in color. Discard.

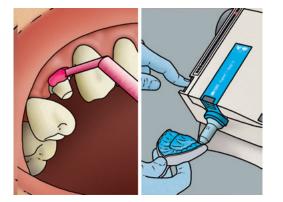

While the tray is being loaded with heavy-body or putty impression material, apply wash material around the preparation using a wiggle and stir motion.

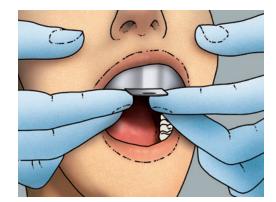


Insert the loaded tray straight and evenly into mouth—avoid rotation. Don't let patient bite down on tray. Immobilize the tray using passive pressure. Follow recommended setting time in the mouth using a timer.

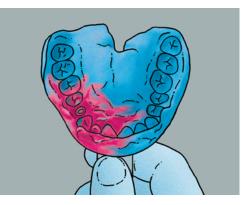


When material is completely set, carefully remove tray from mouth and evaluate the final impression.


## Steps for using Imprint<sup>™</sup> 4 VPS Impression Material with a Garant<sup>™</sup> Triple Tray Technique

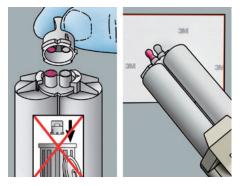

When using new Garant<sup>™</sup> cartridges, extrude a small amount without the mixing tip until base paste and catalyst are evenly extruded, and discard.

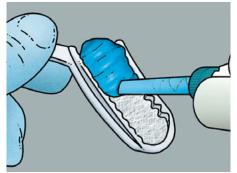


Dispense heavy-body material into the tray, keeping the mixing tip immersed in the paste to avoid formation of air bubbles.

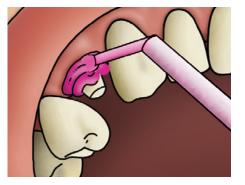

Syringe wash material around the margin.



Place tray and have patient gently bite down, and hold. Follow recommended setting time in the mouth using a timer.

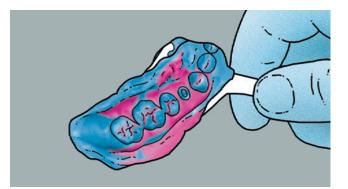

When material is completely set, carefully remove tray from mouth and evaluate the final impression.



3M Oral Care 2510 Conway Avenue St. Paul, MN 55144-1000 USA

Phone 1-800-634-2249 Web 3M.com/dental 3M Canada Post Office Box 5757 London, Ontario N6A 4T1 Canada Phone 1-888-363-3685

## www.3M.com/GreatImpressions

3M, 3M Science. Applied to Life. and Imprint are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. © 3M 2016. All rights reserved. 70-2013-0755-3