

3M™ Paradigm™ Temporization Material

Tips for Success

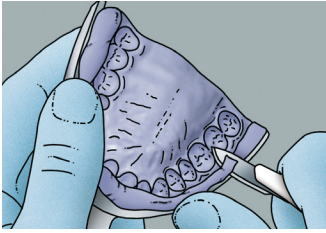


Fig. 1: Remove excess material as needed.

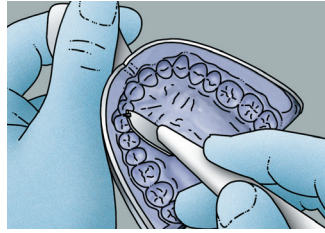


Fig. 2: Remove interproximal septa.

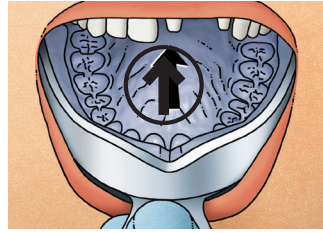


Fig. 3: Reposition matrix in the mouth to confirm proper fit.

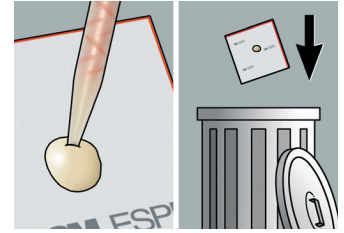


Fig. 4: Bleed a small amount of material through the mix tip before dispensing in the matrix. Discard extruded material.

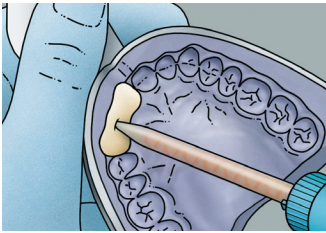


Fig. 5: Dispense material into the impression or matrix.

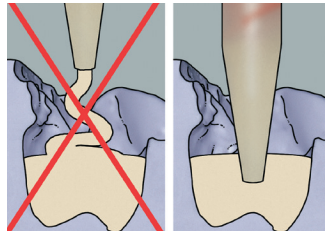


Fig. 6: Make sure to fill from the bottom up and to always keep the tip immersed in the temporary material to prevent voids. (Working time: 40 sec.)

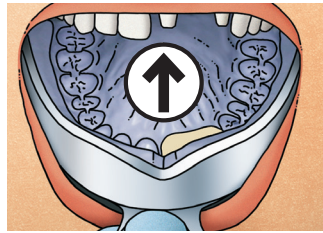


Fig. 7: Seat in the mouth and hold firmly in place.

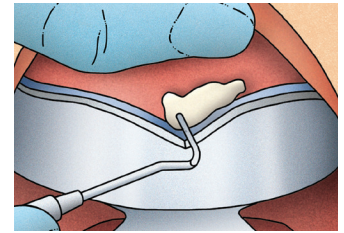


Fig. 8: Check setting progress with excess material.

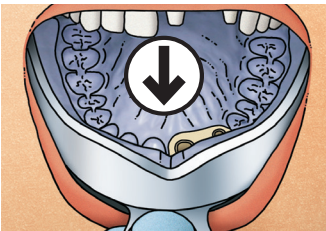


Fig. 9: Remove from mouth between 1:40 and 2:50 (min:sec) from the onset of mixing.

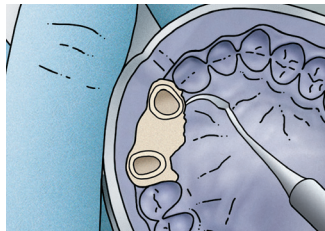


Fig. 10: Leave material in the matrix for 5:00 (min:sec). This is important to achieve a smooth and shiny surface.

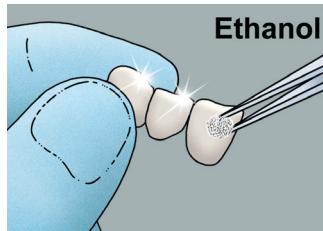


Fig. 11: Remove oxygen inhibited layer with ethanol.

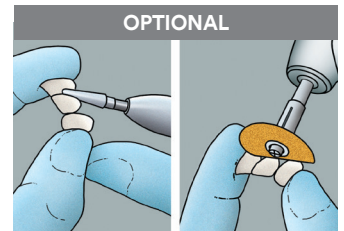


Fig. 12: Finish and polish as needed.

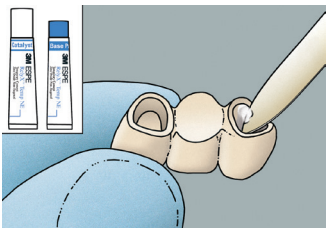


Fig. 13: Cement provisional restoration using a temporary cement.

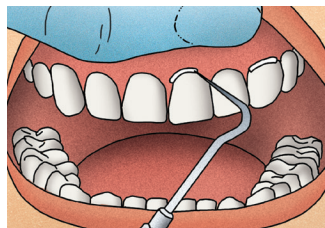


Fig. 14: Remove the excess.

⚠ DON'T FORGET TO:

- Bleed cartridge before first use
- Bleed tip before each use
- Use only approved 3M™ Mixing Tips
- Adhere to working and setting times as detailed in the Instructions for Use
- Following each use, use the mix tip as the cap for sealing cartridge during storage

3M™ Paradigm™ Temporization Material... Innovative solutions for your temporary needs.



Shades match 3M™ Filtek™ Supreme Ultra Flowable Restoratives

Three popular shades (A1, A2 and A3) match beautifully with global market leader Filtek Supreme Ultra Flowable restoratives.



Ordering Information

Item No. Product Information

3M™ Paradigm™ Temporization Material

Refills

46993	3M™ Paradigm™ Temporization Material Refill – A1 Includes: 1 cartridge – 50ml, A1; 16 cartridge mixing tips, blue; Instructions
46994	3M™ Paradigm™ Temporization Material Refill – A2 Includes: 1 cartridge – 50ml, A2; 16 cartridge mixing tips, blue; Instructions
46995	3M™ Paradigm™ Temporization Material Refill – A3 Includes: 1 cartridge – 50ml, A3; 16 cartridge mixing tips, blue; Instructions

Mix Tips

71453	3M™ Garant™ Mixing Tips Includes: 50 – Blue
-------	---



3M™ Paradigm™
Temporization Material
Refill – A1

3M™ Paradigm™
Temporization Material
Refill – A2

3M™ Paradigm™
Temporization Material
Refill – A3

www.3M.com/dental



3M Oral Care
2510 Conway Avenue
St. Paul, MN 55144-1000 USA
Phone 1-800-634-2249
Web 3M.com/dental

3M Canada
Post Office Box 5757
London, Ontario N6A 4T1
Canada
Phone 1-888-363-3685

3M, Filtek, Garant, Paradigm and Protemp are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. Please recycle. Printed in USA. © 3M 2017. All rights reserved. 70-2013-0860-1