# 3M | Photac™ Fil Quick Aplicap™

ESPE

## (en) Light-Curing Glass Ionomer Dental Restorative – Radiopaque

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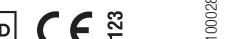
## Instructions for Use Gebrauchsinformation Notice d'utilisation

## Informazioni per l'uso Informatiōn de empleo Instrucciones de uso

## 3M Deutschland GmbH Health Care Business Car-Schurz-Str. 1 41453 Neuss – Germany 3M.com

3M ESPE Customer Care/Safety Data Sheet/Safety Information Sheet  
Information: U.S.A. 1-800-634-2249.3M, ESPE, Aplicap, CapMix, Ketac, Photac and Rotomix are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. © 2021, 3M. Unauthorized use prohibited. All rights reserved.  
3M, ESPE, Aplicap, CapMix, Ketac, Photac and Rotomix sont des marques de commerce de 3M ou 3M Deutschland GmbH. Utilisées sous licence au Canada. © 2021, 3M. Toute utilisation non autorisée est interdite. Tous droits réservés.

## (en) ENGLISH

**Product description:**  
Photac™ Fil Quick Aplicap™ is a light curing glass ionomer cement in a capsule according to ISO 9917-2, Class 2. Since it bonds chemically to enamel and dentin, it permits gentle preparation. It is available in eight shades. The material can be applied without cavity lining. It releases fluoride ions and is radiopaque.

The applicable quantity of a capsule amounts to at least 0.13 ml.

**Disposal:**

Dispose of the contents or container in accordance with the applicable regulations. Please pay special attention to the disposal of contaminated waste to avoid health risks because of improper handling.

The product may only be used when the product labeling is clearly readable. For details on all additionally mentioned products please refer to the corresponding instructions for use.

**Intended Purpose:**

Intended purpose: glass ionomer based dental restorative.

Intended users: educated dental professionals who have theoretical and practical knowledge on usage of dental products.

Clinical performance: restoration of oral esthetics and function or component thereof.

Can also be used for bonding capability if used as fissure sealant.

Patient target group: all patients requiring dental treatment, unless the patient's condition limits the use.

**Indications:**

- Filings in Black's class III and V
- Small fillings in Black's class I
- Wedge-shaped defects
- Filings in deciduous teeth
- Fissure sealing
- Temporary restorations in Black's class I and II

**Contraindications:**

The product is contraindicated for use with patients who have an known allergy to acrylate-based materials or contained allergens as listed in section "Precautionary Measures".

**Composition:**

The ingredients of each component of the product are indicated in decreasing order of their concentration:

Powder: Silicium dioxide powder, aromatic amine, Liquid: Silicium dioxide, methacrylic acid, methacrylate (HEMA), water, phosphorylated methacrylate (CapMix), urethane dimethacrylate (UDMA), stabilizer.

For more information please consult the Safety Data Sheet/Safety Information Sheet ([www.3M.com](http://www.3M.com)), or contact your 3M subsidiary.

**Precautionary Measures:**

**For Patients:**

Photac™ Fil Quick Aplicap liquid contains substances that may cause allergic reactions by skin contact in certain individuals. Avoid use of this product in patients with known acrylate allergies.

If prolonged contact with oral tissue occurs, flush with large amounts of water. If an allergic reaction occurs, remove the product, and discontinue future use; seek medical attention as needed.

**For Dental Personnel:**

Photac™ Fil Quick Aplicap liquid contains substances that may cause allergic reactions by skin contact in certain individuals. To reduce the risk of allergic reaction, avoid exposure to this product in patients with known acrylate allergies by skin contact. Use protective gloves and a no-touch technique is recommended.

Acrylates may penetrate commonly used gloves. If the liquid contacts the glove, remove and discard the gloves. Wash the hands immediately with soap and water and re-glove.

If an allergic reaction occurs, seek medical attention as needed.

Please refer to the section "Safety and clinical performance" for more information.

Basic UDI-DI: 0608223276102000000026DU

**Preparation:**

► Remove carious dentin only. Undercuts are not required.

► Use water spray to remove debris.

► Blow the cavity dry in only 2-3 short intervals with water and oil-free air or dry with cotton pellets. Do not over-dry! The cavity should be just so dry that the surface has a matt shiny appearance. Excessive drying can result in post-operative sensitivity after filling.

► Avoid any contamination by saliva.

**Pulp Protection:**

► To prevent pulpal irritation, the areas around the pulp should be covered by dabbing them with a hardsetting calcium hydroxide preparation.

**Conditioning:**

► No further preparatory procedures are required. In case of an unfavorable cavity geometry, the use of Ketac™ Conditioner can further improve the adhesive stability obtainable.

**Activation:**

► Place the Aplicap™ Activator on a sturdy surface and insert the Aplicap capsule into the activator. Firmly and completely to the stop and hold it for 2 to 4 seconds.

► Firmly and completely depressing the lever to the stop and holding it in this position is the only way to ensure that all of the liquid is pressed into the powder. If inadequate pressure is applied or the lever is not depressed completely, it is possible that too little liquid will be mixed with the powder, increasing the viscosity and changing the product characteristics.

**Mixing:**

► Mix the capsules at approx. 4,300 rpm, in a high frequency mixing device such as CapMix™ (or 15 sec) or the Rotomix™ rotating mixer (for 10 sec), and apply with the Aplicap™ Activator. Both the Aplicap Activator and the Aplicap are color-coded in red.

**Warning!**  
It is imperative to avoid waiting times between activation, mixing, and application as the onset of the curing prevents or renders the extension of the material difficult.

**Application:**

► Insert the capsule into the applier and secure it in place by pressing the lever once. Before applying the paste intradurally, the nozzle to the stop and press the applier several times until the mixed paste becomes visible in the intra-oral tip.

- Use Aplicap™ applier to apply layers of up to 2 mm maximum. Avoid all types of contamination during application.
- Use halogen or LED light for curing.
- Place the light source as closely as possible to the filling.
- The maximum layer thickness (without intermediate curing) may not exceed 2 mm.
- In case of cavities of more than 2 mm in depth, several layers have to be applied.
- The self-curing mechanism ensures that the material also cures in areas shielded from the light. In order to attain its optimum physical properties and to ensure immediate preparation, the material has to be light-cured carefully.
- It is possible to add layers on material already cured if the surface is clean and dry.
- The material adheres to metal instruments and thus has to be washed off in cold water before it sets.

**Times:**  
The following times apply at a room temperature of 23°C / 73°F:  
Activation 0:02  
Mixing in Rotomix 0:10  
Mixing in a high-frequency mixer, such as CapMix 0:15  
Manipulation from the beginning of the mixture 2:00  
Light curing 0:20 (DFO 0:40)

At room temperatures of over 23°C / 73°F the curing is accelerated. Shorter mixing times prolong the manipulation and curing time, whereas longer mixing times shorten the manipulation and curing time. The mixing time in high-frequency mixers can be selected at between 8 and 15 seconds, and the mixing time to set between 7 and 12 seconds. If the manipulation time is exceeded, bonding to the enamel and dentin is lost.

**Finishing:**

Finishing can be started directly after light curing.  
► Use a sharp instrument to remove excess material.

► Use rotary instruments (such as polishing discs or silicone polishers) to finish under cold water.

**Cleaning the Aplicap and Maxicap Activator and Aplicap**

Please refer to the patient instructions for Use for information about cleaning and disinfecting the Aplicap/Maxicap Activator and Aplicap.

**Notes:**

- Aplicap capsules are single-use devices.
- Please keep the empty Aplicap capsule in the original blister to guarantee the traceability of the batch (LOT).
- Humidity in the cavity prevents bonding significantly.
- Photac™ Fil Quick Aplicap polymerizes in daylight or artificial lighting. Therefore, the material should not be dispensed from the capsule until immediately prior to manipulation. **The manipulation time is reduced drastically in case of application under the operating light!**

**Disposal:**

Dispose of the contents or container in accordance with the applicable regulations. Please pay special attention to the disposal of contaminated waste to avoid health risks because of improper handling.

The product may only be used when the product labeling is clearly readable.

For details on all additionally mentioned products please refer to the corresponding instructions for use.

**Intended Purpose:**

Intended users: educated dental professionals who have theoretical and practical knowledge on usage of dental products.

Clinical performance: restoration of oral esthetics and function or component thereof.

Can also be used for bonding capability if used as fissure sealant.

Patient target group: all patients requiring dental treatment, unless the patient's condition limits the use.

**Indications:**

- Fillings in Black's class III and V
- Small fillings in Black's class I
- Wedge-shaped defects
- Fillings in deciduous teeth
- Fissure sealing
- Temporary restorations in Black's class I and II

**Contraindications:**

The product is contraindicated for use with patients who have an known allergy to acrylate-based materials or contained allergens as listed in section "Precautionary Measures".

**Composition:**

The ingredients of each component of the product are indicated in decreasing order of their concentration:

Powder: Silicium dioxide powder, aromatic amine, Liquid: Silicium dioxide, methacrylic acid, methacrylate (HEMA), water, phosphorylated methacrylate (CapMix), urethane dimethacrylate (UDMA), stabilizer.

For more information please consult the Safety Data Sheet/Safety Information Sheet ([www.3M.com](http://www.3M.com)), or contact your 3M subsidiary.

**Precautionary Measures:**

**For Patients:**

Photac™ Fil Quick Aplicap liquid contains substances that may cause allergic reactions by skin contact in certain individuals. Avoid use of this product in patients with known acrylate allergies.

If prolonged contact with oral tissue occurs, flush with large amounts of water. If an allergic reaction occurs, seek medical attention as needed.

Please refer to the section "Safety and clinical performance" for more information.

Basic UDI-DI: 0608223276102000000026DU

**Preparation:**

► Remove carious dentin only. Undercuts are not required.

► Use water spray to remove debris.

► Blow the cavity dry in only 2-3 short intervals with water and oil-free air or dry with cotton pellets. Do not over-dry! The cavity should be just so dry that the surface has a matt shiny appearance. Excessive drying can result in post-operative sensitivity after filling.

► Avoid any contamination by saliva.

**Pulp Protection:**

► To prevent pulpal irritation, the areas around the pulp should be covered by dabbing them with a hardsetting calcium hydroxide preparation.

**Conditioning:**

► No further preparatory procedures are required. In case of an unfavorable cavity geometry, the use of Ketac™ Conditioner can further improve the adhesive stability obtainable.

**Activation:**

► Place the Aplicap™ Activator on a sturdy surface and insert the Aplicap capsule into the activator. Firmly and completely to the stop and hold it for 2 to 4 seconds.

► Firmly and completely depressing the lever to the stop and holding it in this position is the only way to ensure that all of the liquid is pressed into the powder. If inadequate pressure is applied or the lever is not depressed completely, it is possible that too little liquid will be mixed with the powder, increasing the viscosity and changing the product characteristics.

**Mixing:**

► Mix the capsules at approx. 4,300 rpm, in a high frequency mixing device such as CapMix™ (or 15 sec) or the Rotomix™ rotating mixer (for 10 sec), and apply with the Aplicap™ Activator. Both the Aplicap Activator and the Aplicap are color-coded in red.

**Warning!**

It is imperative to avoid waiting times between activation, mixing, and application as the onset of the curing prevents or renders the extension of the material difficult.

**Application:**

► Insert the capsule into the applier and secure it in place by pressing the lever once. Before applying the paste intradurally, the nozzle to the stop and press the applier several times until the mixed paste becomes visible in the intra-oral tip.

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3M, ESPE, Aplicap, CapMix, Ketac, Photac and RotoMix are trademarks of 3M or 3M Deutschland GmbH.