# **3M** ESPE

### RelyX<sup>™</sup> Luting Plus Automix Resin Modified Glass lonomer Cement



- · Porcelain fused to metal (PFM) crowns and bridges
- · Metal crowns, bridges, inlays, onlays
- Crowns made with all-alumina or all-zirconia cores such as 3M<sup>™</sup> ESPE<sup>™</sup> Lava<sup>™</sup> or Procera<sup>®</sup> AllCeram
- · Prefabricated or cast endodontic posts
- · Orthodontic bands and appliances
- Porcelain fused to metal (PFM), metal, all-alumina or all-zirconia core restorations on implant abutments



#### Tooth Preparation

1. Remove provisional restoration and all temporary cement.



2. Clean tooth with oil-free pumice paste.



### RelyX<sup>™</sup> Luting Plus Automix

**Resin Modified Glass Ionomer Cement** 






#### Tooth Preparation (cont.)

3. Rinse and lightly dry. Leave tooth surface moist.



4. Do not over dry the tooth. Over drying can lead to post operative sensitivity.

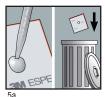

# Working time of mixed cement is 1.5 minutes 5a. Squeeze out and discard a peppercorn-size

quantity of mixed paste.



5b. Apply a thin layer of cement to the inside surface of the restoration.

5b




## RelyX<sup>™</sup> Luting Plus Automix

Resin Modified Glass lonomer Cement






# Placement of Restoration6. Seat restoration with firm pressure.

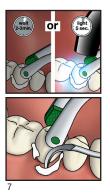

 Excess cement can be removed after 2 minutes in the mouth or tack light curing 5 seconds per surface.

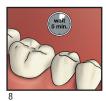

8. Set time is 5 minutes after placement in the mouth.



### RelyX<sup>™</sup> Luting Plus Automix Resin Modified Glass Ionomer Cement





#### Placement of Restoration (cont.)

9. Finish restoration and adjust occlusion.

For best results, please keep the syringe tip clean. For storage, discard the used mixing tip and replace original sealing cap onto the the syringe. Or, leave the used mixing tip on the syringe as a cap until the next application. **NOTE:** Do not store syringe with a new, unused mixing tip attached onto syringe. An unused mixing tip does not properly seal the syringe and may affect product shelf life.



#### 3M ESPE

Dental Products, St. Paul, MN 55144-1000 USA • www.3MESPE.com **3M ESPE Customer Care/MSDS Information: U.S.A. 1-800-634-2249 and Canada 1-888-363-3685** 3M, ESPE, Lava and RelyX are trademarks of 3M or 3M ESPE AG, used

under license in Canada. © 3M 2011. All rights reserved. Procera is a trademark of Nobel Biocare.

3M, ESPE, Lava et RelyX sont des marques de commerce de 3M ou de 3M ESPE AG, utilisées sous licence au Canada. © 3M 2011. Tous droits réservés. Procera est une marque de commerce de Nobel Biocare.

