

# PULI-JET GENTLE

*Effective disinfection for your suction system.*

Daily disinfection of your suction system is vital to keep your suction lines clean and maintain your equipment and suction unit. Puli-jet GENTLE's strength as a disinfectant means that it kills bacteria, helping to maintain a safer work environment.

## 1 bottle makes 30 x 2.5 litre solutions

The concentration of 30ml of Puli-jet GENTLE to 2.5 litres of warm water, means that the 1 litre bottle of Puli-jet GENTLE concentrate makes 30 x 2.5 litre solutions.

Puli-jet GENTLE is suitable for the following dental suction systems:

- Semi-wet / semi-dry systems (i.e. Turbo SMART, Turbo Jet)
- Dry suction systems (i.e. with separator in the surgery)
- Wet suction systems (i.e. Cattani wet pumps)

Puli-jet GENTLE has a simple and accurate dispensing cup, making it easier to measure the solution and to pour into the Pulse Cleaner.



## DIRECTIONS FOR USE:

Puli-jet GENTLE is easy to use, however you must follow these instructions to ensure that it is safe and effective.

**Personal Protective Equipment (PPE) including protective gloves and eyewear must be worn whenever handling Puli-jet GENTLE. Puli-jet GENTLE must be diluted in a well ventilated area.**

At the end of the working day follow these simple steps:



**1.** Fill the Pulse Cleaner with 2.5 litres of warm water (around 40 degrees) – each line on the inside of the Pulse Cleaner represents 1 litre, so fill it up to halfway between the second and third line.



**2.** Place the Puli-jet GENTLE on a level and steady surface.



**3.** Slightly unscrew the red cap (i.e. 1 – 2 revolutions), but do not completely unscrew it.



**4.** Squeeze the bottle softly to force solution into the measuring dispenser.



**5.** Continue to squeeze until the solution reaches the 30ml mark.



**6.** If too much solution has entered the dispensing cup, it can be poured back into the main bottle by tipping the dispensing cup towards the small tube on the side of the bottle.



**7.** Unscrew and remove the red cap. Pour the solution from the dispensing cup into the full Pulse Cleaner. Take care to not allow more solution from the main bottle flow into the dispensing cup.



**8.** With a disposable cloth, wipe any drips of solution from the bottle (and then dispose of the cloth).

### Important note:

Excessive fumes of the product can be generated by:

1. Diluting the Puli-jet GENTLE with water in the wrong order (you must first fill the Pulse Cleaner with water and then add the Puli-jet GENTLE).
2. Using excessively hot water. We recommend water around 40 degrees, however cold water can also be used.
3. Puli-jet GENTLE should never be pre or semi-diluted. It must be diluted (according to the instructions) and then used straight away.

### Please note:

Puli-jet GENTLE leaves a residue on the inside of pipes and suction tubes in order to stop growths on these surfaces. This is the reason that you may sometimes find the surface of a tube to be 'oily' – this is completely normal and the product is designed to do this.