

Surface Disinfection with Clinicare

Clinicare's 2 step process

STEP 1: Decontaminate



A. Dispense **Clinicare** towelette



B. Wipe surface to remove all visible contaminants



C. Dispose of used towelette

STEP 2: Disinfect



A. Dispense another **Clinicare** towelette



B. Wipe decontaminated surface in S pattern to ensure complete coverage



C. Dispose of used towelette



Always follow personal protection standards.
 CHART AVAILABLE FOR DOWNLOAD AT
www.dentalife.com.au
 or contact info@dentalife.com.au

www.dentalife.com.au