

- attention.
  3. Take care to avoid ingestion of the material.
  4. Do not mix with other similar products.
  5. Dispensing tips are for single use only. To prevent cross contamination between patients, do not reuse the dispensing tip. The dispensing tip cannot be sterilized in an autoclave or chemiclave.
  6. Wear plastic or rubber gloves during operation to avoid direct contact with air inhibited resin layers in order to prevent possible sensitivity.
  7. Wear protective eye glasses during light curing.
  8. When polishing the polymerized material, use a dust collector and wear a dust mask to avoid inhalation of cutting dust.

- When polishing the polymerized material, use a dust collector and wear a dust mask to avoid inhalation of cutting dust.
   Avoid getting material on clothing.
   In case of contact with unintended areas of tooth or prosthetic appliances, remove with instrument, sponge or cotton pellet before light curing.
   Do not use SOLARE Flo in combination with eugenol containing materials as eugenol may hinder SOLARE Flo from setting.
   Do not wipe the dispensing tip with excessive alcohol as this may cause contamination of the paste.
   Dispose of all wastes according to local regulations.
   In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

- 15. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

available at: http://www.gceurope.com http://www.gcamerica.com http://www.gcasiadental.com http://www.gcindiadental.com They can also be obtained from your supplier.

## CLEANING AND DISINFECTING:

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between Patients this device requires mid-level disinfection. Immediately derives inspect device and label for deterioration. Discard device if damaged. DO NOT IMMERSE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Undesired effects- Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following lister.

bink: https://ec.europa.eu/growth/sectors/medical-devices/contacts\_en as well as to our internal vigilance system: vigilance@gc.dental In this way you will contribute to improve the safety of this product.

Last revised: 09/2019





## ,'GC,'

MANUFACTURED by GC DENTAL PRODUCTS CORP.

DISTRIBUTED by sunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

## EU: GC EUROPE N.V.

Researchark Haardot-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium TEL: +32 16 74 10 00 GC AMERICA INC. 3737 West 127th Street, Alsip, IL 60803 U.S.A. TEL: +1-708-597-0900 www.gcamerica.com

Www.gcamerica.com GC SOUTH AMERICA 11-0-4ora 399. Santana - 02022-051 São Paulo, SP - Brasil Rua Heliodora, 399, Santana - 02022-051 Sao Paulo, SP - Brasil Tel: +55-11-2925.0965 CNPJ: 08.279.0999/0001-61 Responsável Técnica Mayara de Santis Ribeiro CRO/SP: 105.982

GC ASIA DENTAL PTE. LTD. 5 Tampines Central 1, #06-01 Tampines Plaza, Singapore 529541 TEL: 465 6546 7588

GC AUSTRALASIA DENTAL PTY. LTD. 1753 Botany Rd, Banksmeadow, NSW 2019, Australia TEL: +61 2 9301 8200

PRINTED IN JAPAN

13009757 N 040792DK