

Surface Disinfection

Spray • Wipe • Spray

CLEAN WITH CAVICIDE BEFORE DISINFECTING

1. Spray **CaviCide**® directly onto surface to be cleaned.



2. Wipe with paper towel to remove debris and bioburden.



3. Discard used paper towel.



DISINFECT AFTER CLEANING

1. Spray **CaviCide** directly onto a pre-cleaned surface, thoroughly wetting area. Allow three minutes for maximum disinfection.



Follow product label instructions for appropriate contact times.

Wipe • Discard • Wipe

CLEAN WITH CAVIWIPES BEFORE DISINFECTING

1. Unfold first **CaviWipes**® towelette.



2. Wipe surface or items with towelette to remove debris and bioburden.



3. Discard used towelette.



DISINFECT AFTER CLEANING

1. Wipe pre-cleaned surface with second **CaviWipes** towelette to disinfect.



Follow product label instructions for appropriate contact times.

Use **CaviCide** or **CaviWipes** to clean, disinfect and decontaminate hard, non-porous surfaces or items to prevent cross contamination, such as:

- Procedure lights
- Dental operator and laboratory counter tops/cabinets
- Telephones and office machines

Use **CaviCide** for:

- Potentially contaminated non-critical instruments or equipment
- Dental items prior to sterilization

Wear appropriate personal protective equipment when handling chemicals and contaminated items.

PART NO.	PRODUCT DESCRIPTION	PACKAGING
13-1024	CaviCide 24 ounce spray	12 bottles/case
13-1000	CaviCide gallon	4 bottles/case
13-1025	CaviCide 2.5 gallon	2 bottles/case
13-1008	CaviCide 8 ounce spray	12 bottles/case
13-1100	CaviWipes – 160 wipes/can	12 canisters/case
13-1150	CaviWipes XL – 66 wipes/can	12 canisters/case
13-1155	CaviWipes XL – individual towelettes	6 boxes of 50/case
13-1224	CaviWipes FlatPack – 45 wipes/pack	20 packs/case



For more information, contact us at 800.841.1428 or visit KerrTotalCare.com



77-1040-2