Kerr™

X-Ray Holders
X-Ray Centring Devices

For Film, Sensor and Phosphor Plate Holders

IFUs

Together, we're more.™

Instructions for X-Ray Film Holders and Sensor Holders

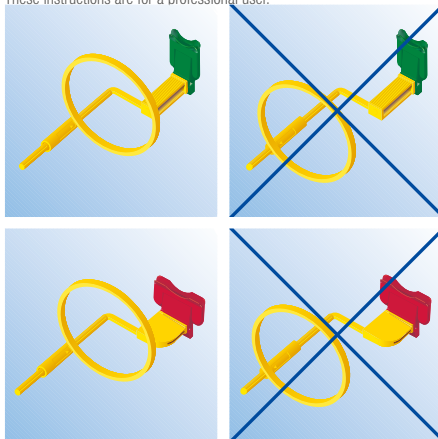
1. Product Brand Name

X-Ray Film and Phosphor Plate Holders and X-Ray Sensor Holders.

2. Indications for Use

X-Ray Film and Phosphor Plate Holders and X-Ray Sensor Holders are made of plastic material and intended to hold X-ray films, X-ray phosphor plates or X-ray digital sensors, respectively. This is to facilitate the adjustment of the X-ray tube positioning in order to get the radiography right the first time, reducing the exposure of the patient to radiation as much as possible.

These instructions are for a professional user.

3. Contraindications

None known.

4. Warnings

X-Ray Film, Phosphor Plate Holders and Sensor Holders are suitable for re-use. It is therefore essential that the devices are handled with special care during use and reprocessing, and the steriliser is subject to a preventive maintenance plan according to the steam steriliser's manufacturer. We strongly recommend the use of equipment complying with EN 285 or EN 13060. Sterilisation with alternative methods such as radiation, hot

air, formaldehyde and ethylene oxide or plasma is not advised, since the products have not been tested for compatibility with these processes. Shelf-life on the product refers to the product stored in its box. During use you need to handle the device carefully, avoid dropping, pulling or over-twisting the device, as this could lead to damaging of the material.

5. Precautions

All auxiliary devices used to clean the products have to be cleaned and disinfected at the end of the procedure to avoid / minimise the risk of cross-contamination.

6. Adverse Reactions

None known.

7. Instructions

The X-Ray Film and Phosphor Plate Holders and X-Ray Sensor Holders can be used without or in conjunction with the X-Ray Centring Devices System as per info cards:

- Endo-Bite Senso Anterior and Endo-Bite Senso Posterior (p. 132-135)
- Super-Bite Anterior and Super-Bite Posterior (p. 116-119)
- Paro-Bite and Kwik-Bite (p. 112-115)
- Endo-Bite Anterior and Endo-Bite Posterior (p. 120-123)
- Super-Bite Senso Anterior and Super-Bite Senso Posterior (p. 128-131)
- Kwik-Bite Senso (p. 124-127)

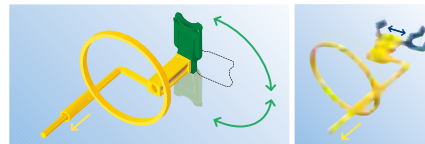
For proper disposal always observe national laws.

8. Cleaning, Disinfection and Sterilisation Instructions

To guarantee successful re-processing, work thoroughly and with great care. Thorough cleaning and disinfection are essential pre-requisites for effective sterilisation.


Cleaning

1. Remove centring ring from the holder stem.
2. Twist holder plate into the loose intermediate position (Super-Bite, Endo-Bite) or open position (Kwik-Bite Senso).

3. Remove visible debris or organic matter from the instruments and surfaces with a disposable wipe or surface brush. Never use metal brushes or wire wool. This is to be done by scrubbing with warm tap water and soap or detergent.

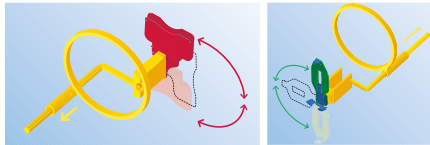4. Inspect instruments for debris. Repeat cleaning step 3 if there is still debris.
5. Immediately soak in an appropriate disinfecting solution for plastic instruments following the manufacturer's instructions.
6. As an alternative to point 5, an ultrasonic cleaner with an appropriate cleaning and disinfecting agent can be used in accordance with the manufacturer's instructions.
7. Rinse all parts with warm tap water.
8. Dry instruments, then place in an autoclave tray as per steriliser instruction for use, in a sterilisation pouch meeting the following requirements:
 - Compliance with ISO 11607 and steam sensitive indicator to identify treated product from those not treated
 - Suitable for steam sterilisation
 - Check the pouch expiry date to ensure it can still be used (Sterilization Instructions step 2, below).
 Packaging protects the sterilised goods from micro bacterial recontamination during handling and storage if not damaged
9. As an alternative to points 3 to 8, thermo disinfection at 93°C with an appropriate cleaning agent can be used in accordance with the manufacturer's instructions.
10. Sterilisation:
 - Ensure centring ring is separated from the holder stem, and holder plates are twisted to the intermediate position.

- Place parts separately in either the steriliser tray or in a pouch. Sterilise in a steam autoclave using distilled water at 134°C (273°F) for 3 minutes.
 - Do not use chemical autoclave or cold sterilisation solutions.
 - Avoid direct contact with the metallic part of the autoclave; the temperature there can exceed the pre-set temperature, which can lead to material deformation.
 - When the product is taken out of the autoclave, check the package for damage. In the case of broken packaging do not use the product; the sterilisation could have been compromised. In this case the sterilisation procedure must be repeated.
11. Inspect the product after reprocessing; defects devices should be immediately discarded. These defect include:
 - Plastic deformation or cracks
 - Bent part of the device

- Untwisted threads
 - Brittle material.
12. Store the product in a dry and dust-free area.
 13. Before use, twist the holder back to the normal locked position (Super-Bite, Endo-Bite).
 14. The devices can be re-used with proper care and if they are not damaged. Over time and following several re-use/treatment plastic can become brittle and/or discoloured. At that point the device **MUST** be discarded. The examination can be with the naked eye or the aid of a magnifier under normal lighting conditions.

9. Disclaimer

The instructions provided above have been validated by KerrHawe SA as being CAPABLE of preparing a medical device for re-use. It remains the responsibility of the processor/user to ensure that the reprocessing actually performed using equipment, materials and personnel is performed as per the above-provided instructions in order to achieve the desired result. Any deviation by the processor/user from the above-provided instructions will be at your own risk; KerrHawe SA will not be able to address for any products that have refund or exchange under guarantee of any products which have not been reprocessed in accordance with the above instructions.

en

de

fr

it

es

pt

nl

dk

fi

se

no

el

pl

hu

cz

sk

sl

ro

ru

bg

lt

lv

et

hr

ar

tk

zh

Instructions for X-Ray Centring Devices

1. Product Brand Name

X-Ray Centring Devices System

2. Indications for Use

X-Ray Centring Devices are intended to facilitate the exact positioning of rectangular or other special X-Ray tubes when using the X-Ray Film and Phosphor Plate Holders or X-Ray Sensor Holders. These instructions are for a professional user.

3. Contraindications

None known.

4. Warnings

The X-Ray Centring Devices System is suitable for re-use; it is therefore essential that the device is handled with care during use and reprocessing. Shelf-life on the product refers to the product stored in its box. During use you need to handle the device carefully; avoid dropping, pulling or over-twisting the device, as this could lead to a damage of the material.

5. Precautions

All auxiliary devices used to clean the products have to be cleaned and disinfected at the end of the procedure to avoid / minimise risk of cross-contamination.

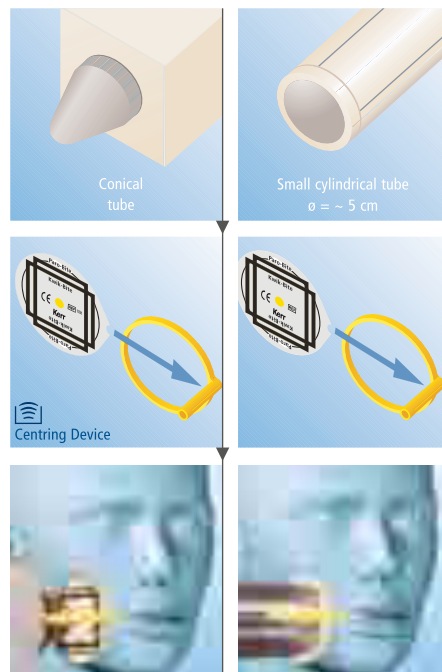
6. Adverse Reactions

None known.

7. Instructions

1. Remove Centring ring from the X-Ray Holder's stem.
2. Assemble the X-Ray Centring Devices on the X-Ray Film and Phosphor Plate Holders or X-Ray Sensor Holder's Centring ring. The X-Ray Centring Devices stay in place on the ring by a slight press-fit.
3. Mount the Centring ring on the X-Ray Holder's stem.

For proper disposal always observe national laws

8. Cleaning, Disinfection

To guarantee successful re-processing, work thoroughly and with great care. Cleaning and disinfecting:

1. Remove the X-Ray Centring Devices' on the X-Ray Film and Phosphor Plate Holders or X-Ray Sensor Holders' Centring ring.
2. Remove visible debris or organic matter from the surfaces with a disposable wipe or by rinsing with warm tap water.
3. Inspect the X-Ray Centring Devices for debris. Repeat cleaning step 3 if there is still debris.
4. Immediately soak in an appropriate disinfecting solution for plastic instruments following the disinfecting solution manufacturer's instructions.
5. As an alternative to point 5, an ultrasonic cleaner with an appropriate cleaning and disinfecting agent can be used in accordance with the manufacturer's instructions.
6. Rinse all parts with warm tap water.
7. Dry the X-Ray Centring Devices at room temperature.

The devices can be re-used with proper care and if they are not damaged. Over time and following several re-use/treatment plastic can become brittle and/or discoloured. At that point the device MUST be discarded. The examination can be with the naked eye or the aid of a magnifier under normal lighting conditions.

9. Disclaimer

The instructions provided above have been validated by KerrHawe SA as being CAPABLE of preparing a medical device for re-use. It remains the responsibility of the processor/user to ensure that the reprocessing actually performed using equipment, materials and personnel is performed as per the above-provided instructions in order to achieve the desired result. Any deviation by the processor/user from the above-provided instructions will be at your own risk; KerrHawe SA will not be able to address for any products that have refund or exchange under guarantee of any products which have not been reprocessed in accordance with the above instructions.

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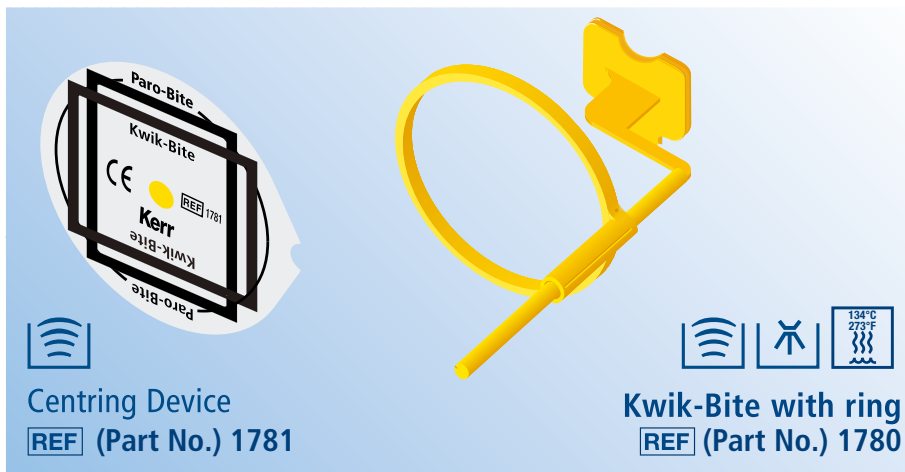
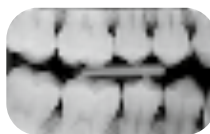
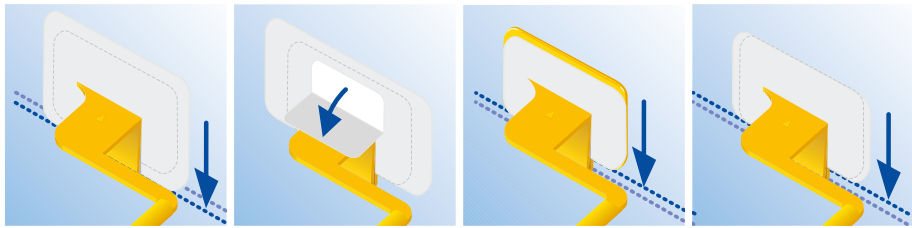
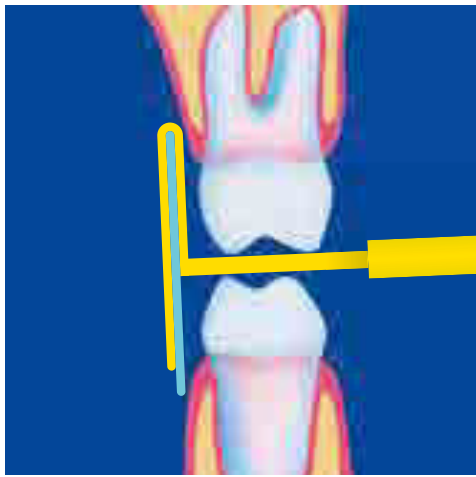
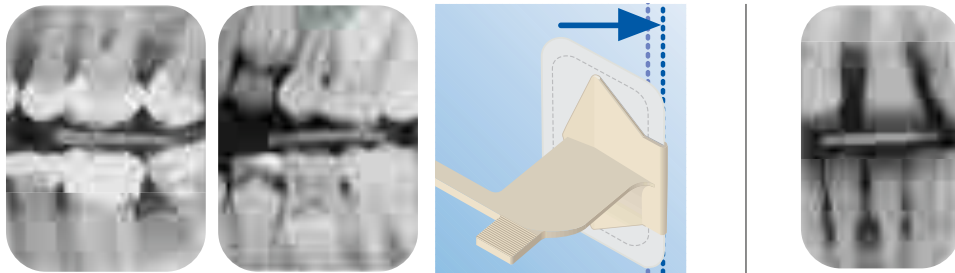
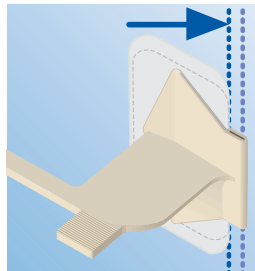
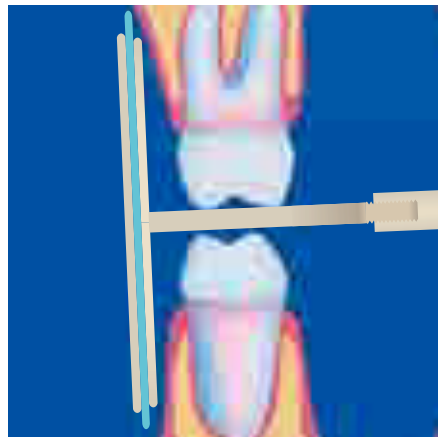
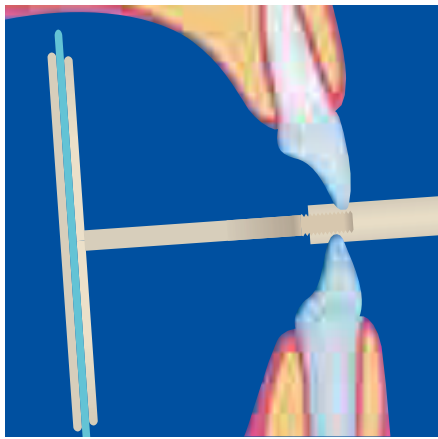
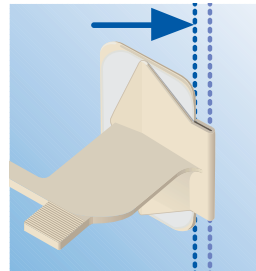
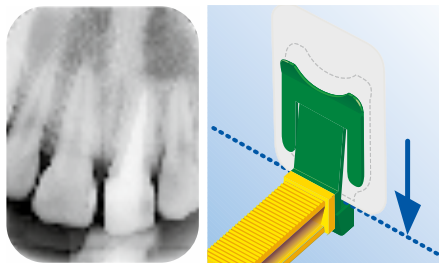
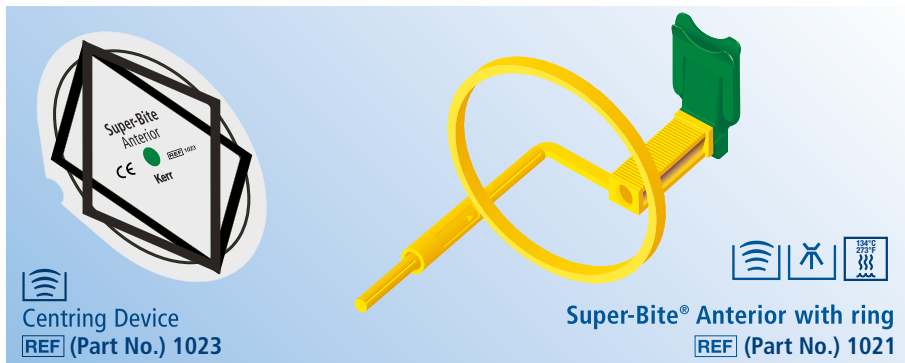
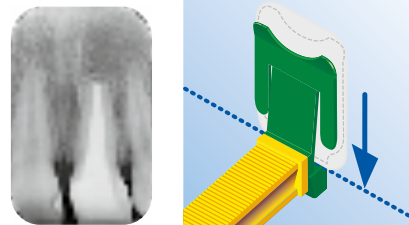
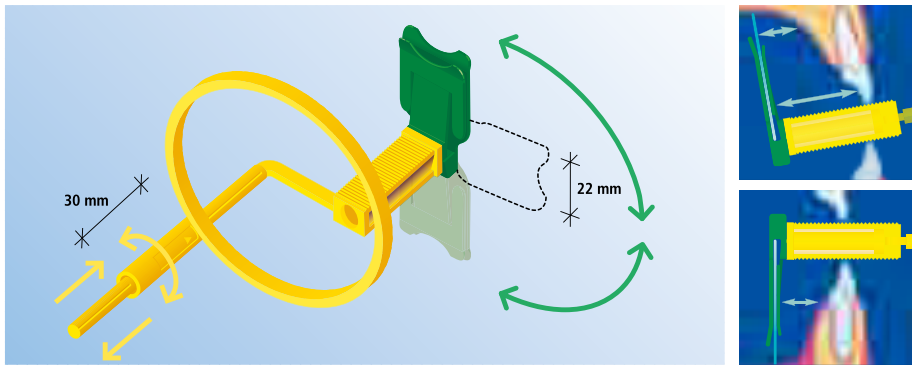
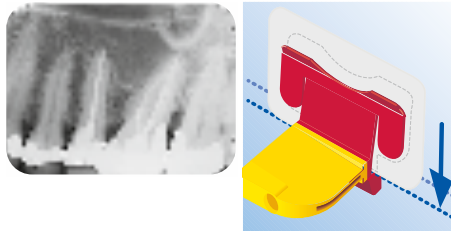
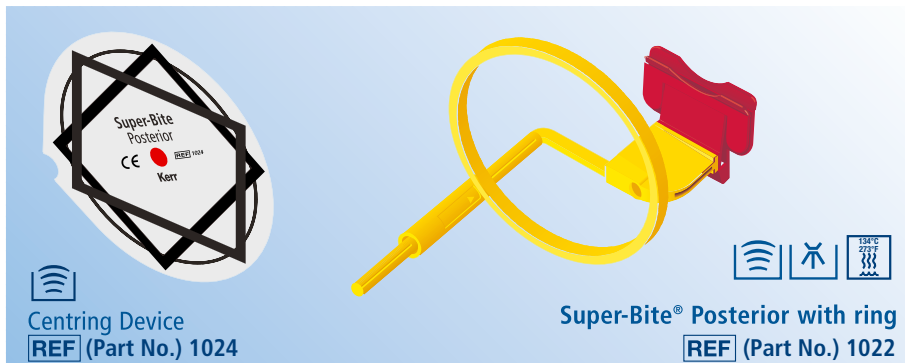
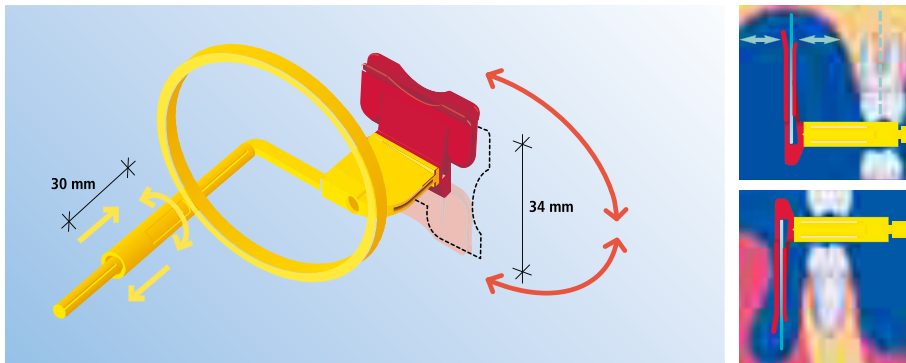
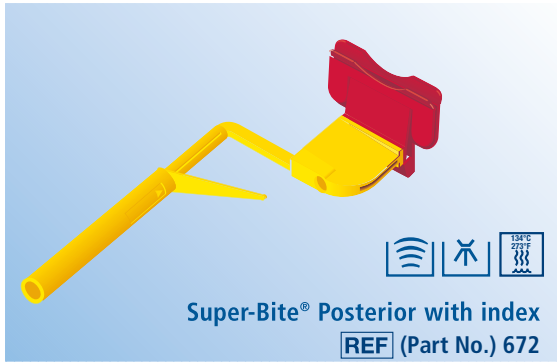
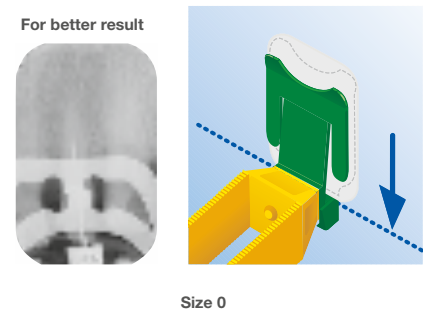
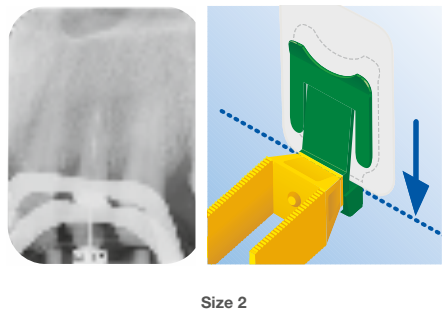
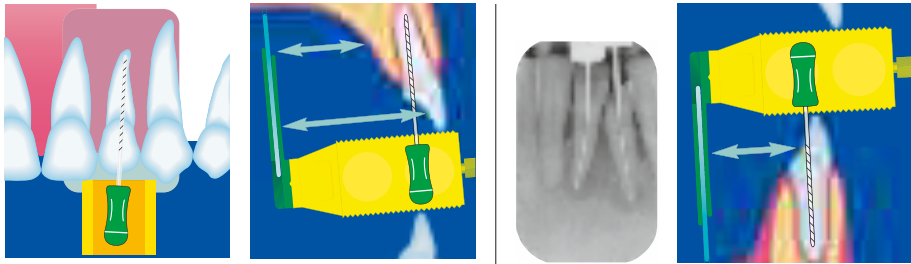
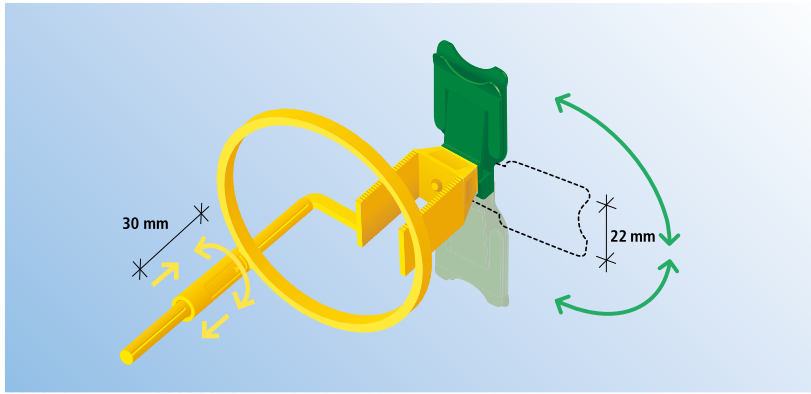
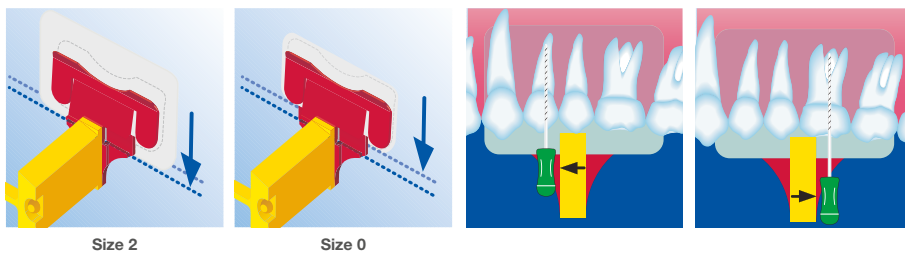
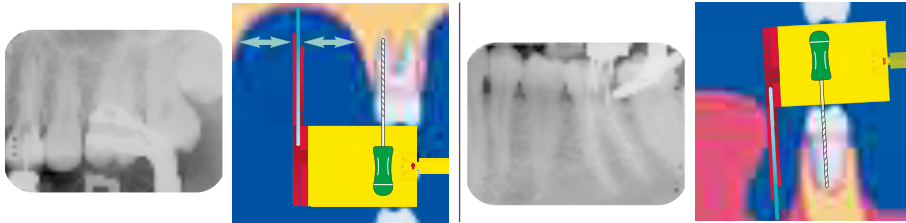
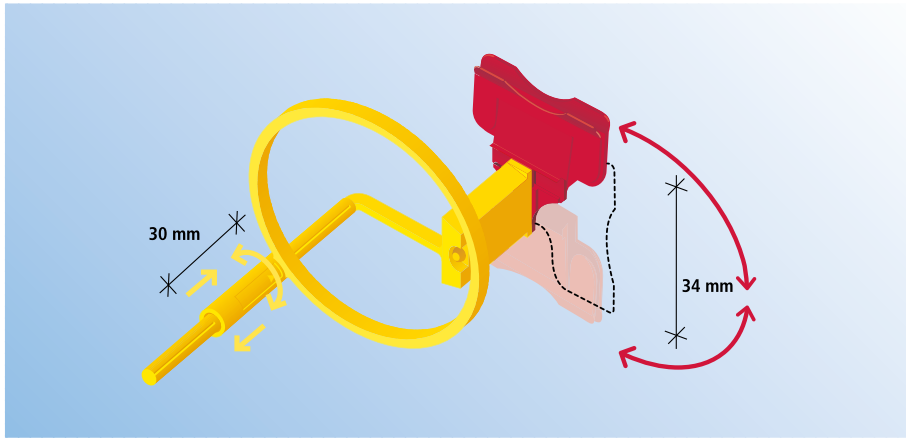
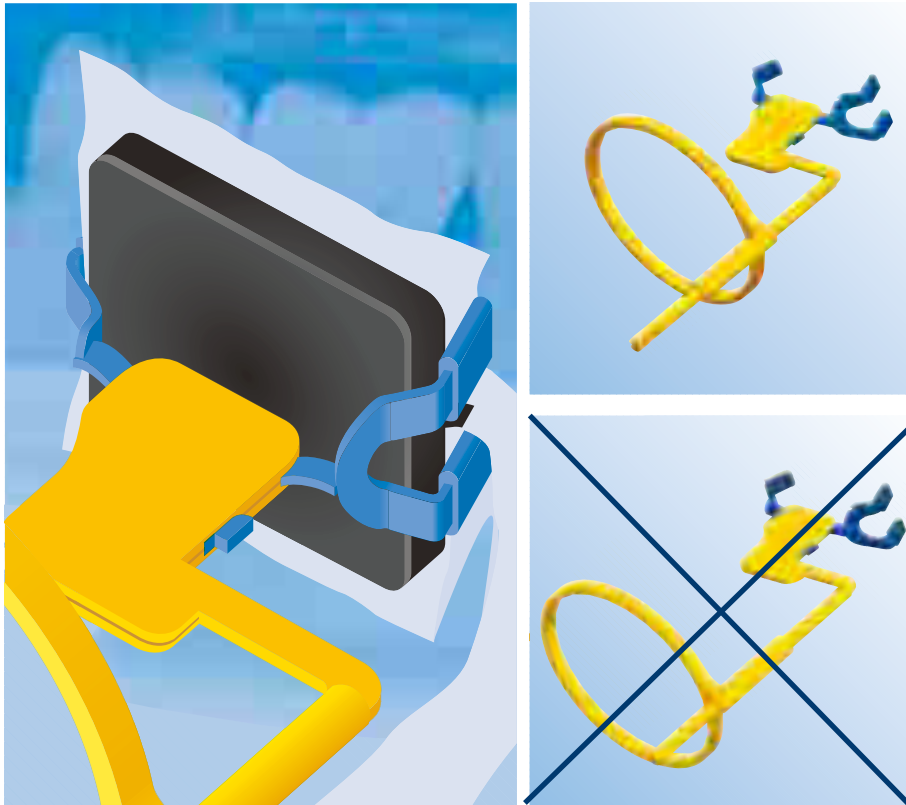
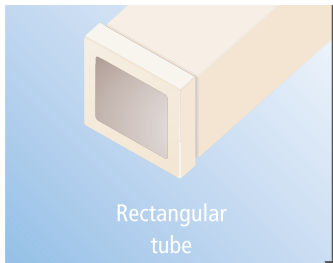
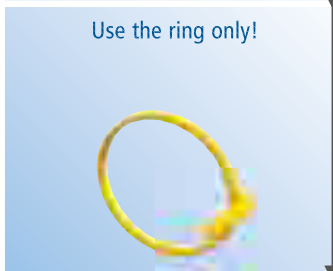
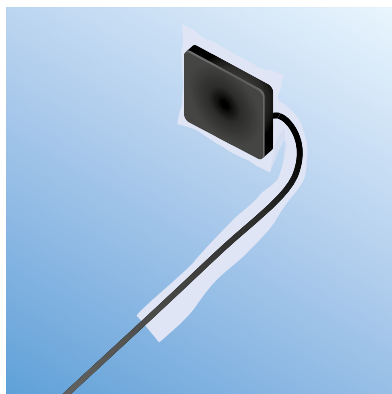
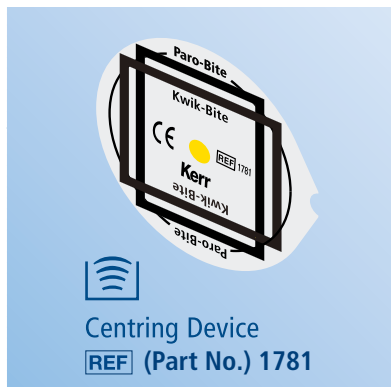
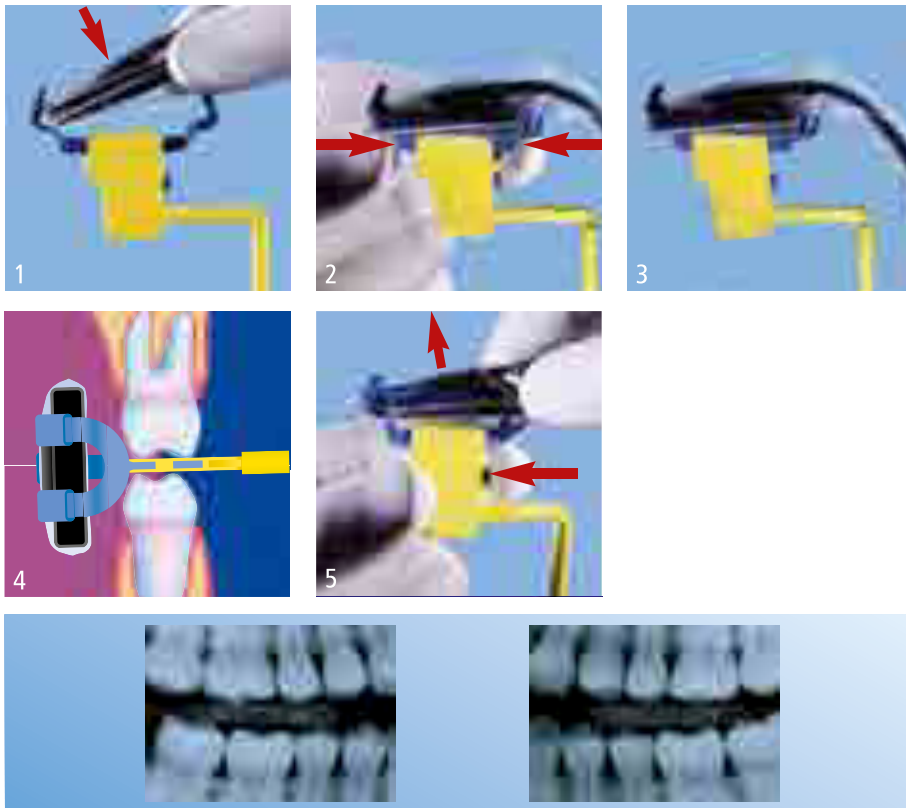
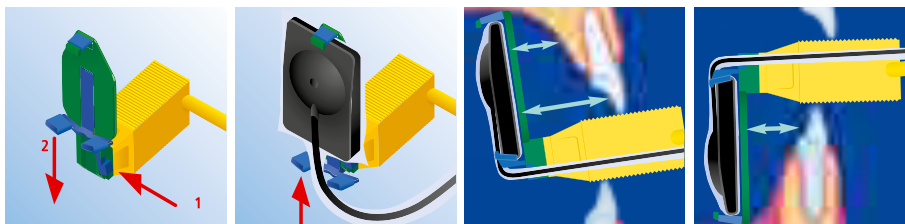
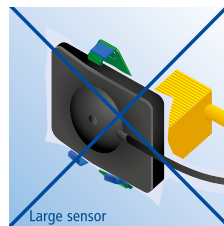
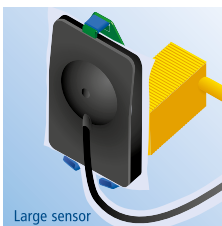
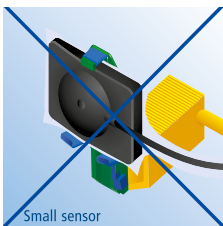
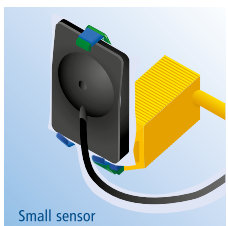
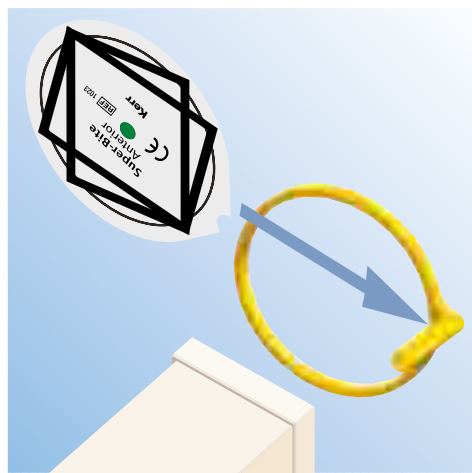
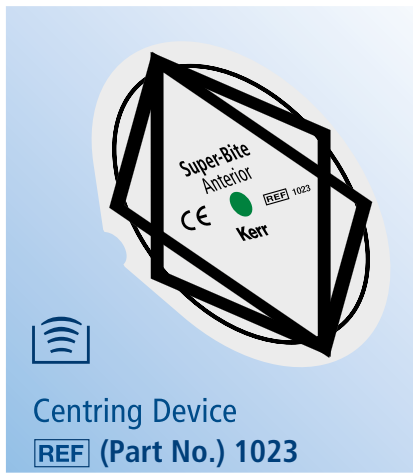
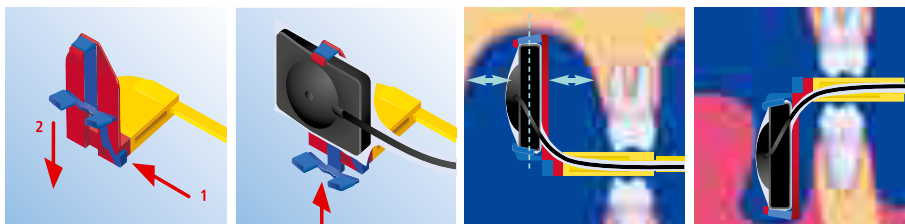
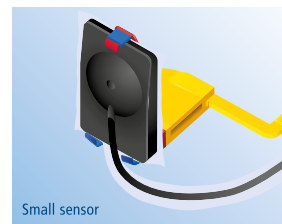
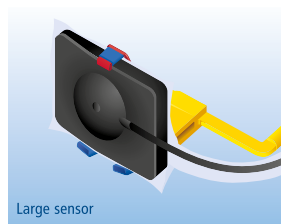
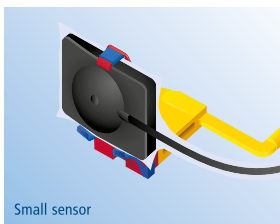
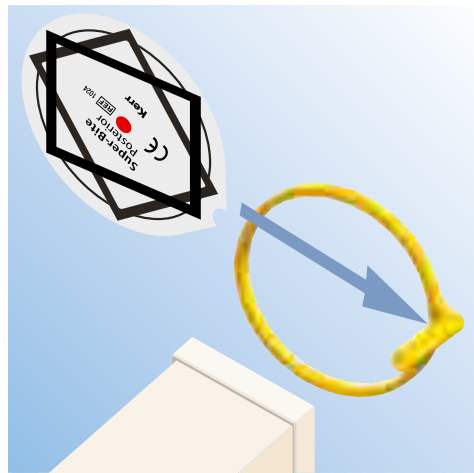
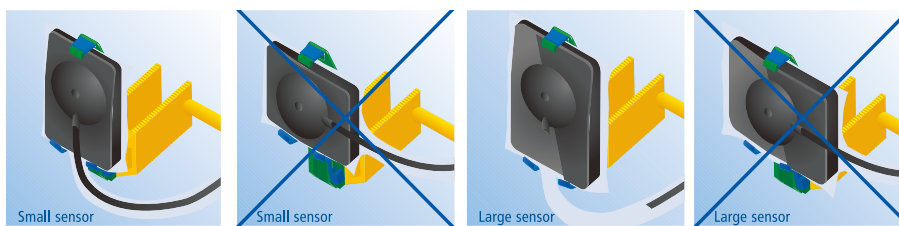
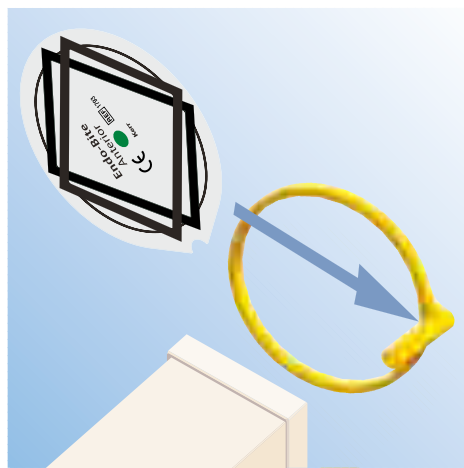
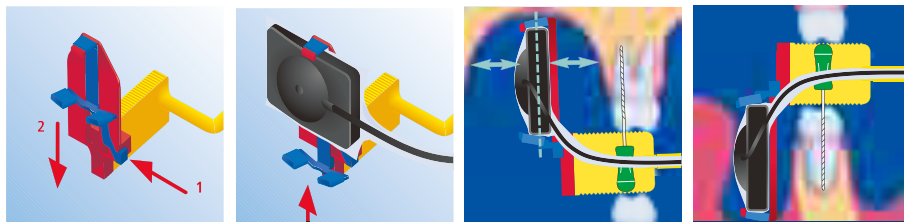
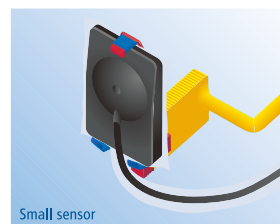
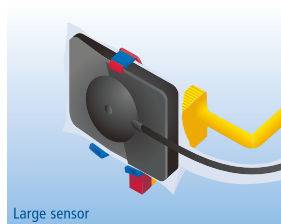
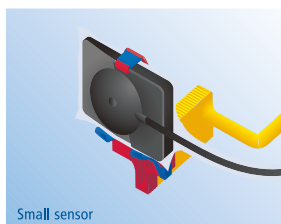
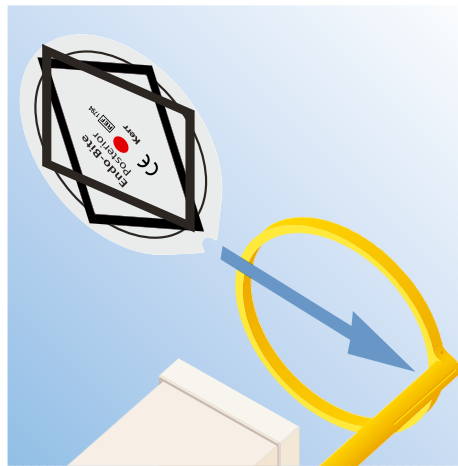
Codes

Together, we're more.™

X-Ray Film and Phosphor Plate Holders

Part No.	Description		
1720	Testset X-Ray Film and Phosphor Plate Holders		
Contents: 6 X-ray holders with ring (1 Kwik-Bite, 1 Super-Bite Anterior, 1 Super-Bite Posterior, 1 Paro-Bite, 1 Endo-Bite Anterior, 1 Endo-Bite Posterior), 5 Centring Devices			
Part No.	Description	Part No.	Description
1780	Kwik-Bite with Ring	670	Super-Bite with Index Assorted Kit
Contents: 5 Kwik-Bite with ring, 1 Centring Device		Contents: 4 Super-Bite Anterior, 4 Super-Bite Posterior, 8 Index	
Part No.	Description	Part No.	Description
270	Kwik-Bite with Index	671	Super-Bite with Index Anterior
Contents: 15 Kwik-Bite with index		Contents: 8 Super-Bite Anterior, 8 Index	
Part No.	Description	Part No.	Description
1770	Paro-Bite with Ring	672	Super-Bite with Index Posterior
Contents: 5 Paro-Bite with Ring, 1 Centring Device		Contents: 8 Super-Bite Posterior , 8 Index	
Part No.	Description	Part No.	Description
1020	Super-Bite with Ring	1790	Endo-Bite with Ring Assorted kit
Contents: 2 Super-Bite Anterior, 2 Super-Bite Posterior, 2 Centring Devices		Contents: 2 Endo-Bite Anterior with ring, 2 Endo-Bite Posterior with ring, 2 Centring Devices	
Part No.	Description	Part No.	Description
1021	Super-Bite with Ring Anterior	1791	Endo-Bite with Ring Anterior
Contents: 4 Super-Bite Anterior with ring, 1 Centring Device		Contents: 4 Endo-Bite Anterior with ring, 1 Centring Device	
Part No.	Description	Part No.	Description
1022	Super-Bite with Ring Posterior	1792	Endo-Bite with Ring Posterior
Contents: 4 Super-Bite Posterior with ring, 1 Centring Device		Contents: 4 Endo-Bite Posterior with ring, 1 Centring Device	

X-Ray Sensor Holders for Direct Digital Imaging

Part No.	Description
2720	Testset X-Ray Sensor Holder System

Contents: 5 sensor holders with ring (1 Kwik-Bite Senso, 1 Super-Bite Senso Anterior, 1 Super-Bite Senso Posterior, 1 Endo-Bite Senso Anterior, 1 Endo-Bite Senso Posterior), 5 Centring Devices

Part No.	Description
2700	Kwik-Bite Senso Standard Kit

Contents: 4 Kwik-Bite Senso with ring, 1 Centring Device

Part No.	Description
2800	Super-Bite Senso Assorted Kit

Contents: 2 Super-Bite Senso Anterior with Ring, 2 Super-Bite Senso Posterior with Ring, 2 Centring Devices

Part No.	Description
2801	Super-Bite Senso Anterior

Contents: 4 Super-Bite Senso Anterior with Ring, 1 Centring Device

Part No.	Description
2802	Super-Bite Senso Posterior

Contents: 4 Super-Bite Senso Posterior with Ring, 1 Centring Device

Part No.	Description
2900	Endo-Bite Senso Assorted Kit

Contents: 2 Endo-Bite Senso Anterior with ring, 2 Endo-Bite Senso Posterior with ring, 2 Centring Devices

Part No.	Description
2901	Endo-Bite Senso Anterior

Contents: 4 Endo-Bite Senso Anterior with ring, 1 Centring Device

Part No.	Description
2902	Endo-Bite Senso Posterior

Contents: 4 Endo-Bite Senso Posterior with ring, 1 Centring Device

X-Ray Centring Rings

Part No.	Description
1721	Centring Rings refill

Contents: Refill Pack of 3

X-Ray Centring Devices

Part No.	Description
1795	Mixed set centring-devices

Contents: Packs of 20 (5x Super-Bite Anterior centring-devices, 5x Super-Bite Posterior centring-devices, 6x Kwik-Bite and Paro-Bite centring-devices, 2x Endo-Bite Anterior centring-devices, 2x Endo-Bite Posterior centring-devices)

Part No.	Description
1781	Kwik-Bite and Paro-Bite centring-devices Refill

Contents: Packs of 20

Part No.	Description
1793	Endo-Bite Anterior centring-devices Refill

Contents: Packs of 20

Part No.	Description
1023	Super-Bite Anterior centring-devices Refill

Contents: Packs of 20

Part No.	Description
1794	Endo-Bite Posterior centring-devices Refill

Contents: Packs of 20

Part No.	Description
1024	Super-Bite Posterior centring-devices Refill

Contents: Packs of 20

Kerr™

 KerrHawe SA
Via Strecce 4
6934 Bioggio / Switzerland
www.kerrdental.eu
00800 41 05 05 05

CE

504951-11-'15

Together, we're more.™