KerrTM

NexusTM **RMGI**

Resin-Modified Glass Ionomer Luting Cement

Instruction For Use

Cement

NEXUS™ RMGI Luting Cement

Description

Nexus™ RMGI is a radiopaque resin-modified glass ionomer luting cement offered in a paste/paste formulation that provides an option for tack light-curing of excess cement. This dual-cure material is packaged in dual barrel syringes with single-use automix tips. Optional intra-oral and root canal dispensing tips are available to allow the user to deliver the desired volume of cement directly into the restoration and/or tooth prep.

Indications for Use

Nexus RMGI is intended for use as a permanent dental cement. It is indicated for the following indirect restorations:

1. Cementation of metal-based inlays, onlays, crowns and bridges
2. Cementation of resin inlays, onlays, crowns and bridges
3. Cementation of all ceramic inlays
4. Cementation of high strength (zirconia based) all ceramic crowns and bridges
5. Cementation of metal, ceramic and fiber posts

Contraindications

Pulp capping. In rare cases, the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Instructions for Use

Note: Verify that the laboratory has pre-treated the final restoration in accordance with manufacturer's instructions. Automix tips are for single patient use only, to prevent cross-contamination between patients.

1. Prepare tooth in usual manner. For pulp capping use calcium hydroxide or other suitable pulp capping material.
2. Remove provisional restoration. Clean the preparation with pumice and water. Rinse thoroughly with water.
3. Air dry preparation (do not desiccate).

4. The preparation or cavity should be just dry enough that the surface has a slightly glossy appearance. Over drying can lead to post-operative sensitivity.
5. Try in restoration to ensure proper fit. Adjust if needed. Thoroughly clean the bonding surfaces of the restoration.
6. Remove the syringe from the foil pouch.
7. Note the date the syringe was removed on the syringe label.
8. Remove the sealing cap from the syringe.
9. Check the syringe openings for blockage; remove any paste plugs.
10. Before the first use or after a long interval between uses, squeeze out a small quantity of paste onto a pad to equalize the base and catalyst paste in the syringe. Discard the paste which has been squeezed out.
11. Attach a new mixing tip and secure it by turning it to the right.
12. Squeeze out and discard a peppercorn-size quantity until an evenly mixed paste in a homogenous color flows out of the tip.
13. The paste requires a certain amount of time to flow through the mixing tip. The flow through speed cannot be accelerated by increasing the pressure on the plunger.
14. As soon as the pressure on the plunger decreases, the material flow stops and the paste begins to set up. Do not use force to press out paste that has set as this could cause damage to the mixing tip and the syringe.

Dispensing of the material can be done as follows:

Inlay/Onlay – Dispense the cement directly into the cavity preparation covering all surfaces. Seat the restoration gently onto the tooth prep, allowing the cement to flow from all sides.

Crown – Dispense the cement directly into the restoration, covering all surfaces. Seat the restoration gently onto the tooth prep, allowing the cement to flow from all margins.

Post – Prepare the post space. Size and fit the post. Dispense the cement onto the post or directly into the canal space. Seat the post and vibrate slightly to avoid trapped air. Allow the cement to slowly flow from the canal space.

15. When the restoration is properly seated, remove excess cement. Excess cement is best removed in its gel state with a scaler or explorer. Gel state can be achieved by tack-curing excess with a light for approximately 2-3 seconds, or by allowing the cement to self-cure until it feels rubbery.
16. Finish the restoration and check occlusion when material has completely set. Material should completely set within 4'30" after placement in the mouth.
17. Remove the used mixing tip and replace it with the sealing cap.
18. Do not leave the used mixing tip on the syringe for a long period of time.
19. When only a small amount of paste remains in the syringe, the paste may not be dispensed from the mixing tip.

Note: Material minimum working time is 1'30" at ambient temperature; Set time is less than 4'30" at oral temperature. These times may vary based on storage conditions, temperature, humidity, and age of product.

STORAGE: Store between 4°C (39°F) and 25°C (77°F). This product is designed to be used at room temperature. If stored in a cooler, allow the product to reach room temperature prior to use. Ambient temperatures greater than 25°C (77°F) may affect shelf life or product performance. See outer package for expiration date.

Keep product in foil package until initial use. Once the foil pouch is opened, use the product within 4 months and prior to expiration date printed on syringe label.

Do not allow the pastes to dry out. Store with cap securely attached to the syringe. Leaving a used mixing tip on the syringe as a cap until the next application may result in evaporation and drying of the paste (not recommended).

Note: Mixing tips, intra-oral tips and root canal tips are for single patient use only to prevent cross-contamination between patients.

CAUTION: Uncured methacrylate resin may cause contact dermatitis and damage the pulp. Avoid contact with skin, eyes and soft tissue. Wash thoroughly with water after contact. Get medical attention for eyes. Do not take internally.

Limited Warranty - Limitation of Kerr's Liability

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr's product. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr's product. In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

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Kerr and Nexus are trademarks owned by
Kerr Corporation
Orange, CA 92867, U.S.A.
Kerr Australia Pty. Ltd.
Kerr Corporation. Kerr is registered with