

TRANSLUCENT



FLEXIBLE



PRECISE

The advantage is clearly visible.

- Translucent disc. Enhances work surface visibility.
- Flush-mounted mandrel. Protects neighboring teeth and tissue.
- Identifiable color-coding system. Color indicates grit type and abrasive side.
- Three-step polishing system. Achieves natural high gloss with fewer steps.1
- Contouring disc. Extra-Coarse grit available for contouring prior to finishing and polishing.










OptiDisc Finishing & Polishing Discs

The OptiDisc finishing and polishing system gives your restorations a final polish similar to that of natural dentition. With a three-grit system instead of the traditional four, you can achieve a higher surface gloss with fewer steps than with competitive products. OptiDiscs are translucent for enhanced working surface visibility. Kerr also offers OptiDisc Extra-Coarse for contouring composite restorative materials prior to finishing and polishing.

The color-coded discs clearly identify the grit size. Three disc diameters are available: 9.6 mm, 12.6 mm and 15.9 mm.

3-Step OptiDisc™ System vs. 4-Step 3M Sof-Lex™ XT



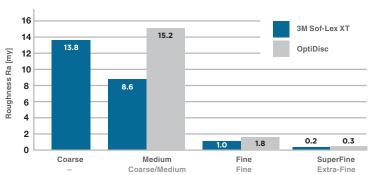

Kerr Internal Data

OptiDisc achieves a higher surface polish in just three steps compared with other four-step systems. The charts above demonstrate that the Coarse/Medium OptiDisc has higher mass revoval properties than the Coarse Sof-Lex XT, yet inflicts less surface roughness on a restoration, resulting in an easier-to-achieve, smoother final polish.

Gross Reduction

Extra-Coarse O O O

Polishing



Other Features.

- Long-lasting. Made of durable polyester, impregnated with aluminum oxide particles.
- Precise. Flush-mounted mandrel and patented retention system ensure precise operation, protecting neighboring teeth and tissue.
- No mess. Pumice or slurry not required. Use with or without water.

Ordering

33475 OptiDisc Assorted Kit

Contains 30 discs of each:

9.6 mm and 12.6 mm - Extra-Fine

9.6 mm and 12.6 mm - Fine

9.6 mm and 12.6 mm - Coarse/Medium

9.6 mm and 12.6 mm - Extra-Coarse 3 Mandrels

OptiDisc Refills

Contains 30 discs of each: 33476 9.6 mm Extra-Fine 33477 9.6 mm Fine 33478 9.6 mm Coarse/Medium 33479 9.6 mm Extra-Coarse 33480 12.6 mm Extra-Fine 33481 12.6 mm Fine 33482 12.6 mm Coarse/Medium 33483 12.6 mm Extra-Coarse 33484 15.9 mm Extra-Fine 33485 15.9 mm Fine 33486 15.9 mm Coarse/Medium 33487 15.9 mm Extra-Coarse



©2015 Kerr Corporation MKT-1080

