

# OptiBond™

Technique Guide

## eXTRa Universal

### Preparation

#### Primer



Unidose™

#### Adhesive



Bottle



# OptiBond™

Technique Guide

## eXTRa Universal

### Direct Restorations

#### Primer

1. Dispense primer from the Unidose™ applicator. 2. Apply primer to the tooth surface with a brush. 3. Dry the surface with a blue air syringe.

Unidose™

00:00:20

00:00:05

1. Dispense primer from the bottle.

Bottle

#### Adhesive

4. Dispense adhesive from the Unidose™ applicator. 5. Apply adhesive to the tooth surface with a brush. 6. Dry the surface with a blue air syringe. 7. Cure the adhesive with a curing light.

Unidose™

00:00:15

00:00:05

00:00:10\*

4. Dispense adhesive from the bottle.

Bottle

\* Demi™  
Demi™ Plus  
Demi™ Ultra → 00:00:05

# OptiBond™

Technique Guide

## eXTRa Universal

### Indirect Restorations: 1

Lithium Disilicate/Porcelain

① 30 PSI  
② Hydrofluoric etchant  
③ Unidose Bottle  
④  
⑤  
⑥\*

00:01:00  
00:00:10\*\*

Metal and Zirconia

① 60 PSI  
② Unidose Bottle  
③  
④  
⑤\*

00:00:10\*\*

Composite

① 15 PSI

\* Kerr NX3 (2, 5, 6)

\*\* Demi™  
Demi™ Plus →  
Demi™ Ultra 00:00:05


# OptiBond™

Technique Guide





## eXTRa Universal

### Indirect Restorations: 2

#### Primer

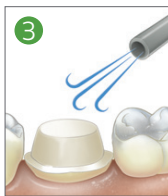

#### Adhesive

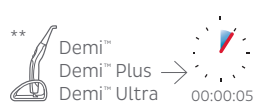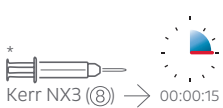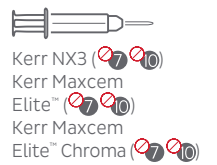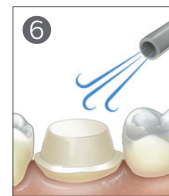




# OptiBond™

## Technique Guide




### eXTRa Universal

#### Post and Core Buildup




##### Primer

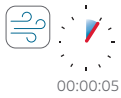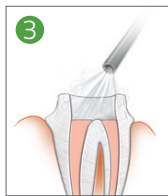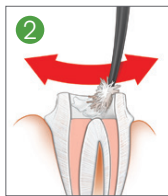


##### Adhesive

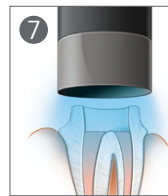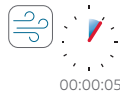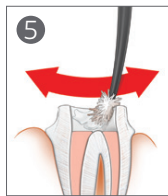

Kerr NX3  
Kerr Maxcem Elite™  
Kerr Maxcem Elite™ Chroma

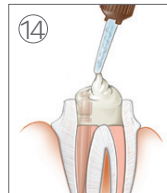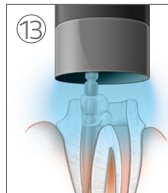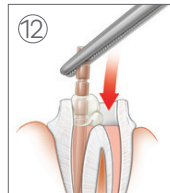

Kerr NX3 (7, 10)  
Kerr Maxcem Elite™ (7, 10)  
Kerr Maxcem Elite™ Chroma (7, 10)

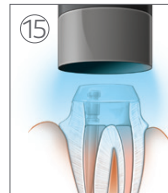

\* Demi™  
Demi™ Plus  
Demi™ Ultra → 00:00:05

# OptiBond™

## Technique Guide

### eXTRa Universal

#### Two-Component Universal Adhesive

OptiBond, Unidose, NX3, Maxcem Elite, Demi, and Kerr are all trademarks owned by Kerr Corporation and are registered with the U.S. PTO and other countries.

**es** Adhesivo universal de dos componentes | **pl** Dwuskładnikowy, uniwersalny system wiążący | **bg** Двукomпонентен универсален адхезив | **fr** Adhésif universel à deux composants | **hu** Kétkomponensű univerzális ragasztó | **cs** Dvousložkové univerzální adhezivum | **it** Adesivo universale a due componenti | **pt** Adesivo universal de dois componentes | **fi** Kaksikomponenttinen yleiskäyttöinen sidosaine | **el** Συγκολλητικό γενικής εφαρμογής δύο συστατικών | **et** Kahekomponendiline universaalne adhesiiv | **lt** Dviejų komponentų universalusis rišiklis | **da** Tokomponent-universaladhæsiv | **sk** Dvojsložkové univerzálné adhezívum | **nl** Universeel tweecomponentenadhesief | **sv** Tvåkomponents universaladhesiv | **tr** İki Bileşenli ÜniVersal Yapıştırıcı | **lv** Divkomponentu universāls adhezīvs | **ro** Adeziv universal cu două componente | **sl** Dvokomponentno univerzalno lepilo | **de** Zweikomponenten-Universaladhäsiv | **hr** Dvokomponentni univerzalni adheziv

**en** GPDM Monomer Technology™ | **fr** Technologie Monomère GPDM™ | **es** Tecnología del monómero de GPDM™ | **de** GPDM-Monomer-Technologie™ | **pt** Tecnologia de monómero GPDM™ | **it** Tecnologia a monomero GPDM™ | **hu** GPDM monomer technológia™ | **fi** GPDM-monomeeriteknologia™ | **da** GPDM-monomerteknologi™ | **cs** Monomerní technologie GPDM™ | **bg** Мономерна технология GPDM™ | **pl** Technologia monomeru GPDM (GPDM Monomer Technology™) | **ro** Tehnologie cu monomer GPDM™ | **sk** GPDM monomérová technológia™ | **hr** Tehnologija GPDM monomera™ | **sl** Tehnologija monomera GPDM™ | **sv** GPDM-monomerteknologi™ | **tr** GPDM Monomer Teknolojisi™ | **el** Τεχνολογία μονομερούς GPDM™ | **et** GPDM Monomeertechnologia™ | **lt** GPDM monomerų technologija™ | **lv** GPDM monomēra tehnoloģija™ | **nl** GPDM-monomeertechnologie™ | **uk** Технологія з використанням мономерів GPDM™

 Kerr Corporation  
1717 W. Collins Ave.  
Orange, CA 92867 USA  
Tel: 1-800-KERR-123  
kerrdental.com

 EC REP  
Kerr Italia, S.r.l.  
Via Passanti, 332  
I-84018 Scafati (SA) Italy  
Tel: +39-081-8508311

Distribution Facility  
Kerr Australia Pty. Ltd.  
Unit 10  
112-118 Talavera Road  
North Ryde, NSW, 2113  
Australia



Уповноважений  
представник в Україні:  
Представництво «Спофа-Дентал а.с.»  
бульвар Лесі України, 26, офіс 717,  
місто Київ, 01133, Україна  
Тел: +38 (044) 286 49 12  
Факс: +38 (044) 286 10 03

  
0086



Rx ONLY