SimpliShade™

Universal

Nanohybrid Universal Restorative Composite

Instructions for Use

DMC4043_Rev01
2020-08

Kerr Corporation
1717 W Collins Ave.
Orange, CA 92667 USA
1-800-KERR-123 | kerrdental.com

EC REP **Kerr Italia S.r.l.**
Via Passanti, 332
84018 Scafati (SA), Italy
+39 081 850 8311 | kerrdental.com

Rx only
*for US law only

Importer: Ormco B.V.
Basicweg 20 NL-3821 BR, Amersfoort,
The Netherlands

KAVO **Kerr**

SYMBOLS DESCRIPTION

Full explanation of symbols used on Kerr packaging located at:
<http://www.kerrdental.com/symbols-glossary>

	Catalogue number
	Manufacturer
	Batch Code
	Consult Instructions for Use
	Consult Instructions for Use at www.kerrdental.com/resource-center
	Caution
	Authorized representative in the European Community
	Caution: Federal law (USA) restricts this device to sale by or on the order of a licensed healthcare practitioner.
	Use-by date
	Do not re-use
	Keep away from sunlight
	1. Medical Device
	2. Temperature Storage Limitations: Low 2°C/36°F and High 28°C/82°F

CONTENTS

	3. Unidose Tip
	4. Syringe

en - ENGLISH

SimpliShade™

Nanohybrid Universal Restorative Composite
ISO 4049: Polymer-Based Restorative Material, Class 2/Group 1

GENERAL INFORMATION

SimpliShade is a light-cured, esthetic, resin-based nanohybrid dental restorative designed for the placement of direct restorations. SimpliShade is designed to be used after placement of a dental adhesive, such as a product from the KaVo Kerr OptiBond™ family. SimpliShade is available in Unidose composite compules and syringe delivery systems.

This product is intended to be used by qualified dental professionals on a general patient population.

PRODUCT COMPOSITION

100% Aluminum has a radio-opacity equivalent to that of dentin and 200% Aluminum is equivalent to enamel.

The restorative is available in three composite shades (Light, Medium, Dark) and two special shades (Bleach White and Universal Opaque). All shades are radiopaque with about 250% Aluminum.

The fillers comprise selection of chemically infused mixed oxides, pre-polymerized filler, barium glass filler, silica, and ytterbium trifluoride (about 81.5% by weight or 65.1% by volume for the Light, Medium, Dark, and Universal Opaque shades, about 78.5% by weight or 60% by volume for the Bleach White shade). The inorganic fillers is up to 75% by weight or 55% by volume. The Light, Medium, Dark, and Universal Opaque shades have a range particle size of 40 nm to 6 microns. The Bleach White shade has a range particle size of 40 nm to 30 microns. SimpliShade contains Bis-EMA, Bis-GMA, and TEGDMA resins. None of the ingredients in the product contain pharmacologically active components.

INDICATIONS FOR USE

SimpliShade, a nanohybrid dental restorative, is indicated for direct placement in all cavity classes in anterior and posterior teeth. Additional indications include: repair of porcelain/composite restorations, and core build-ups.

CONTRAINDICATIONS

None known.

WARNINGS

- Do not use this product on patients with known methacrylate allergies. Uncured methacrylate resin may cause contact dermatitis and damage to the pulp.
- Avoid contact with skin, eyes, and soft tissue. Wash thoroughly with water after contact.
- Get medical attention for eyes.
- Do not take internally.

PRECAUTIONS

Kerr recommends the following precautions when using SimpliShade or any other composite material:

- Provide the patient with protective eyewear.
- Use a rubber dam to isolate the oral cavity.
- Avoid fluid contamination during placement.
- Use of protective gloves and a no-touch clinical technique is strongly recommended. If the composite is touched with commonly used gloves, acrylate may penetrate the glove. Dispose of glove immediately and wash hands.
- Protect the material from ambient light that may initiate polymerization.
- The Unidose tips are designed for SINGLE PATIENT USE ONLY to prevent cross-contamination between patients.

Kerr, SimpliShade, and Unidose are trademarks owned by Kerr Corporation.

ADVERSE EVENTS

If a serious incident occurs with this medical device, report it to the manufacturer and to the competent authority for that country in which the user and/or patient is established.

STEP-BY-STEP INSTRUCTIONS

- Wear personal protective equipment when using and handling this product.
- Do not use this product near eugenol-containing products.
- Clean the tooth with non-fluoridated pumice and water to remove any debris or stain that may interfere with the shade selection process.
Select the desired shade based on the chart listed below. A composite mock-up or VITA®-matched shade guide may be used to check the color of the tooth to be restored and any adjacent teeth. In many cases, a single shade of SimpliShade will be sufficient to restore the tooth. The shade of the restorative material is affected by the thickness of the composite. Shade selection and mock-ups should match the thickness of the tooth to be restored:

Composite Shades	VITA Shade Guide Equivalent
Light - LT	A1, B1, B2, C1, D2
Medium - MD	A2, A3, C2, D3, D4
Dark - DK	A3.5, A4, B3, B4, C3, C4
Accessory Shades	
Bleach White - BW	Bleach white
Universal Opaque - UO	Blocking discoloration

- After shade selection, isolate the tooth with your preferred isolation method.
- Prepare the tooth using your preferred rotary instruments and using contemporary preparation designs. SimpliShade offers excellent blending (chameleon effect) which may be further enhanced by the use of small bevels on preparation margins. If replacing a restoration, be sure to remove all existing restorative material from the operative site.
- If a pulp exposure has occurred, place a small amount of calcium hydroxide followed by the placement of a conventional glass ionomer base.
- SimpliShade is compatible with all methacrylate dental adhesive systems, such as a product from the KaVo Kerr OptiBond family. Complete the bonding procedure per manufacturer's instructions.
- If desired, place a liner using a low viscosity flowable composite and light cure according to manufacturer's instructions.
- In situations of discoloration of the tooth structure or repair of porcelain/composite, the Universal Opaque shade can be used to block out the discoloration or hide a metal portion if it exists.
- SimpliShade features optimal, non-sticky handling. It permits the use of hand instruments to define most of the restoration anatomy, which will help minimize the time and effort associated with rotary instrumentation.
For Unidose delivery: Inject composite material from the Unidose tip directly into the cavity, starting with the deepest portion.
For Syringe delivery: After extruding material onto a pad, back off syringe at least two full turns to relieve pressure on the material.
- Apply composite layers up to 2 mm in thickness each layer for all shades except for Universal Opaque and up to 1.5 mm in thickness each layer for Universal Opaque, taking care to properly adapt the material to the cavity surface.
- Cure each layer*. Refer to the table at the end of these instructions for recommended curing times of the various shades at various light intensities.
- Adjust occlusion as needed. Complete the finishing and polishing using preferred polishing system.

*RECOMMENDED CURING TIMES

Adequate curing is essential for restorative function. Uncured or undercured composite material is far more likely to experience some form of clinical failure. Kerr recommends regular inspections of your curing lights to ensure they are functioning and providing the expected irradiance.

SimpliShade is intended to be cured by a halogen or LED light with a minimum light intensity of 650 mW/cm² and a light wavelength output within the 400-520 nm range. Please refer to the table below for curing recommendations. All times refer to a single occlusal surface cure. In any posterior restoration, Kerr recommends additional 10 second cures on both the buccal and lingual surfaces of the tooth.

	Universal Opaque	Dark Shade	Light, Medium, and Bleach White Shades
Incremental Depth	1.5 mm	2.0 mm	2.0 mm
Optilux™ (or light with output 650 – 1000 mW/cm ²)	40 Seconds	40 Seconds	20 Seconds
Demi™ Ultra / Demi™ Plus (or light with output >1000 mW/cm ²)	20 Seconds	20 Seconds	10 Seconds

For all other lights, see manufacturer's recommendation.

CLEANING AND DISINFECTION

Unidose Tips: Do not re-use Unidose tips.

Unidose tip guns: Refer to the reprocessing instructions for the manufacturer of the tip gun you are using.

Composite syringes: To prevent syringes from getting contaminated with saliva or other body fluids, use protective barriers. Always handle the syringe with clean/disinfected gloves. Do not reuse if it becomes contaminated. Replace the syringe cap after each use to prevent the resin from hardening in the syringe.

STORAGE AND DISPOSAL

Product should be stored at temperature between 2°C to 28°C (36°F to 82°F). If product is stored in a refrigerator allow the product to reach room temperature prior to use. Every effort should be made to protect the composite from visible light prior to use. The expiration date is marked on packaging. Do not use restorative after expiration date.

For proper disposal always follow Federal or State laws applicable to that country.

The SDS for this product is available at: www.kerrdental.com. It can also be obtained from your usual supplier.

LIMITED WARRANTY - LIMITATION OF KERR'S LIABILITY

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr's product. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr's product. In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages. EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

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All rights reserved. ©2020 Kerr Corporation