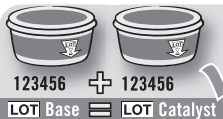
(DE)	Gebrauchsanweisung .....	5
(GB)	Instructions for use .....	13
(FR)	Mode d'emploi.....	21
(ES)	Instrucciones de uso.....	29
(IT)	Istruzioni per l'uso .....	37
(PT)	Instruções de uso .....	45
(NL)	Gebruiksaanwijzing .....	53
(SE)	Bruksanvisning .....	61
(DK)	Brugervejledning .....	69
(NO)	Bruksanvisning .....	77
(FI)	Käyttöohjeet.....	85
(GR)	Οδηγίες χρήσης.....	93
(CZ)	Návod k použití.....	101
(HU)	Használati utasítás.....	109
(LV)	Lietošanas instrukcija .....	117
(LT)	Naudojimo instrukcija .....	125
(PL)	Instrukcja obsługi.....	133

**A**

**B**

**C**

**D**

**E**

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Stand: 2014-04

#### **Description of applicational/packaging icons**

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**K** = Intraoral Tip

## Instructions for use

### **Elastomeric, silicone-based precision impression material, addition-cured**

The Flexitime range is an optimally matched system and its different types of material can be combined for all impression techniques to suit specific requirements.

Flexitime Monophase Pro Scan is prepared without requiring a previous surface treatment for optical scanning in dental scanners designed for scanning impression materials, such as the 3 Shape D 700 laser scanner.

#### **Material**

Flexitime Easy Putty

Flexitime Dynamix Putty

Flexitime Heavy Tray

Flexitime Dynamix Heavy Tray

Flexitime Dynamix Monophase Pro Scan

Flexitime Monophase

Flexitime Dynamix Monophase

Flexitime Medium Flow

Flexitime Correct Flow

Flexitime Light Flow

#### **Pack**

Tub, 1:1, 2 x 300 ml

Cartridge, 5:1, 380 ml

Cartridge, 1:1, 50 ml

Cartridge, 5:1, 380 ml

Cartridge, 5:1, 380 ml

Cartridge, 1:1, 50 ml

Cartridge, 5:1, 380 ml

Cartridge, 1:1, 50 ml

Cartridge, 1:1, 50 ml

Cartridge, 1:1, 50 ml

**Side effects:**

This product or one of its components may in particular cases cause hypersensitive reactions. If suspected, information on the ingredients can be obtained from the manufacturer.

**Notes on use**

Not to be used in conjunction with other silicone materials.

**Close tubs properly after use. Do not mix up caps/lids/measuring spoons. Dispense a small amount of material from a 50 ml cartridge before attaching a mixing tip. After use, leave the mixing tip on the cartridge to keep it closed. Immediately before the next use, remove the mixing tip and attach a new one. Mixing Tips and Intraoral Tips are single-use products. To prevent cross-contamination between patients, Intraoral Tips must be discarded after each use.**

In all addition-curing silicone impression materials, the setting process may be affected by contact with products containing active Sulfur or Nitrogen, or by fresh composite fillings or superstructures. These products include latex gloves, Polysulfide impression materials, retraction solutions, etc. Flexitime impression materials should be at room temperature 23°C (73°F) for processing. Polyvinyl siloxanes are chemically resistant. Material which has not set can soil clothing. Visually inspect for damage before first use. Damaged products must not be used.

Flexitime	EN ISO 4823	Product colour		Mixing ratio	Mixing time	Extraoral Working time* 23°C (73°F)	Intraoral Working time 35°C (95°F)	Min. time in mouth	Accessories	
		Base	Catalyst						Mixing Tips	Intraoral Tips
<b>Flexitime Easy Putty</b>	Type 0 putty	violet	light grey	1:1	30	2:30	–	2 :30	n.a.	n.a.
<b>Flexitime Dynamix Putty</b>	Type 0 putty	violet	light grey	5:1	auto	2:30**	–	2 :30	dynamic yellow 66035999	n.a.
<b>Flexitime Heavy Tray</b>	Type 1 heavy bodied	violet	light grey	1:1	auto	2:30	–	2 :30	green 66001989	n.a.
<b>Flexitime Dynamix Heavy Tray</b>	Type 1 heavy bodied	violet	light grey	5:1	auto	2:30**	–	2 :30	dynamic yellow 66035999	n.a.
<b>Flexitime Dynamix Monophase Pro Scan</b>	Type 2 medium bodied	pink	white	5:1	auto	2:30**	1:00	2 :30	dynamic yellow 66035999	n.a.
<b>Flexitime Monophase</b>	Type 2 medium bodied	purple	light grey	1:1	auto	2:30	1:00	2 :30	green 66001989	white 65687514
<b>Flexitime Dynamix Monophase</b>	Type 2 medium bodied	purple	light grey	5:1	auto	2:30**	1:00	2 :30	dynamic yellow 66035999	n.a.
<b>Flexitime Medium Flow</b>	Type 2 medium bodied	light green	light grey	1:1	auto	2:30	1:00	2 :30	yellow 66001988	yellow 66000782
<b>Flexitime Correct Flow</b>	Type 3 light bodied	light green	light grey	1:1	auto	2:30	1:00	2 :30	yellow 66001988	yellow 66000782
<b>Flexitime Light Flow</b>	Type 3 light bodied	green	light grey	1:1	auto	2:30	1:00	2 :30	yellow 66001988	yellow 66000782

\* at room temperature of 23°C (73°F), 50% rel. humidity. Higher temperatures shorten, lower temperatures prolong these times.

\*\* measured using the Dynamix® mixer from Kulzer.

**If there are substantial undercuts or highly divergent teeth in the impression area, appropriate blocking out must be performed if using Flexitime Heavy Tray or Putty.**

**Optical scanning:**

The quality of the scanned data depends on the impression scanner. Follow the manufacturer's recommendations for preparation of teeth and the instructions for the optical scanning procedure.

**Model fabrication:**

The impression can be cast 30 minutes after removal from the mouth – there are no other time constraints. The impression is stable in long-term storage.

**Recommended model material:**

All standard model materials can be used.

We recommend:

Class III plaster, MOLDANO®

Class IV plaster, MOLDASTONE®/MOLDASYNT®

**Electroplating:**

Impressions can be electroplated with copper and silver in conventional baths.

**Disinfection:**

Impressions can be treated with an aqueous sodium hypochlorite solution (5.25%, for up to 10 minutes\*) or with other disinfectant solutions suitable for silicones. When doing so, please observe the disinfection/processing instructions issued by the relevant manufacturers.

\*(DGZMK 3/93 V1.0)

## Adhesion:

We recommend using our Universal Silicone Adhesive.

## Storage conditions:

Do not use Flexitime impression materials if the expiry date has elapsed.

Do not store above 25°C (77°F).

## Dynamix System, 380 ml – step by step

Fig. 1  
Remove cap.  
Insert cartridge  
into the mixer

Fig. 2  
Attach Dynamix  
Mixing Tip

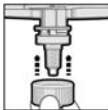
Fig. 3  
Position Dynamix  
fixation ring

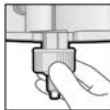
Fig. 4.1  
Attach Dynamix  
fixation ring.  
Slide over  
recesses

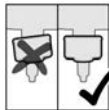
Fig. 4.2

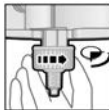
Fig. 5  
Lock by giving  
fixation ring  
a ¼ turn  
clockwise

### **Inserting the Dynamix cartridge:**

For information on how to handle the mixer being used please refer to the detailed instructions issued by the relevant manufacturer.

1. Hold the tab on the cap of the outlets and bend it upwards through 90° without applying any excessive force.
2. Take the tab between your thumb and index finger whilst you hold it securely with your other hand and remove the cap from the cartridge completely (Fig. 1).
3. Place one of the enclosed Dynamix Mixing Tips over the outlets of the cartridge (Fig. 2).
4. If it proves difficult to attach the mixing tip, check to make sure that the central internal hexagon of the mixing tip is properly aligned with the hexagon of the drive spindle.
5. When the mixing tip is in the correct position (Fig. 3), push the Dynamix fixation ring over the mixing tip until it stops and turn it clockwise (1/4 turn) until it is securely locked in place (Figs. 4.1, 4.2 and 5).
6. When impression material has been mixed, transfer it to a separate intraoral impression syringe, e.g. Impression Jet (Flexitime Dynamix Monophase only) or a suitable impression tray.
7. When the material has been dispensed, retract the mixer plungers from the cartridge in order to avoid continuous pressure and thus prevent excess material from continuing to emerge from the mixing tip.

### **Notes about safe use of the Dynamix System**

Dynamix cartridges are heavy and can be damaged if they are dropped.

Damaged cartridges must no longer be used.

Only use Dynamix cartridges in mixers with a closed protective flap.

Before using for the first time, adjust the levels of the two material components. Dispense approx. 2–3 cm material and discard this amount before starting to fill the tray. When the cartridge is used in different mixers, 2–3 cm of material has to be discarded after each device change. Each time before you use the material check the outlet channels of the cartridge for blockages.

At the end of a working day or if the mixer is not going to be used for a lengthy period, remove the cartridge from the device and store it in a horizontal position with the outlets closed off by the mixing tip or cap.

**After use, leave the mixing tip on the cartridge to keep it closed.**

**Immediately before the next use, remove the mixing tip and attach a new one.**

**To do so, detach the Dynamix fixation ring and use it again for the new mixer attached.**

Only use Dynamix System components for dispensing and mixing the impression material in Dynamix cartridges.

Check the system components for physical damage on a regular basis and only use components which are intact.

Empty cartridges must be disposed of safely in accordance with the EU directive 94/62/EC.

### Warnings and safety instructions

If material comes into contact with the eyes, rinse it out with water for several minutes. If symptoms persist, consult an eye specialist.

Do not swallow and ingest. If health problems arise after swallowing impression material, seek medical attention immediately. Intestinal blockage may arise in rare cases.

**For use by dental personnel only.**

Dated: 2014-04

#### Description of applicational / packaging icons

- A** = Only use pastes with same LOT No
- B** = Icon "Use of the special program at the Dynamix® *speed* Mixing Unit possible"
- C** = Mixing ratio
- D** = Mixing time

- E** = Extraoral working time
- F** = Intraoral working time
- G** = Time in mouth
- H** = Static mixing tip
- K** = Intraoral tip

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#### **Matériau**

Flexitime Easy Putty  
Flexitime Dynamix Putty

Flexitime Heavy Tray  
Flexitime Dynamix Heavy Tray

Flexitime Dynamix Monophase Pro Scan

Flexitime Monophase  
Flexitime Dynamix Monophase

Flexitime Medium Flow  
Flexitime Correct Flow  
Flexitime Light Flow

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Kulzer GmbH
Kulzer, LLC