MEMOREG® 2/MEMOSIL® 2

DE	Gebrauchsanweisung	3
GB	Instructions for use	6
FR	Mode d'emploi	9
ES	Instrucciones de uso.....	12
IT	Istruzioni per l'uso	15
PT	Instruções de uso	18
NL	Gebruiksaanwijzing	21
SE	Bruksanvisning	24
DK	Brugervejledning	27
NO	Bruksanvisning	30
FI	Käyttöohjeet.....	33
GR	Οδηγίες χρήσης	36

KULZER
MITSUI CHEMICALS GROUP

Cartridge Delivery

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GB

MEMOREG® 2

Polyvinyl siloxane for direct application
very high hardness

Indications

- Standard interoclusal records
- location material for intra-oral pin tracing template

Contraindications

Memoreg 2 should not be used for conventional impressions since its high final hardness makes it very difficult to remove from the mouth. Undesired tooth extractions may result.

Side effects

Hypersensitive persons may experience an allergic reaction to the product.

MEMOSIL® 2

Polyvinyl siloxane for direct application
transparent

Indications

- Front, occlusal record
- Transfer matrix for brackets (indirect bonding)
- Fixing of x-ray balls when planning the implants

	MEMOREG® 2	MEMOSIL® 2
Working time*1	30 seconds	30 seconds
Minimum time in mouth*2	60 seconds	3 minutes
Shore A hardness	90	72
Product colours	pink	transparent

*1 Total working time (including mixing) at 23°C (73°F); shortened to 20s at oral temperature (35°C (95°F))

*2 Minimum time that the bite registration material must be in a closed mouth.

The data given are furnished to the best of our knowledge and are based on internal test results.

Instructions for use

Visually inspect for damage before first use. Damaged products must not be used. Mixing tips and bite registration tips are for single use only. Dispense a small amount of material from a 50 ml cartridge before attaching a mixing tip. Attach mixing tip and fix by rotating clockwise. Dosing and mixing is done automatically on application.

When bite registration materials based on polyvinyl siloxane are touched with latex gloves, retraction solutions or freshly inserted composite fillings, it can change the curing behaviour.

Memoreg 2/Memosil 2 should be used at room temperature (23°C (73°F)). Lower temperatures, for example when the materials are stored in a refrigerator can extend the working time, and higher temperatures can shorten it.

Hardened polyvinyl siloxanes are chemically resistant. Clothes can be permanently stained if the contact material is not completely cured.

Memoreg 2/Memosil 2 are applied with the green mixing tip (1:1). It can also be combined with the Bite Registration Tip.

Disinfection

Memoreg 2/Memosil 2 can be treated with an aqueous sodium hypochlorite solution (5.25% for up to 10 minutes*). The treatment does not affect the surface or dimensions.

When using other disinfectant solutions suitable for silicones, follow the disinfection and processing instructions of the manufacturer. *(DGZMK 3/93 V1.0)

Storage conditions

Do not store above 25°C (77°F)!

Do not use the material after the expiration date.

Please quote batch number and expiry date in all correspondence about the product.

Expiration date  and batch number : See note on packaging.

Warning and safety instructions

If the material contacts your eyes, rinse with copious amounts of water for several minutes. If symptoms persist, consult an eye specialist.

Do not inhale, swallow or ingest. If health problems arise after swallowing impression material, seek medical attention immediately.

Intestinal blockage may arise in rare cases. Immediately consult a doctor if the product is inhaled.

Do not leave any residual material in the sulcus or oral cavity.

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Kulzer GmbH
Kulzer, LLC