



# Brite Shield™

Instrument Cleaner and Protectant



**premier**® Inspired solutions for daily dentistry®



Manufacturer:  
Premier® Dental Products Company  
1710 Romano Drive,  
Plymouth Meeting, PA 19462 U.S.A.  
888-670-6100 • 610-239-6000  
premierdentalco.com • Made in U.S.A.



[premierdentalco.com/promise](http://premierdentalco.com/promise)



MDSS GmbH  
Schiffgraben 41  
30173 Hannover, Germany



**Intended Use:**

Brite Shield™ is a static soak and ultrasonic cleaner that chemo-mechanically debrides clogged dental instruments and aids to protect instruments through a unique passivation process to restrict rust and corrosion.

**Device Description:**

Premier® Brite Shield™

Power enzymatic cleaning action to remove proteins, light stains, set cements, and protects instruments from rust and corrosion.

**Contraindications:**

Do not use Brite Shield™ under the following conditions:

- If the container or cap is found broken/damaged
- Should not be used on anodized aluminum instruments

**PRECAUTIONS:**

- Do not mix with other detergents or contaminants
- Avoid over dilution.
- Keep product stored only in original container.
- Keep containers tightly closed when not in use.
- Do not place a wet scoop into container.
- Wash exposed skin and hands thoroughly after handling.

**WARNINGS:**

- Remove contact lenses, if exposed.
- EYES – Rinse with large volumes of water for 15 minutes if exposed. If irritation persists, seek medical attention.
- SKIN – Rinse with large volumes of water.
- INGESTION – Drink large volumes of water. Contact physician promptly.

**Adverse Events:**

- IF SWALLOWED –
  - » Harmful, may cause nausea and vomiting.
  - » May be irritating to mouth and throat.
- May cause skin irritation, eye damage and allergic skin reactions.

If a serious incident has occurred in relation to the device, please report the incident to the Manufacturer and/or competent authority of the Member State where the device was used.

**Contents and Setup:**

Item No.	Product Specifications
9011996	Brite Shield SAMPLE (1pk)
9011997	Brite Shield 10 Pack
9011100	Brite Shield (CASE OF 6)
8101100	Brite Shield (Single Jar)
9011911	Brite Shield Sample INTL w/Illustration Card

**Materials:**

Environmentally friendly Powerful Enzymatic formula.

**DIRECTIONS FOR USE:**

- For daily cleaning and rust protection, place one scoop of Brite Shield powder and 64oz. (1894ml) of warm water in ultrasonic unit.
- For heavy contaminated instruments, use two scoops of powder.
- For cement/stain removal, place two scoops of Brite Shield powder and 16oz. (473ml) of warm water in ultrasonic unit.
- Stir until dissolved.
- Immerse instruments in solution and clean in ultrasonic unit for 1-5 minutes.
- Rinse instruments thoroughly with water and dry before sterilization.
- For optimal results, change solution per use or daily

**Disposal:**

After use, Dispose as biohazardous medical waste in accordance to facility or institutional guidelines/procedures.

**Storage:**








Store in a cool, dry place away from direct sunlight.

**Shelf Life:**

The product does not have a shelf life.

**For Further Information:**

To obtain SDS or IFU, visit [premierdentalco.com](http://premierdentalco.com) or call Premier at 610-239-6000.

Symbols	Symbol Meaning
	Manufacturer
	Batch number
	Warning, See IFU
	Store in dry cool place
	Avoid direct sunlight
	Medical Device
	UKCA