



# Vac Attak™

High Performance Evacuation System Cleaner



**premier**® Inspired solutions  
for daily dentistry®



Manufacturer:  
Premier® Dental Products Company  
1710 Romano Drive,  
Plymouth Meeting, PA 19462 U.S.A.  
888-670-6100 • 610-239-6000  
premierdentalco.com • Made in U.S.A.



[premierdentalco.com/promise](http://premierdentalco.com/promise)



MDSS GmbH  
Schiffgraben 41  
30173 Hannover, Germany



**Intended Use:**

VacAttak is intended to be high-performance evacuation system chlorinated cleaner, which is non-foaming, non-corrosive to metals and safe for use with all pumps.

**Device Description:**

Premier® VacAttak™ is a high-performance evacuation system chlorinated cleaner used in dental practices. VacAttak™ is available in powdered form.

**Contraindications:**

Do not use VacAttak™ under the following conditions:

- If the container or cap is found broken/damaged
- Should not be used on anodized aluminum instruments

**PRECAUTIONS:**

- Do not mix with other detergents or contaminants
- Avoid over dilution.
- Keep product stored only in original container.
- Keep containers tightly closed when not in use.
- Do not place a wet scoop into container.
- Wash exposed skin and hands thoroughly after handling.

**WARNINGS:**

- Remove contact lenses, if exposed.
- EYES – Rinse with large volumes of water for 15 minutes if exposed. If irritation persists, seek medical attention.
- SKIN – Rinse with large volumes of water.
- INGESTION – Drink large volumes of water. Contact physician promptly.

**Adverse Events:**

- IF SWALLOWED –
  - » Harmful, may cause nausea and vomiting.
  - » May be irritating to mouth and throat.
  - » May cause allergic type reaction.
- May cause skin irritation, eye damage and allergic skin reactions.

If a serious incident has occurred in relation to the device, please report the incident to the Manufacturer and/or competent authority of the Member State where the device was used.

**Contents and Setup:**

Item No.	Product Specifications
9011998	VacAttak 10 Pack
9011101	VacAttak (CASE OF 6)
8101111	VacAttak Single Jar
9011999	VacAttak SAMPLE (1pk)
9011912	VacAttak Sample INTL w/illustration Card

**Materials:**

VacAttak – Powerful organically chlorinated formula and has fresh lemon scent.

**DIRECTIONS FOR USE:**

- Fill Evacuation sink or other container with 32oz. (946ml) of warm tap water.
- Remove one scoop (8gm) of cleaning powder from container.
- Sprinkle powder into water and allow 30-45 seconds to thoroughly dissolve.
- Flush system with solution.
- Flush system again with tap water after cleaning.

**Attention:**

Upon initial use, the powerful cleaning action of VacAttak may cause accumulated debris to dislodge. Be sure to check the filter/trap after use for maximum efficiency.

- Highly effective at removing all types of organic debris.
- Use fresh solution daily.

**Storage:**

Store in a cool, dry place away from direct sunlight.

**Shelf Life:**








The product does not have a shelf life.

**Disposal:**

After use, Dispose as biohazardous medical waste in accordance to facility or institutional guidelines/procedures.

**For Further Information:**

To obtain SDS or IFU, visit premierdentalco.com or call Premier at 610-239-6000.

Symbols	Symbol Meaning
	Manufacturer
	Batch number
	Warning, See IFU
	Store in dry cool place
	Avoid direct sunlight
	Medical Device
	UKCA