SupaSorb Lint Free Cloth



Benefits of Using SupaSorb

- Over millions of tiny micro fibres disperse oil, grease, dirt and dust and collect it deep in the cloth.
- Cleans fast, easier, better, more economically and in a totally environmentally manner, i.e. No use of checmicals!

Directions of Use

- Perfroms with or without chemicals, just fold cloth and gently wipe surfaces.
- · Use cloth slightly damp or wet for tough, dried on mess.

Washing Instructions

- Hand or machine wash to remove dirt. Rinse thoroughly, do NOT use fabric softener.
- Hot water and dryer safe.
- Do NOT wash with mixed load as the cloth will pick up lint from other fabrics.
- Autoclavable place folded in a sterilisation pouch.

Caution

- Cloth itself does not scratch surfaces, but can pickup particles that do scratch.
- Test in an inconspicuous area before use.