En Stainless steel Abrasive Strips with aluminum oxide, indicated for removal of composite excesses in the interproximal region, available in 4mm and 6mm widths.

Use: Insert the strip into the interproximal space. Perform back and forth movements pressing the abrasive face against the restoration to provide the proximal restoration finishing. Once concluded, remove the strip.

Warnings, precautions and restrictions: Professional use only. Single use only. Its reuse can cause cross contamination. If there is proven allergy to any of its components, the product must not be used. The strips in contact with the gum can cause minor injury to the patient. The protection of the gum is recommended. Any serious incident related to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Composition: Stainless steel and aluminum oxide.

Storage: To be stored in a cool and dry place, protected from sunlight.

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