



Finishing and Polishing Strips Lija de Acabamiento Proximal



The Finishing and Polishing Strips are composed by two grits medium and fine providing finishing and polishing in composite restorations. Available in 2.5mm and 4mm widths

Use: Use the neutral center to insert the strip into the proximal space. Start the restoration finish with the medium grit (gray) side and then polish with the fine (white color) side.

Warnings, precautions and restrictions: Professional use only. Single use only. Its reuse may cause cross contamination. If there is proven allergy to any of its components, the product must not be used. The product is a medical device. The product is a medical device. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Storage: To be stored in dry and fresh place, protected from sunlight.

Composition: Polyester, resin and aluminum oxide.



La Lija de acabamiento proximal es constituida de dos granulaciones, media y fina, proporcionando el acabado y pulido en restauraciones de resinas compuestas. Disponible en los anchos de 2,5mm y 4mm.

Uso: Introducir la lija por su centro neutro en el espacio interproximal. Iniciar el acabado de la restauración con la extremidad de la granulación media (color gris) y luego realizar el pulimento con la lija fina (color blanco).

Advertencias, precauciones y restricciones: Uso profesional. Uso único. Su reutilización puede causar la contaminación cruzada. No utilizar en pacientes que tengan antecedentes de alergia a cualquiera de los componentes de este producto. El dispositivo es un producto sanitario. El dispositivo es un producto sanitario. Cualquier incidente grave relacionado con el producto debe comunicarse al fabricante y a la autoridad competente del Estado miembro en el que estén establecidos el usuario y/o el paciente.

Almacenamiento: Almacenar en local seco, fresco y al abrigo de la luz solar.

Composición: Poliéster, resina y óxido de aluminio.

Ref.:	Code	Content
3061	- 0006020002	- Approximal finishing and polishing strips 2,5mm x 150 un
3062	- 0006020003	- Approximal finishing and polishing strips 4mm x 150 un
3065	- 0006020006	- Approximal finishing and polishing strips 2,5mm x 100 un
3066	- 0006020007	- Approximal finishing and polishing strips 4mm x 100 un

Ref.:	Código:	Contenido:
3061	- 0006020024	- Lija de acabamiento proximal 2,5mm x 150 un
3062	- 0006020023	- Lija de acabamiento proximal 4mm x 150 un
3065	- 0006020004	- Lija de acabamiento proximal 2,5mm x 100 un
3066	- 0006020005	- Lija de acabamiento proximal 4mm x 100 un

TDV Dental Ltda.

CNPJ 81.591.786/0001-60
XV de Novembro, 9944 | CEP 89107-000
Pomerode/SC – Brasil
Resp. Técn. Tais Cúgola C. Pereira
CRF/SC 13884
Reg. ANVISA 10291220069
☎ 55 47 3395-6115
info@tdv.com.br | www.tdv.com.br



Obelis S.A
Bd. Général Wahis 53
1030 Brussels, Belgium
Tel: +(32) 2.732.59.54
E-mail: mail@obelis.net

Version 10 - 11/2020