DISINFECTION AND STERILIZATION PROCEDURES FOR ULTRADENT ADJUSTING, POLISHING, AND FINISHING INSTRUMENTS

Jiffy Polishers

Warning: Attention should be paid to the speed of work (RPM)

- Always refer to the product packaging for the Maximum RPM
- Operating a polisher with too high of an RPM may generate undesirable heat
- Operating a polisher with too high of an RPM may cause the polisher to fail

Cleaning and Sterilization

Scope

These instructions are applicable to all Ultradent® Dental Polishers. They are applicable before initial use and after each subsequent use. Devices are provided mechanically clean, but are not sterile. Therefore, they should be sterilized before first use.

Warnings

1. Cleaning agents with chlorine or chloride as the active ingredient are corrosive to stainless steel and must not be used. Cleaning agents with neutral pH are recommended.
2. Do not use cleaning agents or disinfectants containing alcohol.
3. Do not use Cold Sterilizing Methods for the sterilization of devices. These agents often contain strong oxidizing chemicals that may dull or weaken devices.

Reprocessing Limitations

The end of life is determined by wear and damage and the devices should be inspected for defects (i.e. broken tips, device deterioration, etc.) during the cleaning process.

Point of Use

Delay in reprocessing must be kept to a minimum to avoid contaminants drying thereby making cleaning more difficult.

Containment/ Transportation

Devices can be transported wet or dry and should be protected from damage. If transported wet there is an increased chance of staining or deterioration. Prolonged storage in disinfectant solutions may result in degradation of the product and must be avoided.

Manual Cleaning Procedure

If hand cleaning is the only available option, devices should be cleaned in a sink reserved for cleaning instruments.

Rinse the device (and dedicated instrument block, if applicable) under cool running water for at least one (1) minute.

Prepare a fresh bath of neutral-pH cleaning solution. Follow the agent's manufacturer's instructions. Immerse the device (and instrument block) and, keeping it immersed, brush thoroughly away from the body using the neutral cleaning agent. Care should be taken to avoid spreading contaminants by spraying or splashing during the brushing process.

Special care should be taken to clean crevices and other hard-to-reach areas thoroughly. Visually inspect to confirm the removal of debris. Repeat the cycle if needed.

Thoroughly rinse the device (and instrument block) under running warm water for at least one (1) minute and until visibly clean.

Dry the device using a non-shedding wipe or clean compressed air.

Automated Cleaning Procedure

Prepare a fresh pH-neutral cleaning solution; place the device in the dedicated instrument block (if applicable) and then place in a sonication unit. Follow the agent manufacturers' instructions for correct concentration, exposure time, temperature and water quality. Completely submerge the device in the cleaning solution and sonicate for at least fifteen (15) minutes.

Perform a final thorough rinse of the device and instrument block (if applicable) under running warm tap water for at least (1) minute. Visually inspect to confirm the removal of debris. Repeat the cycle if needed.

Dry the device using a non-shedding wipe or clean compressed air.

Inspection Testing

1. Carefully inspect each device to ensure that all debris has been removed.
2. Visually inspect the device for damage/wear that would prevent proper operation.
 - a. Do not use if the tip is broken
 - b. Do not use if there is deterioration in the material
 - c. Do not use if there is evidence of corrosion

Packaging

Singly: Pack the device in pouches validated for sterilization

In Sets: Place the device in the dedicated instrument block.

Sterilization

Use the following cycle for steam sterilization

Cycle Type	Sterilization Exposure Time (minutes)	Sterilization Exposure Temperature	Dry Time (minutes)
Gravity	10	135°C (275°F)	30
Pre-vacuum (4 Pulses)	5	134°C (273°F)	30

Ensure that the sterilizer manufacturer's maximum load is not exceeded.

Storage

The device should be stored in the sterilization pouch (or instrument block) until required.

Additional Information

These processes have been validated as being capable of preparing devices for reuse. Any deviation from these instructions should be properly validated for effectiveness and potential adverse results.

Diamond Burs and Discs

Cleaning and Sterilization

Scope

These instructions are applicable to all UPI Diamond Dental Burs and Discs. They are applicable before initial use and after each subsequent use. Diamond Burs and Discs are provided mechanically clean, but are not sterile (unless labeled "STERILE"). Therefore, Diamond Burs and Discs should be sterilized before first use.

Warnings

1. Cleaning agents with chlorine or chloride as the active ingredient are corrosive to stainless steel and must not be used. Cleaning agents with neutral pH are recommended.
2. Do not use Cold Sterilizing Methods for the sterilization of Diamond Burs or Discs. These agents often contain strong oxidizing chemicals that may attack the substrate that bonds the diamond particles to the steel blanks.

Reprocessing Limitations

The end of life is determined by wear and damage in use and Diamond Burs and Discs should be inspected for defects (i.e. broken tips, missing diamond particles, etc.) during the cleaning process.

Point of Use

Delay in reprocessing must be kept to a minimum to avoid contaminants drying thereby making cleaning more difficult.

Containment/ Transportation

Diamond Burs and Discs can be transported wet or dry and should be protected from damage. If transported wet there is an increased chance of staining or corrosion. Prolonged storage in disinfectant solutions may result in degradation of the product and must be avoided.

Manual Cleaning Procedure

If hand cleaning is the only available option, Diamond Burs and Discs should be cleaned in a sink reserved for cleaning instruments.

Optional step: Insert the Diamond Bur into a handpiece and operate at the Maximum RPM. Insert the Diamond Bur head fully into a wet diamond cleaning stone for at least 2 seconds. Remove Diamond Bur from handpiece.

Rinse the Diamond Bur or Disc (and dedicated instrument block, if applicable) under cool running water for at least one (1) minute.

Prepare a fresh bath of neutral-pH cleaning solution. Follow the cleaning agent's manufacturer's instructions. Immerse the Diamond Bur or Disc (and instrument block) and soak for at least ten (10) minutes.

After soaking, and keeping it immersed, brush thoroughly away from the body using the neutral cleaning agent for at least one (1) minute. Care should be taken to avoid spreading contaminants by spraying or splashing during the brushing process. Use wire brushes with caution as brass particles may result in galvanic corrosion and steel particles may cause discoloration of stainless steel.

Special care should be taken to clean crevices and other hard-to-reach areas thoroughly. Visually inspect to confirm the removal of debris. Repeat the cycle if needed.

Thoroughly rinse the Diamond Bur or Disc (and instrument block) under running warm water for at least one (1) minute and until visibly clean.

Dry the device using a non-shedding wipe or clean compressed air.

Ultrasonic Cleaning Procedure

Prepare a fresh pH-neutral cleaning solution; place the Diamond Bur or Disc in the dedicated instrument block (if applicable) and then place in a sonication unit. Follow the cleaning agent manufacturers' instructions for correct concentration, exposure time, temperature, and water quality. Completely submerge the device in the cleaning solution and sonicate for at least fifteen (15) minutes.

Perform a final thorough rinse of the device and instrument block (if applicable) under running warm tap water for at least (1) minute.

Visually inspect to confirm the removal of debris. Repeat the cycle if needed.

Dry the device using a non-shedding wipe or clean compressed air.

Inspection Testing

1. Carefully inspect each device to ensure that all debris has been removed.
2. Visually inspect the device for damage/ wear that would prevent proper operation.

- a. Do not use if the tip is broken.
- b. Do not use if there are missing or worn diamond particles.
- c. Do not use if there is evidence of corrosion.

Packaging

Singly: Pack the Diamond Bur or Disc in pouches validated for sterilization
In Sets: Place the Diamond Bur or Disc in the dedicated instrument block.

Polishing Brushes

General Instructions

1. Instruments must be disinfected, cleaned and sterilized before and after each use according to appropriate and professional procedures.
2. Instruments must be pre-disinfected immediately after use using a bur sterilizer by complete immersion for a holding period defined by the manufacturer or the disinfecting solution.

Cleaning and Sterilization

Manual Cleaning Procedure

Rinse instruments thoroughly in running water.

The subsequent mechanical cleaning (e.g. brush, running water) will remove any tissue residue. Best results are achieved using a treatment in ultrasonic bath.

Sterilization

For sterilization we recommend using an autoclave at +134°C with 2,2 bar pressure and a holding time of at least 5 minutes. Treatment at temperatures exceeding 180°C is not suitable and should not be used. Please observe the instructions for use of the manufacturer of the equipment and materials used.

Carbide Burs

General Instructions

1. Clean and sterilize non-sterile burs in accordance with the validated procedures provided below prior to first use and prior to each reuse.
2. Do not force a bur into the handpiece. In case of difficult access, check both handpiece turbine and bur and refer to handpiece instructions for troubleshooting.

Cleaning and Sterilization

Scope

These instructions are applicable to all UPI Carbide Dental Burs. They are applicable before initial use and after each subsequent use. Carbide Burs are provided mechanically clean, but are not sterile (unless labeled "STERILE"). Therefore, Carbide Burs should be sterilized before first use.

Warnings

1. Cleaning agents with chlorine or chloride as the active ingredient are corrosive to stainless steel and must not be used. Cleaning agents with neutral pH are recommended.
2. Do not use Cold Sterilizing Methods for the sterilization of Carbide Burs. These agents often contain strong oxidizing chemicals that may dull or weaken Carbide Burs.

Reprocessing Limitations

The end of life is determined by wear and damage in use. Carbide Burs should be inspected for defects (i.e. broken tips, broken sections on flutes, etc.) during the cleaning process.

Point of Use

Delay in reprocessing must be kept to a minimum to avoid contaminants drying thereby making cleaning more difficult.

Containment/ Transportation

Carbide Burs can be transported wet or dry and should be protected from damage. If transported wet there is an increased chance of staining or corrosion. Prolonged storage in disinfectant solutions may result in degradation of the product and must be avoided.

Manual Cleaning Procedure

If hand cleaning is the only available option, Carbide Burs should be cleaned in a sink reserved for cleaning instruments.

Rinse the Carbide Bur (and dedicated instrument block, if applicable) under cool running water for at least one (1) minute.

Prepare a fresh bath of neutral-pH cleaning solution. Follow the cleaning agent's manufacturer's instructions. Immerse the Carbide Bur (and instrument block) and soak for at least ten (10) minutes.

After soaking, and keeping it immersed, brush thoroughly away from the body using the neutral cleaning agent for at least one (1) minute. Care should be taken to avoid spreading contaminants by spraying or splashing during the brushing process. Use wire brushes with caution as brass particles may result in galvanic corrosion and steel particles may cause discoloration of stainless steel.

Special care should be taken to clean crevices and other hard-to-reach areas thoroughly. Visually inspect to confirm the removal of debris. Repeat the cycle if needed.

Thoroughly rinse the Carbide Bur (and instrument block) under running warm water for at least one (1) minute and until visibly clean.

Dry the device using a non-shedding wipe or clean compressed air.

Ultrasonic Cleaning Procedure

Prepare a fresh pH-neutral cleaning solution; place the Carbide Bur in the dedicated instrument block (if applicable) and then place in a sonication unit. Follow the cleaning agent manufacturers' instructions for correct concentration, exposure time, temperature, and water quality. Completely submerge the device in the cleaning solution and sonicate for at least fifteen (15) minutes.

Perform a final thorough rinse of the device and instrument block (if applicable) under running warm tap water for at least (1) minute.

Visually inspect to confirm the removal of debris. Repeat the cycle if needed.

Dry the device using a non-shedding wipe or clean compressed air.

Inspection Testing

1. Carefully inspect each device to ensure that all debris has been removed.
2. Visually inspect the device for damage/ wear that would prevent proper operation.
 - a. Do not use if the tip is broken.
 - b. Do not use if there is a broken section of a flute.
 - c. Do not use if there is evidence of corrosion.

Packaging

Singly: Pack the Carbide Bur in pouches validated for sterilization

In Sets: Place the Carbide Bur in the dedicated instrument block.

Sterilization

Use the following cycle for steam sterilization

Cycle Type	Minimum Sterilization Exposure Time (minutes)	Minimum Sterilization Exposure Temperature	Minimum Dry Time (minutes)
Gravity	10	135°C (275°F)	30
Pre-vacuum (4 Pulses)	3	134°C (273°F)	30

Ensure that the sterilizer manufacturer's maximum load is not exceeded.

The minimum dry time has been validated to ensure that the devices will not be left wet. Failure to achieve the minimum dry time may cause moisture to remain on the files that could result in corrosion.

Storage

The Carbide Bur should be stored in the sterilization pouch (or instrument block) until required.

Additional Information

These processes have been validated as being capable of preparing Carbide Burs for reuse. Any deviation from these instructions should be properly validated for effectiveness and potential adverse results.

Jiffy Strips

General Instructions

1. Abrasive Strips are to be used on the instruction of, or by a dentist or other licensed practitioner.
2. Clean and sterilize prior to first use and each subsequent reuse.
3. Do not store when wet as this may cause corrosion of the stainless steel or interaction with other stored instruments.

Cleaning and Sterilization

Scope

These instructions are applicable to all REUSABLE UPI Abrasive Strips made from stainless steel. They are applicable before initial use and after each subsequent use. Abrasive Strips are provided mechanically clean, but are not sterile. Therefore, Abrasive Strips should be cleaned and sterilized before first use and subsequent reuses.

Warnings

1. Cleaning agents with chlorine or chloride as the active ingredient are corrosive to stainless steel and must not be used.
2. Do not use cold sterilizing methods for the sterilization of Diamond-Coated Strips. These agents often contain strong oxidizing chemicals that may attack the substrate that bonds the diamond particles to the steel blanks.

Reprocessing Limitations

The end of life is determined by wear and damage in use and REUSABLE Abrasive Strips should be inspected for defects (i.e. missing diamond particles, broken edges, etc.) during the cleaning process.

Point of Use

Delay in reprocessing must be kept to a minimum to avoid contaminants drying thereby making cleaning more difficult.

Containment/ Transportation

REUSABLE Abrasive Strips can be transported wet or dry and should be protected from damage. If transported wet there is an increased chance of staining or corrosion of the stainless steel. Prolonged storage in disinfectant solutions may result in degradation of the product.

Manual Cleaning Procedure

If hand cleaning is the only available option, REUSABLE Abrasive Strips should be cleaned in a sink reserved for cleaning instruments.

Rinse the device under cool running water for at least one (1) minute.

Prepare a fresh bath of neutral-pH cleaning solution (such as Enzol) following the manufacturer's directions. Immerse the device and soak for at least ten (10) minutes.

After soaking, and keeping it immersed, brush thoroughly away from the body using the neutral cleaning agent for at least one (1) minute. Care should be taken to avoid spreading contaminants by spraying or splashing during the brushing process. Use wire brushes with caution as brass particles may result in galvanic corrosion, which may cause discoloration of stainless steel.

Special care should be taken to thoroughly clean perforations, crevices and other hard-to-reach areas using a pH neutral cleanser. Visually inspect to confirm the removal of gross debris. Repeat the cycle if needed.

Thoroughly rinse the device under running warm water for at least one (1) minute and until visibly clean.

Dry the device using a non-shedding wipe or clean compressed air.

Ultrasonic Cleaning

Prepare a fresh pH-neutral cleaning solution following the manufacturer's instructions for correct concentration, Procedure exposure time, water temperature and quality. Place the device in a sonication unit ensuring the device is completely submerged, and sonicate for at least fifteen (15) minutes.

Perform a final thorough rinse of the device under running warm tap water for at least (1) minute.

Visually inspect to confirm the removal of gross debris from perforations, crevices and other hard to reach areas. Repeat the cycle if needed until visibly clean.

Dry the device using a non-shedding wipe or clean compressed air.

Inspection Testing

1. Carefully inspect each device to ensure that all debris has been removed.
2. Visually inspect the device for damage/ wear that would prevent proper operation.
 - a. Do not use if Abrasive strip is dull, chipped, broken, missing diamond particles or is damaged in any manner
 - b. Do not use if Abrasive strip exhibits signs of corrosion or damage

Packaging

Singly Pack REUSABLE Abrasive Strips in pouches validated for sterilization

In Sets: Pack select REUSABLE Abrasive Strips into the dedicated Bur Block for steam sterilization

Sterilization

Abrasive Strips must be sterilized using the following validated steam sterilization parameters ONLY:

Cycle Type	Minimum Sterilization Exposure Time (minutes)	Minimum Sterilization Exposure Temperature	Minimum Dry Time (minutes)
Gravity	10	135°C (275°F)	30
Pre-vacuum (4 Pulses)	3	134°C (273°F)	30

Ensure that the sterilizer manufacturer's maximum load is not exceeded.

Storage

The REUSABLE Abrasive Strip should be stored in the sterilization pouch until required. The Bur Block is not intended to maintain the sterility of the contained devices.

Additional Information

These processes have been validated as being capable of preparing REUSABLE Abrasive Strips for reuse. Any deviation from these instructions should be properly validated for effectiveness and potential adverse results.

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Manufactured for Ultradent Products Inc.
Ultradent Products GmbH