# PermaFlo®

Flowable Composite  
Dickflüssiges Komposit  
Ciment dentaire composite fluide  
Sroombaar composiet

Composito scorporevole  
Composite Fluido  
Composito fluide  
Flytende komposit

Flytende komposit  
Jukovaya yidisleimamovi  
Projevo otvrdno  
Жуковий компазит

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## General Precautions:

- 1. For Professional use only.
- 2. Follow instructions, precautions, and MSDS before beginning treatment. Use only as directed.
- 3. Keep products out of reach/sight.
- 4. Avoid skin exposure to resin.
- 5. Dilute strong chemicals to area of treatment.
- 6. Confirm that patient has no known allergies to treatment materials.
- 7. Test flow of material on wax or glass and tip before using intraorally.
- 8. Never force syringe plungers.

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Ultrafren syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers. The letter is a lot number symbol and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.

For immediate reader and/or complete descriptions of Ultrafren's product line, refer to Ultrafren's catalog or call toll free 1-800-552-5512. Outside U.S. call (801) 572-4200 or visit [www.ultrafren.com](http://www.ultrafren.com).

**PermaFlo®**  
Flowable Composite

Manufactured by:  
Ultrafren Products Inc  
505 West 10200 South  
South Jordan, Utah 84095 USA

**For product SDS please see our website:  
[www.ultrafren.com](http://www.ultrafren.com)**

**CE REP** Ultrafren Products GmbH,  
Am Westhofer Berg 30,  
51149 Cologne Germany.

**ULTRADENT**  
PRODUCTS, INC.

© Copyright 2011 Ultrafren Products, Inc. All Rights Reserved. 94800-U-9 082715

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**PermaFlo®**  
Flowable Composite

Great Translucency! 68% filled!

Description:  
PermaFlo™ is a light cured, radiopaque, methacrylate-based flowable composite. PermaFlo contains ~1µm average particle size with narrow upper limit particle distribution. PermaFlo is available in 9 shades. It provides superior polishability (particularly for a hybrid and superior excellent translucency for matching, blending, and depth of cure. It is thixotropic, 68% filled mass to mass, and has ~6µm fill particle thickness.

Indications:  
PermaFlo can be used for anterior and posterior restorations. It is also used for restoring missing subgingival tooth structure (where light cure is accessible) before endodontic procedures (referred to as the "donut technique"). It is highly recommended for Class III and V restorations, or for the lower layers when using incremental building of composites.

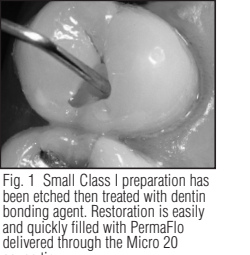
Fig 1. Small Class I preparation has been etched then treated with dentin bonding agent. Restoration is easily and quickly filled with PermaFlo delivered through the Micro 20 gauge tip.

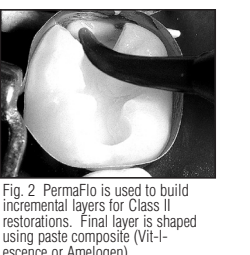
Fig 2. PermaFlo is used to build incremental layers for Class II restorations. Final layer is shaped using paste composite (Vit-I-essence or Amelogen).

Procedure:  
1. Remove luer lock cap and attach a disposable Micro 20 gauge tip securely onto syringe. This tip can be used when intimate adaptation is required in narrow angles (e.g. class I, V, and proximal line angles of class II).  
2. For dentin bonded restorations, etch dentin/enamel 20 seconds using Ultra-Etch. Wash 5 seconds. Blow excess water off (leaving surface visibly wet). Apply Peak Universal Bond to preparation and air thin until milky appearance of filler disappears, maintaining a high gloss surface and no dry spots. Light cure 20 seconds if using a standard halogen light, 10 seconds if using a high energy light. Apply a thin layer of PermaFlo to ensure complete polymerization at pulpal

floor areas.  
3. For matching posteriors, or in areas of deeper layers, use PermaFlo shades closer to or slightly lighter than dentin. Use Translucent or other appropriate shades to match enamel for last increment. PermaFlo has tremendous chameleon properties. Apply first layer of PermaFlo 0.5-1.5mm and light cure. Follow with 2-4mm increments. Cure each resin layer 20 seconds with a high-quality curing light. Place light guide as close as possible to composite without touching. For sculptability and highest wet, use an appropriate paste composite for your last layer (e.g., Vit-I-essence™ or Amelogen™).

4. Shape with appropriate finishing diamonds or multi-fluted carbide burs. Finish with strips, polishing cups, and wheels or disks as appropriate. We recommend Jiffy™ Polishing system for shine. An excellent luster may be obtained with Ultrafren™ Diamond Polish.  
5. Use PermaFlo for Class V restorations as shown in Figs. 3-7.

a. Etch dentin/enamel 20 seconds using Ultra-Etch. Wash 5 seconds. Blow excess water off (leaving surface visibly wet).  
b. Apply Peak Universal Bond to preparation and air thin until milky appearance of filler disappears. Maintaining a high gloss surface and no dry spots. Light cure 20 seconds if using a standard halogen light, 10 seconds if using a high energy light.  
c. Restore using PermaFlo delivered through the Micro 20 Gauge tip. Light cure 20 seconds.  
6. Use PermaFlo for the "donut technique" to achieve extension of root when retention of clamp would otherwise be difficult or impossible (Fig. 7). Make sure area is accessible to light from curing tip. a. Isolate tooth with UltraFrac™ knitted displacement cone to ensure fluid control before hybrid layer formation.  
b. Prepare dentin for bonding by etching with Ultra-Etch, then applying Peak Universal Bond. Light cure.  
c. Place donut-shaped layers of PermaFlo. Cure between each layer.  
d. If using during root canal treatment, UltraTemp is recommended as temporary cement allowing easy access to complete the root canal. Use of the more opaque shades of PermaFlo to contrast with dentin, aiding in removal of material that extend onto root.  
e. Donut build-up functions as a matrix for composite core with direct post placement.

The Inside of the PermaFlo donut is freshened with a diamond bur and treated with PermaFlo™ DC Primers to maximize adhesion of composite core. Finish with PermaFlo™ DC composite luting/restorative resin.

Precautions:  
1. Partially cured material will have deficient physical properties, including strength, wear resistance, retention, marginal integrity and color stability. Incompletely cured material could also cause pulp irritation. Use only high-quality curing units and check units periodically with a light meter.  
2. Restoration necessitates extensive shaping and sculpting for direct veneers, minimize light exposure from overhead light unit until all material is placed to prevent premature set.  
3. The oxygen inhibition layer should not be removed or disturbed during incremental layering because it optimizes the bond to each succeeding layer.  
4. Placing posterior composite correctly is much more technique-sensitive than for amalgams. Remember to polymerize interproximal areas adequately and to check radiographically at regular checkup intervals.  
5. Any resin can be sensitizing to skin or can cause allergic reaction if individual is already sensitized. Therefore, wash area of skin exposed to resin completely to remove material.  
If dermatitis, rash or any other allergic reaction is observed, see a physician.  
6. Syringe caps should be replaced following use to prevent polymerization (as composite resins are light activated).  
7. Do not reuse tips to avoid cross-contamination.  
8. Use Ultrafren™ Syringe Covers and/or clean and disinfect syringes between patient.

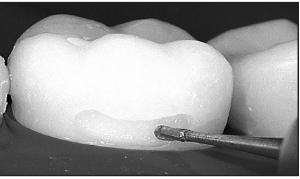
Fig 3. Class V surface is prepared.

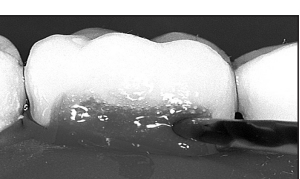
Fig 4. Etch 20 seconds.

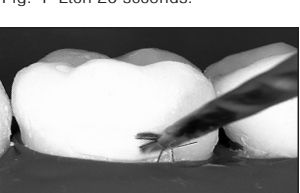
Fig 5. Apply dentin / enamel bonding agent(s).

Fig 6. Restore with PermaFlo.

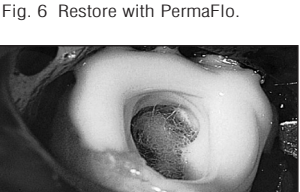
Fig 7. Clamp Retention: After dentin bonding agent, place donut shaped layers of PermaFlo one upon another. Cure between each layer. Clamp, retention and seal are ensured now that missing tooth structure has been replaced.