Structur® 2 SC/QM

Carefully read instructions prior to use

Product description:

Structur 2 SC/QM is a fluorescent cold polymerizing paste-paste system for the production of temporary crowns, bridges, inlays and onlays.

Structur 2 SC/QM consists of base paste and catalyst paste.

Structur 2 SC is supplied in 8 shades: A1, A2, A3, A3.5, B1, B3, C2 and BL.

Structur 2 QM is supplied in shades: A1, A2, A3 and B1.

Indications:

Structur 2 SC/QM is indicated for the fabrication of temporary crowns, bridges, inlays and onlays.

Contraindications:

Structur 2 SC/QM contains methacrylates, amines, terpenes, benzoyl-peroxide, BHT. **Structur 2 SC/QM** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.

Instructions:

Structur 2 SC in the cartridge:

Insert the **Structur 2 SC** cartridge into the dispenser type 2 or a compatible dispenser with a suitable piston. Due to technical reasons, slight differences in the filling level between both cartridge chambers may occur. In this case, remove cartridge cap and squeeze out the material until it is delivered evenly from both chambers. Then, insert a mixing tip type 6 and lock with a 90° clockwise turn. By pressing the lever, the material is extruded and automatically mixed in the correct ratio.

The warranted product characteristics can only be achieved when using the original Sulzer Mixpac mixing tips as approved by **VOCO**. All the mixing tips for use with our respective products are available from **VOCO**.

Do not remove the mixing tip from the cartridge after use. Replace the used mixing tip with a new one just before the next application. Check the cartridge opening to ensure material flowability when changing the mixing tips.

Structur 2 QM in the QuickMix syringe:

Due to technical reasons slight differences in the filling level between the syringe chambers can occur prior to the first use. Therefore remove syringe cap and squeeze the material out until the material is delivered evenly from both chambers. Attach a mixing tip type 10. Match the nooks on the syringe to the mixing tip and lock the mixing tip into place with a 90° clockwise turn. The material is automatically mixed in the correct ratio when dispensing and can be applied directly.

All QM syringes come with a detachable piston. The piston can simply be pulled from the QM syringe and replaced without causing any material leakage. The plugs in the syringe chambers will stay in place preventing any cross-contamination of the material in the syringe.

Application:

Level out any undercuts in the preparation. Use only eugenol-free cements. In cases where no prefabricated crowns or similar restorations are used, prepare an impression by levelling undercuts and cutting drainage channels. If the wall of the temporary restoration is too thin, enlarge the impression, especially removing septums in the interdental embrasures. Then clean and air dry the impression.

Apply **Structur 2 SC/QM** directly into the impression. Do not overfill.


Structur 2 SC/QM attains an elastic phase after 1 - 1.5 min from beginning of mixing (approx. 0.5 - 1 min after inserting into the mouth).

Remove the temporary from the mouth no later than 1.5 min (elastic phase) after mixing, to ensure a trouble-free removal.

The setting process can additionally be monitored by checking the excess material in the mouth.

The setting time of non-used material (in mixing tips) **cannot** be compared with the setting time of the material in the mouth.

At a room temperature of 23°C and 50% humidity, working times are as follows:

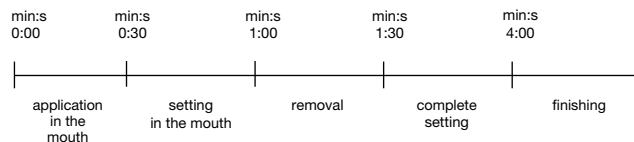Slight differences may result from variations in room temperatures.

After approx. 2 - 3 min, remove the temporary from the impression and wipe dry. If necessary, use a solvent, e.g. ethyl alcohol. This removes the thin inhibition layer caused by oxygen and allows easier finishing. The surface of the temporary restoration should appear silky-matt.

Carefully trim any excess material and roughen areas near the gingiva. Cement the temporary restoration using an eugenol-free temporary cement.

Caution:

- Before making the temporary, the prepared cores should be insulated for desensitization (fluoride varnishes, core varnishes, etc.)

- In order to avoid fractures or injuries from debris, the temporary should not be exposed to excessive loads (molar area).
- Linings and cores built up with resin-based materials (composites) should be sufficiently insulated (e.g. glycerine) for easy separation of the temporary from composite resin material.
- Inadequate hygiene or intensive use of some mouthwashes may lead to discoloration.
- Finishing should only be started after complete setting of the material (approx. 4 min after mixing).

Repairs:

If the temporary work breaks despite **Structur 2 SC/QM**'s extreme stability, the following procedures are recommended:

- **Air bubbles:** fill with **Structur 2 SC/QM**.
- **In case of fracture immediately after preparing the temporary:** re-connect the fracture with freshly mixed **Structur 2 SC/QM**.
- **Restoration of already used temporaries:** Roughen the fracture with a bur and place undercuts. Apply a bonding agent to the prepared fractures and cure according to the respective instructions for use. Apply a flowable composite to the prepared areas. Bond the fractured pieces together and cure according to the instructions for use.

Storage:

Store at temperatures between 39°F - 73°F (4°C - 23°C). Do not use after expiry date.

Do not expose Structur 2 SC/ QM to direct sunlight or the chairlight.

Product Ordering Information:

Structur 2 SC

Intro kit 75gm cartridge A2, mixing tips type 6, dispenser type 2	REF 1499
Intro kit 75gm cartridge A3, mixing tips type 6, dispenser type 2	REF 1478
pack, shade A1 75gm cartridge, mixing tips type 6	REF 1479
pack, shade A2 75gm cartridge, mixing tips type 6	REF 1480
pack, shade A3 75gm cartridge, mixing tips type 6	REF 1481
pack, shade A3.5 75gm cartridge, mixing tips type 6	REF 1490
pack, shade B1 75gm cartridge, mixing tips type 6	REF 1482
pack, shade B3 75gm cartridge, mixing tips type 6	REF 1483
pack, shade C2 75gm cartridge, mixing tips type 6	REF 1486
pack, shade BL 75gm cartridge, mixing tips type 6	REF 1485
bulk pack, shade A2 5 x 75gm cartridges, mixing tips type 6	REF 1492
bulk pack, shade A3 5 x 75gm cartridges, mixing tips type 6	REF 1491
mixing tips for Structur 2 SC , 50 pieces, type 6	REF 2185
dispenser type 2	REF 2158

Structur 2 QM

pack, shade A1 8gm QuickMix syringe, mixing tips type 10	REF 1309
pack, shade A2 8gm QuickMix syringe, mixing tips type 10	REF 1310
pack, shade A3 8gm QuickMix syringe, mixing tips type 10	REF 1311
pack, shade B1 8gm QuickMix syringe, mixing tips type 10	REF 1312
mixing tips for Structur 2 QM , 50 pieces, type 10	REF 2189

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. **VOCO** recognizes its responsibility to replace products if proven to be defective. **VOCO** does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith. Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

Keep this material out of reach of children.

For dental use only.

An explanation of the symbols used in labeling can be found at www.voco.dental/symbols

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VOCO GmbH Phone +49 (4721) 719-0