Bifix QM

Carefully read instructions prior to use

Bifix QM is a radiopaque, dual-curing resin-based luting system for durable adhesive fixing of pre-treated metal, ceramic and synthetic pieces. **Bifix QM** is available in three shades: universal, transparent and white-opaque. QuickMix syringes deliver bubble-free, perfectly mixed material that can be directly applied from the syringe. The **Bifix QM** system achieves excellent adhesion with suitable dentine/enamel bonds (**Futurabond DC**) and **Ceramic Bond** for silanization of the restoration.

Indications:

Adhesive luting of inlays, onlays, facettes, veneers, crowns, partial crowns, pins, posts, bridges and adhesive bridges (Maryland bridges).

Contraindications:

Bifix QM contains BIS-GMA, benzoyl peroxide and amines. Do not use in patients with known allergies or sensitivities to these ingredients. In case of contact with the eyes, immediately rinse with plenty of clean water and consult a doctor.

Bifix QM is not suitable for fixing brackets due to its high adhesion values.

Method of use:

Please consult the instructions for use for **Futurabond DC** and **Ceramic Bond** for detailed information and application steps for each product.

1. Preparation

Remove the temporary if applicable. Clean, thoroughly rinse and slightly dry the tooth with air.

Note

The work area must be kept clean and dry. A rubber dam should be applied. Teeth covered with a rubber dam may dry out which can slightly influence the impression of the shade.

2. Tooth surfaces

Protect dentin in close proximity to the pulp with a suitable lining, e.g. calcium hydroxide and/or glass ionomer cement.

3. Conditioning the restoration

a. Full ceramic restorations

The inside adhesive surfaces of the restorative should be micro-etched with a hydro-fluoric acid etchant. Normally, the dental laboratory has already done this. Etch the restoration for 15 sec with a phosphoric acid gel, then rinse and dry. Disperse **Ceramic Bond** (silane) in a mixing well. Apply **Ceramic Bond** to the inside of the restorative with a brush and allow to work for 60 sec. Afterwards, dry for 5 sec with oil-free air. Do not touch or otherwise contaminate the conditioned surfaces.

b. Metal, composite and zirconium dioxide restoratives

Sandblast the surfaces with maximum 1.5 – 2 bar pressure and a suitable agent. Follow the manufacturer's instructions for use for the respective material. Carefully remove the blasting abrasive residue and dry the adhesive surfaces. Apply **Ceramic Bond** with a brush to the conditioned surfaces, allow 60 s for reaction and then carefully dry with oil-free air. Do not touch the adhesive surfaces afterwards and quickly lute the restoration with **Bifix QM** while observing the following points.

c. Chairside luting of inlays

Clean the bonding surface with alcohol and dry with oil-free air. Do not touch the adhesive surface thereafter – insert and bond the restoration in accordance with the following points.

4. Application of the bonding

Using **Futurabond DC** is recommended for bonding **Bifix QM**. Follow instructions for use for the respective bond.

5. Luting the restorative with Bifix QM

Insulate any removable prosthodontic elements before luting. Only mix **Bifix QM** immediately before luting, as described in the following: Due to technical reasons slight differences in the filling level between the syringe chambers can occur prior to the first use. Therefore remove syringe cap and squeeze the material out until the material is delivered evenly from both chambers. Attach a mixing tip type 11. The notches on the syringe and the mixing tip must line up exactly. Turn the mixing tip 90° clockwise to lock the cannula into place. The material is automatically mixed in the mixing tip and can be directly applied onto the prepared contact areas. If needed, the optional intraoral tip type 4 can be placed onto the mixing tip to directly apply the material onto the tooth or prepared root canal.

Working time under reduced light is approximately 2.5 - 3.5 min. Setting time is 3 min from the time of inserting the prosthodontic piece. Insert the piece and fix it with slight pressure. Remove excess material with a foam pellet or a disposable brush. Using dental floss is recommended for removing excess in interdental spaces. Leave a slight excess (see below).

Advice for luting Maryland bridges

For luting Maryland bridges, the enamel must either be roughened or conditioned with a phosphoric acid etching gel before applying **Futurabond DC**.

6. Luting of fibre-reinforced resin posts (in combination with Futurabond DC):

1. Massage the mixed **Futurabond DC** (liquid 1 and 2) into the occlusal surface of the core and root canal for 20 sec with an applicator. Disperse solvent with oil-free air for 5 sec. Remove any material excess in the canal with an absorbent strip of paper. Do **not** light-cure the bonding layer.
2. It is recommended to apply **Bifix QM** directly into the root canal using thin intraoral tips type 1. Do not use a lentulo! After preparing the root pin according to the manufacturer's instructions, it can be additionally moistened with **Bifix QM**. Insert the post making sure to leave a slight excess.
3. To fix the post and cure the **Futurabond DC** applied to the occlusal surface, light-cure for at least 40 sec.

7. Light-curing

The polymerization time can be controlled through additional light-curing. Light-cure the material at the luting gap with a halogen or LED light for a minimum of 20 s per segment. The light output should be a minimum of 500 mW/cm² on halogen polymerization devices and 300 mW/cm² on LED devices. Hold the tip of the light rod as close as possible to the surface to be cured. The restorative piece must be fixed in its desired position with a suitable instrument during light-curing.

Advice on oxygen inhibition:

Like all resin-based materials, **Bifix QM** surfaces in contact with oxygen do not cure completely (approximately 50 µm inhibited layer). Therefore, using one of the two following methods is recommended:

- a. Leave a slight excess and remove it during the subsequent finishing; or
- b. Remove the excess completely and cover the margins of the restorative piece with oxygen-impermeable protective gel.

8. Finishing/Fluoridation

The removal of excess is possible immediately after polymerization. Remove cured excess with a finishing diamond or flexible polishing disk. Finish approximate areas with finishing and polishing strips. Enamel surfaces which have been excessively etched or damaged during finishing should then be fluoridated.

Precautionary measures:

Only use suitable bonding materials.

Some enamel/dentin bonding agents are not compatible with dual-curing composites. When in doubt, consult the manufacturer's instructions for use for the respective bond. For best results, use **Futurabond DC**.

Irritation to the pulp may occur with insufficient dentin thickness (less than 1 mm). Pulp protection or core build-up is recommended.

Residue from thymol, eugenol and/or other preparations containing oil of cloves disturbs the curing process and should therefore be avoided. Only eugenol-free temporary cements should be used for temporary luting.

The warranted product characteristics can only be achieved when using the original Sulzer Mixpac mixing tips as approved by **VOCO**. All the mixing tips for use with our respective products are available from **VOCO**.

Storage:

Store at temperatures between 39°F - 73°F (4°C - 23°C). Refrigeration is recommended. Allow **Bifix QM** to reach room temperature before use. Do not use after the expiry date.

Ordering information:

Bifix QM

Set 10gm QuickMix syringe universal,		
Futurabond DC 4ml Liquid 1 / 4ml Liquid 2, 5ml Ceramic Bond ,		
mixing tips type 11, intraoral tips type 4, accessories		REF 1217
Pack 10gm QuickMix syringe universal,		
mixing tips type 11, intraoral tips type 4, accessories		REF 1218
Pack 10gm QuickMix syringe transparent,		
mixing tips type 11, intraoral tips type 4, accessories		REF 1219
Pack 10gm QuickMix syringe white-opaque,		
mixing tips type 11, intraoral tips type 4, accessories		REF 1759
Pack mixing tips type 11, 50 pieces		REF 2191
Pack intraoral tips type 4, 50 pieces		REF 2142
Refill 5ml Ceramic Bond		REF 1106

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use.

VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith. Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

Keep this material out of reach of children.

For dental use only.

An explanation of the symbols used in labeling can be found at www.voco.dental/symbols

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VOCO GmbH