



NEW
PINA COLADA



VOCO Profluorid® Varnish

FLUORIDE-CONTAINING DENTAL DESENSITISING
VARNISH



VOCO Profluorid® Varnish

HIGHLY EFFECTIVE



Sensitive teeth are a widespread problem for a large number of people. Usually, the condition causes only transient pain which patients do not always report to their dentists. Hypersensitivity is mainly caused by exposed dentinal tubules. Therefore, the aim of treatment is to seal these tubules. Profluorid Varnish is a fluoride-containing varnish to effectively seal dentinal tubules exposed on the necks of teeth or after cavity preparation.

Components and effect of VOCO Profluorid® Varnish

Profluorid Varnish is a colophony-based varnish containing 5 % sodium fluoride (22,600 ppm fluoride). The fluoride ion, together with the calcium ions accumulated in the tubules, causes a precipitation of calcium fluoride, which effectively seals the tubules. According to a user survey, over 85 % of patients were completely free of pain after only one treatment with Profluorid Varnish.

As well as rapidly sealing the dentinal tubules, Profluorid Varnish also causes calcium fluoride deposits to form on the tooth surface and contributes to the formation of fluorapatite in the long-term. The fluorapatite protects the tooth from acid attacks.

Xylitol, which is said to have a cariostatic effect in addition to its taste-enhancing properties, is also added to the varnish.

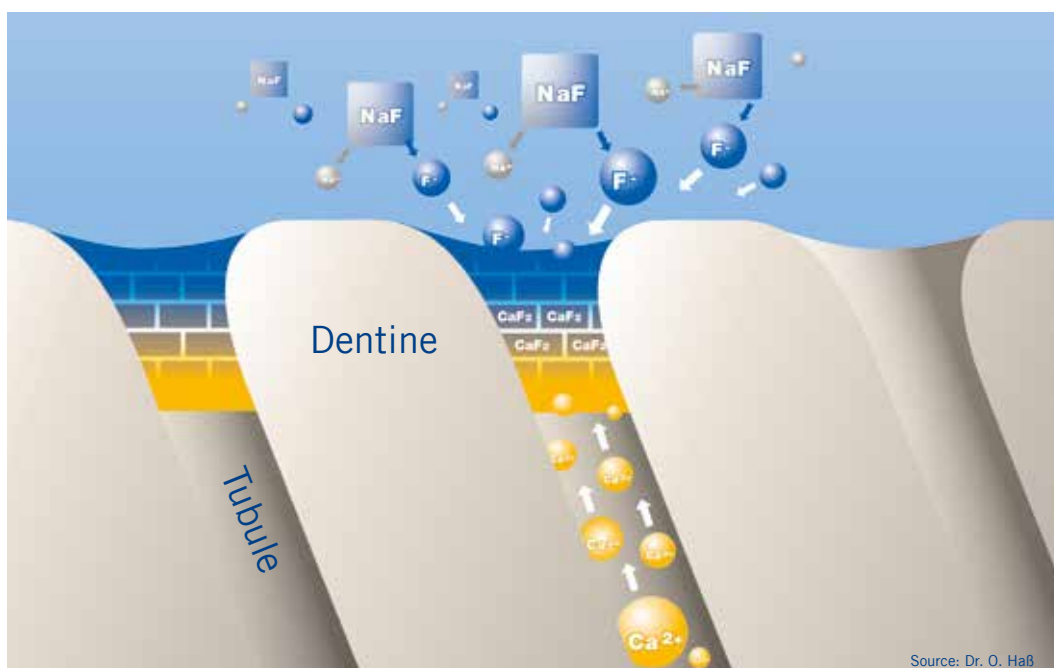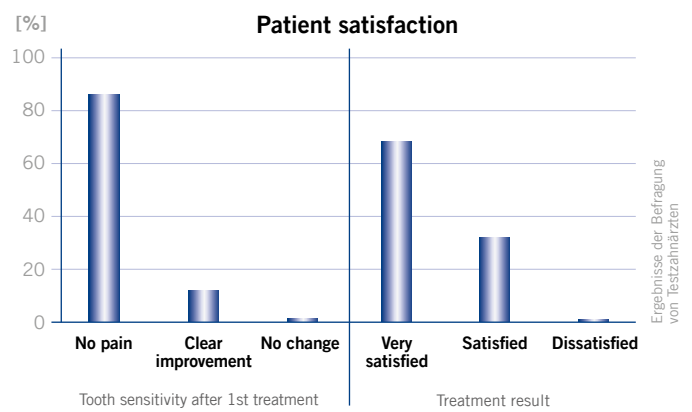

VOCO Profluorid® Varnish


SIMPLE AND AESTHETIC

Easy to apply

Due to the rosin matrix Profluorid Varnish tolerates moist surfaces. A previous complete drying of the areas to be treated is not necessary, so that the application is very time-saving. This improves cooperation of patients with low levels of compliance (e.g. children).

Aesthetics and taste

Profluorid Varnish is a tooth-shaded varnish which does not impair aesthetics. Profluorid Varnish is available in pleasant flavours: bubble gum and cherry – aimed especially at children and younger patients, mint – a classic, particularly for adults, caramel as a creamy flavour, melon and pina colada – to provide a slightly more exotic option and the fruity tingly freshness of cola lime. These refined flavours will enable you to provide an even better service by tailoring it to the individual patient.

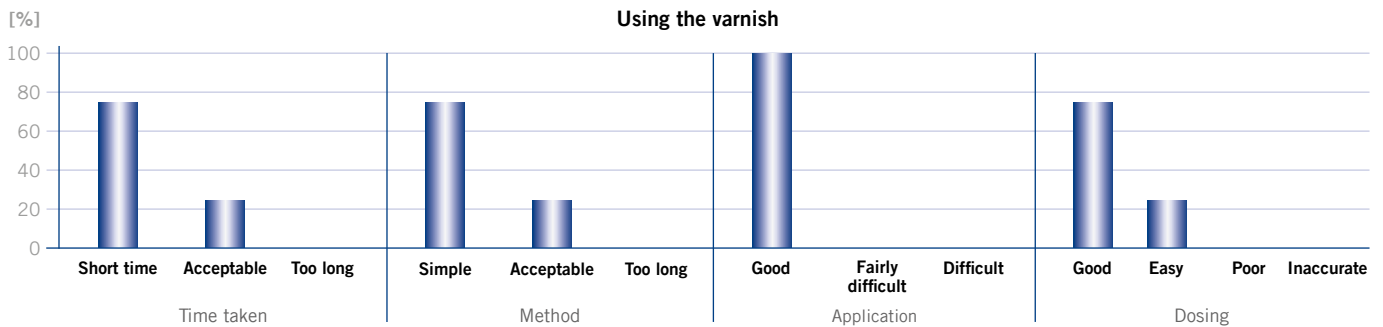

Survey of testing dentists

Opinion of dentists

Dentists using the product all confirmed its good handling and aesthetic properties. A vast majority also reported positive differences between Profluorid Varnish and competing products. Profluorid Varnish was equally well accepted by the patients.

After the treatment, nearly all of the patients reported that they were either symptom-free or had a significantly reduced level of sensitivity.

Profluorid Varnish: much more than just good taste!

Simple and aesthetic desensitising with VOCO Profluorid® Varnish



Advantages of the SingleDose

- Easy, quick and hygienic
- No additional devices required
- No spilling in any position

Perfect application with VOCO Profluorid® Varnish



Advantages of the cartridge

- Multiple use with disposable cannulae
- Perfectly precise application
- Extremely easy application in proximal areas
- Complete fluoride application in only a single step

VOCO Profluorid® Varnish

YOUR CHOICE

Indications

Treatment of hypersensitive teeth
 Treatment of sensitive root surfaces

Advantages

- Moisture tolerant
- Aesthetic, tooth-shaded varnish
- Fast desensitisation and fluoride release (5 % NaF \triangleq 22,600 ppm fluoride)
- In seven pleasant flavours available: melon, mint, cherry, caramel, bubble gum, cola lime and pina colada
- Economical application in thin layers, high yield
- Ideal for treating cervical areas after professional cleaning and calculus removal
- Optimal in case of MIH



Presentation

Flavour	SingleDose 48 x 0.40 ml	SingleDose 50 x 0.25 ml	SingleDose 50 x 0.40 ml	SingleDose 200 x 0.40 ml	Tube 10 ml	Tube 50 ml	Cartridge 5 x 1.70 ml
Melon	–	REF 1268	REF 1269	REF 1271	REF 1267	REF 1272	REF 2235
Caramel	–	–	REF 2226	REF 2229	REF 2232	–	–
Cherry	–	–	REF 2227	REF 2230	REF 2233	–	–
Mint	–	–	REF 2228	REF 2231	REF 2234	–	–
Bubble gum	–	–	REF 2238	REF 2239	–	–	–
Cola lime	–	–	REF 2240	REF 2241	REF 2236	–	–
Pina Colada	–	–	REF 2242	REF 2243	REF 2237	–	–
Mixed	REF 2225	–	–	–	REF 2971*	–	–

REF 2167 Application cannulae type 43, 100 pcs.
 *4 x 10 ml (melon, caramel, cherry, mint)

VOCO GmbH
 Anton-Flettner-Straße 1-3
 27472 Cuxhaven
 Germany

Freecall: 00 800 44 444 555
 Fax: +49 (0) 4721-719-140

info@voco.com
 www.voco.dental

Available from:

