

- (EN)** see [www.fkg.ch](http://www.fkg.ch) for additional information
- (FR)** informations supplémentaires sur [www.fkg.ch](http://www.fkg.ch)
- (DE)** weitere Informationen auf [www.fkg.ch](http://www.fkg.ch)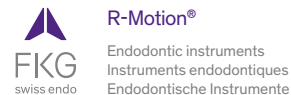

FKG REF. 99.7AA.10.21A.XX - n°112 - 2021/10

## R-Motion® Description - Beschreibung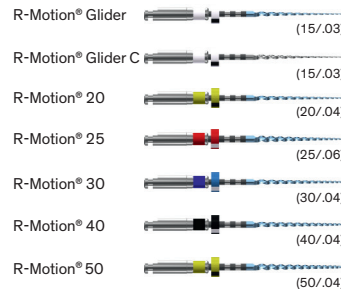

ISO diameter (1) identification  
Identification diamètre ISO (1)  
Identifizierung des ISO-Durchmessers (1)

**Depth marks ▲ (in millimetres)**  
Available on 21 / 25 / 31 mm instruments  
**Marques de profondeur ▲ (en millimètres)**  
Disponibles sur instruments de 21 / 25 / 31 mm  
**Längenmarkierungen ▲ (in Millimeter)**  
Verfügbar für Instrumente der Länge 21 / 25 / 31 mm

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## (EN) General information

### Parameters for use:

R-Motion® instruments must only be used in **reciprocating** (counterclockwise) mode.

### Indications for use:

R-Motion® endodontic instruments are intended for use in medical or hospital facilities, by qualified health professionals.

R-Motion® Glider and R-Motion® Glider C instruments are designed for catheterisation of root canals in reciprocating mode.

R-Motion® 20/25/30/40/50 instruments are designed for shaping and cleaning root canals in reciprocating mode.

### Contraindications:

Nickel-Titanium instruments must not be used on individuals with a known allergic sensitivity to these metals.

**Adverse reactions:** None known.

### Notice and precautions:

- Take several radiographs from different angles to determine the anatomy of the root canals (length, width and curvature).
- Irrigate the canal thoroughly and frequently after the R-Motion® instrument is removed.
- Regularly inspect the R-Motion® instrument during use, and discard if it shows any signs of wear (for example : straightening).
- If the R-Motion® instrument is not progressing easily, withdraw it from the canal, clean it, check its cutting edges, then irrigate the canal and repeat the process with an ISO 010 hand file.
- Single use instruments must not be retreated for subsequent use. This could compromise their integrity.
- Always inspect the instrument(s) before use, and discard (it) them if there are any visible defect(s).
- When an instrument reaches the end of its life, please dispose of it in accordance with the applicable laws and regulations.

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1. Create straight-line coronal and radicular access.
2. Use an ISO 010 hand file to explore the canal.
3. Determine the working length using a radiograph and/or an apex locator.
4. Use the mechanised **R-Motion® Glider** instrument, applying gentle strokes of 2-3 mm to establish a glide path until the working length is reached. After 3 strokes, remove and clean the instrument, then irrigate; recapitulate with an ISO 010 hand file. *Use of the **R-Motion® Glider C** is recommended for canals which require an instrument with more rigidity, and/or if the **R-Motion® Glider** is no longer progressing and can not reach the working length.*
5. Select the R-Motion® single-file shaping instrument which is best suited to the anatomy of the canal being treated. In most cases, and depending on your approach to apical preparation, the **R-Motion® 25** or **R-Motion® 30** will be appropriate. *If the radiographs do not allow you to clearly identify the anatomy of the canal, it can be gauged using hand files.*
6. Start the shaping procedure, applying your usual irrigation/ disinfection protocol.
7. Use gentle inward-outward movements of 2-3 mm, applying very light apical pressure and allowing the instrument to progress passively along the canal. After 3 strokes, remove and clean the instrument, then irrigate; recapitulate with an ISO 010 hand file.
8. Repeat the previous step until the instrument has reached its working length. *If necessary, a brushing motion may be applied to remove coronal interferences and/or ensure uniform shaping of irregular canals. If the instrument is no longer progressing, make one or more passes with the **R-Motion® 20** instrument, then resume using the **R-Motion® 25** or the **R-Motion® 30**. Once the working length has been reached, remove the instrument to prevent over-enlargement of the foramen.*

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Once all the canals have been shaped, proceed with the cleaning of the canals. *For an optimal result, the use of XP-endo® Finisher is recommended.*

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Once the cleaning of the canals is completed, proceed to the next phase of treatment:

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9. Obturate with gutta-percha and sealer. *TotalFill® BC Points™ and TotalFill® BC Sealer™ are recommended.*

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