Instructions for use

- EN** see www.fkg.ch for additional information
- FR** informations supplémentaires sur www.fkg.ch
- DE** weitere Informationen auf www.fkg.ch

XP-endo® Finisher R

Endodontic instruments
Instruments endodontiques
Endodontische Instrumente

FKG REF. 99.X00.10.04A.XX - n°107 - 2021/12

XP-endo® Finisher R

Description - Beschreibung

30/00

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Speed : 800-1000 rpm

Torque : 1 Ncm

- In multirooted teeth, begin with the largest canal.
- Place a drop of solvent into the access cavity before inserting the XP-endo® Finisher R into the canal.
- Work the entire length of the canal for 30 seconds with solvent, 30 seconds with irrigant and 15 seconds with EDTA.

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General information

Indications for use :

These endodontic instruments are intended for use in medical or hospital facilities, by qualified health professionals.

XP-endo® Finisher R is used for endodontic retreatments, to finish off the filling removal phase. XP-endo® Finisher R Files fit into a dental handpiece allowing the user to perform root canal debridement.

Contraindications :

This product contains nickel, and must not be used on individuals with a known allergic sensitivity to this metal.

Adverse reactions : None known

Warnings and precautions :

- Single use instruments must not be retreated for subsequent use. That could compromise their integrity.
- Always inspect the instrument(s) before use, and discard (it) them if there are any visible defect(s).
- When an instrument reaches the end of its life, please dispose of it in accordance with the applicable laws and regulations.

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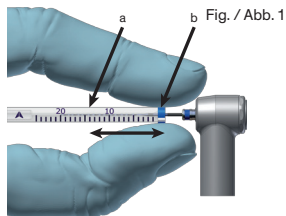
Protocol for use

Speed: 800-1000 rpm
Torque: 1 Ncm

1. XP-endo® Finisher R (XP-FR) must only be used after preparing the canal to at least ISO 30.
 2. Apply a drop of solvent in the canal.
 3. Fix the canal working length by using the ruler on the plastic tube (a) to adjust the endo stop (b) (Fig. 1).
 4. Place the XP-FR into the canal by lining up the tip facing the cusp, insert slightly into the canal (Fig. 2). Then start the motor and slowly thread the XP-FR into the canal.
 5. Use the XP-FR for 30 seconds, using slow and gentle 7-8 mm longitudinal movements to contact the full length of the canal (Fig. 3). Make small multidirectional movements and avoid removing the file from the canal.
 6. Repeat the job, making sure to alternate the solutions (30 seconds with irrigant, then 15 seconds with EDTA) in order to eliminate the suspended debris.
- Between each solution, clean off the debris remaining on the instrument using an alcohol soaked gauze.
- A syringe tip 31G is recommended (or no bigger diameter than 30G).*

Once the cleaning of the canals is completed, proceed to the next phase of treatment:

7. Obturate with gutta percha and sealer. *TotalFill® BC Points™ and TotalFill® BC Sealer™ are recommended.*

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