



Sybron Dental Specialties

Axis Dental  
Coppell, TX 75019 USA  
Tel. 800-355-5063 • Fax 888-462-8879  
e-mail: [custserv@axisdental.com](mailto:custserv@axisdental.com)  
<http://www.axisdental.com>

---

## **Perforated Diamond Finishing Strips**

### **User Information for Perforated Diamond Finishing Strips**

The diamond strips can be used for the removal of excess material and interproximal shaping.

#### **Instructions for cleaning and sterilization:**

1. Please use gloves for your safety
2. Diamond strips should be cleaned with water and soap or a cleaning solution.
3. Diamond instruments are suitable for ultrasonic baths as well, if appropriate clamping implements are used, which prevent instruments from touching.
4. Avoid excessive touching of the strip and ensure all debris is removed prior to use. Each strip must be washed and disinfected before every use.
5. Diamond strips can be vapor-sterilized in an Autoclave with steam at 273°F (134°C) for 5 minutes.
6. Heat or hot air sterilization and thermal disinfection are not suitable for finishing strips.