# GLUMA® Desensitizer

DE	Gebrauchsanweisung.....	3
GB	Instructions for use.....	8
FR	Mode d'emploi .....	13
ES	Instrucciones de uso .....	18
IT	Istruzioni per l'uso .....	23
PT	Instruções de uso.....	28
NL	Gebruiksaanwijzing .....	33
SE	Bruksanvisning .....	38
DK	Brugervejledning .....	43
FI	Käyttöohjeet .....	48
PL	Instrukcja obsługi .....	53
RO	Instrucțiuni de utilizare.....	58
TR	Kullanma talimatı .....	63
RU	Инструкция по применению .....	68
KR	사용 설명서.....	73

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# GLUMA® Desensitizer

Instructions  
for use

**Medical Device – for use by dental health care professionals only.**

## **Intended purpose**

GLUMA Desensitizer is used for the treatment of hypersensitive dentine. It achieves its effects by precipitation of plasma proteins, which reduces dentinal permeability and occludes the peripheral dentinal tubules. This inhibits the flow of fluid through the tubules which is the cause of sensitivity. This dental material is suitable for patients requiring dental treatment for the following indications with consideration of the contraindications. For pregnant and nursing women, the treatment risks must be weighed carefully against the benefits taking into consideration the unborn child or infant. Kulzer medical devices ensure the rehabilitation of oral functions such as chewing, speaking and aesthetics. They stabilize the remaining dentition and/or the alveolar ridge restoratively or prosthetically or they aid in the rehabilitation process.

## **Indications**

- To reduce or even eliminate pain in exposed cervical areas not requiring restoration.
- To alleviate or prevent dentinal sensitivity after preparation of teeth to receive indirect or direct restorations.

## **Contraindications**

The use of this product is contraindicated in case of known or suspected allergies against components of this product. Do not use in case of known or suspected allergy against hydroxyethyl-methacrylate or glutardialdehyde.

GLUMA Desensitizer must not be used:

- if the necessary precautions cannot be taken or the specified application technique cannot be used,
- if the pain is due to pulpitis,
- if the pulp is exposed or the dentine adjacent to the pulp is perforated.

### **Potential side effects**

Hypersensitivities to the product or its components cannot be excluded in individual cases.

### **Product features**

Package sizes/Delivery forms: 5 ml bottle, 0.075 ml Single Dose

### **Composition**

- water • hydroxyethyl-methacrylate • glutardialdehyde

### **Product application**

#### **Treatment of hypersensitive dentine**

The treatment of hypersensitive dentinal surfaces is only indicated in cases of severe pain in reactions to thermal, osmotic or tactile stimuli. In cases of mild pain, alternative treatments such as desensitizing toothpastes should be used.

- Clean the dentine, under local anaesthesia if necessary, and then rinse off with water.
- Protect mucous membranes by using a rubber dam. Make sure that GLUMA Desensitizer only comes into contact with the area to be treated.
- Apply the minimum amount of GLUMA Desensitizer required for one treatment to the dentine surface to be treated (e.g. using an applicator tip, brush or similar) and gently massage in for approx. 30 to 60 seconds.
- Then dry the surface carefully in an air stream until the liquid film has disappeared and the surface is no longer shiny, then rinse with plenty of water under suction.

If a single application of GLUMA Desensitizer does not produce sufficient and sustained pain relief, the treatment can be repeated at the same or a subsequent appointment before alternative treatments are used.

#### **Treatment of prepared dentin to receive indirect or direct restorations**

- Protect mucous membranes by using a rubber dam. Make sure that GLUMA Desensitizer only comes into contact with the area to be treated.
- Apply the **minimum amount** of GLUMA Desensitizer required for one treatment to the dentine surface to be treated (e.g. using an applicator tip, brush or similar) and gently massage in for approx. 30 to 60 seconds.
- Then dry the surface carefully in an air stream until the liquid film has disappeared and the surface is no longer shiny, then rinse with plenty of water under suction.

**Note:** Irritation of the pulp may be observed in the presence of deep cavities without suitable pulp protection. Deep cavities and/or areas near the pulp should be covered with a suitable cavity lining before the desensitizer is applied.

#### **Application notes**

Visually inspect for damage before use. Damaged products must not be used.

#### **General information**

Please pay attention to the following safety instructions and all processing instructions in other sections. This product must be used according to the instruction for use only. Any use not complying with these instructions is at the discretion and sole responsibility of the dental professional itself.

### **Warnings and safety instructions**

GLUMA Desensitizer contains glutardialdehyde. Inhalation or swallowing may be a health hazard. If swallowed, consult a doctor immediately.

May cause skin and eye irritation. Avoid contact with skin. Skin contact may cause sensitization. Wear suitable protective gloves. In case of contact with skin, immediately wash with plenty of water and soap. Avoid contact with eyes. In case of contact with eyes, rinse with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If symptoms persist, consult an eye specialist. May cause respiratory irritation and sensitization. Avoid breathing dust/fume/gas/mist/vapours/spray. Toxic to aquatic life, with long lasting effects. Before use, put on the protective goggles and cover the patient's eyes to protect against splashes of material. Remove only the smallest quantity necessary and close the container immediately. Avoid swallowing at all costs.

### **Special advice**

For SSCP please see EUDAMED database (<https://ec.europa.eu/tools/eudamed>) or contact [sscp-service@kulzer-dental.com](mailto:sscp-service@kulzer-dental.com)

To find the SSCP in the EUDAMED use the UDI code attached at the packaging of the product.

### **Storage conditions**

Avoid direct sunlight. Store in a sufficiently well ventilated dry place. Store at temperatures of -10 to 25°C (14 to 77°F). Incorrect storage may shorten the shelf-life and the product may prematurely lose its effectiveness.

Do not use after expiry date.

**Disposal information**

Recommendation: Dispose of in accordance with official regulations. Do not dispose of contents or partially emptied packaging in the household waste or allow them to enter the sewage system. European Waste Catalogue: 180106 Chemicals consisting of or containing hazardous substances.

**Complaint notice**

Please quote batch number and article number in all correspondence about the product. Article number **REF** and batch number **LOT**: See note on bottle/packaging.

According to EU Medical Device Regulation users/ patients are obliged to report serious incidents with a medical device to the manufacturer and to the competent authority of the country, the incident occurred.

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Kulzer GmbH
Kulzer, LLC