# System B HeatSource™ Model 1005

# SybronEndo

## Instruction Guidelines

### Warnings and Precautions

**DANGER: Not for use in the presence of flammable anesthetics.**  
**WARNING:** Use of other accessories which are not authorized for use in connection with the System B 1005 may cause malfunction and may compromise patient safety.  
**CAUTION:** This device has been investigated with regard to safety from electrical shock and fire hazard as well as electromagnetic compatibility (EMC). The device has not been investigated for other physiological effects.  
**CAUTION:** For use by qualified and trained personnel only.

### Package Contents

P/N	Description
100-500	System B HeatSource Unit
100-535	Endodontic Training Block
500-173	Pinvise Nut
500-320	Bent Contact Clip
500-010	110 V Charge Adapter (for 110V unit only)
500-011	220 V Charge Adapter (for 220V unit only)

### Available System B Tips

952-0003	Kit of four Buchanan Pluggers
952-0004	Fine, Buchanan Plugger
952-0005	Fine Medium, Buchanan Plugger
952-0006	Medium, Buchanan Plugger
952-0007	Medium Large, Buchanan Plugger

### Available Accessories

952-0008	Handpiece Assembly
952-0017	Autoclave Tip Stand
951-0020	110V Charge Adapter
951-0021	220V Charge Adapter
951-0009	Pinvise Nut with Contact Spring
951-0010	Standard Anterior Heat Carrier
951-0011	Standard Posterior Heat Carrier
951-0012	Narrow Anterior Heat Carrier
951-0013	Narrow Posterior Heat Carrier
951-0014	Spoon Tip
951-0016	Hot Pulp Test Tip
951-0017	Thick Plugger (1mm)
951-0018	Thin Plugger (.5mm)
951-0019	Searing Tip

### Symbols

#### Front Panel Symbols

	Attention! Consult the manual before use.
	Temperature Adjustment

#### Back Panel Symbol

	Direct Current.
--	-----------------

#### Bottom Panel Symbols

	Attention! Consult the manual before use.
	CSA mark with "NRTL/C" indicator for certified products.
LR 113060	CSA file number.
	Type BF Patient Applied Part

<b>Index:</b>	page
English	1
Deutsch	3
Français	5
Italiano	7
Dansk	9
Español	11
Suomi	13
Nederlands	15
Svenska	17
Português	19
Norsk	21
Ελληνικά	23
日本	25

### Technique Guide

Downpacking	27
Backfilling	31

### Packaging/ Handling Symbols

	This way up.
	Fragile, handle with care.
	Keep away from rain.
	Temperature limits for storage and transportation.

### Product Features


- The unit operates from a rechargeable battery.**  
A sturdy, sealed lead-acid battery provides quick reliable power. At the maximum temperature the battery will provide one hour of heating power before being depleted.
- Recharge the battery with the charge adapter.**  
The unit is delivered fully charged. To recharge the battery use only the plug-in adapter which comes with the System B HeatSource. Put the unit in the Store mode by using the panel switch. Remove the tip from the probe handle. Plug the charge adapter into a grounded wall outlet and insert the jack at the end of the cord into the receptacle at the back of the unit. The yellow Charge light will come on.

*Note: The System B HeatSource will not operate while charge adapter is plugged into unit.*

*For longest battery life, keep the unit on charge at all times when not in use.* The battery cannot be overcharged.


The battery is fully charged when the yellow Charge light turns off. If the charge adapter is re-inserted when the unit is fully charged, the Charge light will flash once.

The amber **Low** light will light up when you need to fully recharge your battery.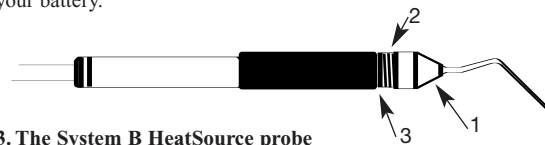

### 3. The System B HeatSource probe handle has special features.

The pinvise nut (1) provides a convenient holder for the different tips. The **contact spring (Ring Switch)** (2) on the pinvise nut allows you to touch activate the System B from any direction while wearing gloves. When you want to heat a tip, depress the spring so that it contacts the **shaft** (3).

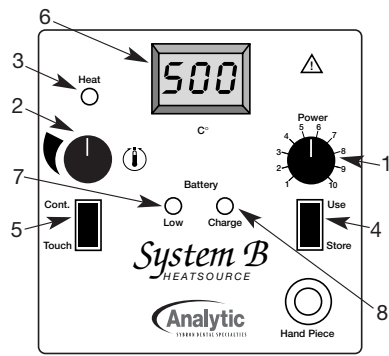**4. The front panel displays all power controls and indicators.** The **power control dial (1)** adjusts the amount of heating power supplied to the tip. The power increases linearly over a range of 1 to 10, 1 being minimal power and 10 being maximum power. The **temperature dial (2)** sets the tip operating temperature in the range 100 to 600°C as shown on the LED display **(6)**. The green **Heat light (3)** comes on when the tip is heating. The **right rocker switch (4)** selects the Use or the Store mode. ALWAYS PUT THE UNIT ON STORE WHEN NOT IN USE. The **left rocker switch (5)** selects either the continuous (Cont.) or the touch mode (Touch). The low battery light **(7)** comes on when the battery needs to be recharged, and the charge light **(8)** is illuminated during charging.

**5. Modes of operation.** In the **continuous mode** the tip is heated continuously at the temperature shown on the display. The heat, but not the display, will turn off automatically after a time delay of 1-2 minutes unless the touch switch has been activated. The time out is reset by operating the touch switch, providing another 1-2 minutes of operation. In the **touch mode** the tip will only be heated when the touch switch is activated.

Whenever the unit is not being used it should be put in the *Store* mode with the panel switch. The unit is deactivated in the *Store* mode preventing battery drain. This will also prevent the tip from heating in case the touch switch is activated accidentally. *This feature should always be used when changing tips.*

**6. Patented design.** The System B HeatSource uses a patented design by which the temperature of each tip is sensed at the very end and controlled automatically in order to keep the tip operating at its preset temperature. The tip reaches this preset temperature in less than a second using maximum available power while heating up. As soon as the final temperature has been reached, the heating power is cut off to the low level needed to keep the tip at the preset temperature. If the tip temperature drops, the heating power is immediately increased to compensate for this temperature drop. Both the design of the tip and the operation of the control unit are protected by patents.

**7. The power output and control feature.** The Power dial adjusts the amount of power available to heat the tip. The maximum output power is about 20W. When the tip is cooled down, for example by making contact with gutta percha, the heating power is automatically increased. Best performance is obtained when operating at full power, i.e. a power setting of 10. The tip life can be extended to some degree by operating at a lower power setting. However, this will decrease the heating performance.

## System B HeatSource User Guidelines

**1. Installing the tip.** Put the unit in the Store mode using the panel switch. Loosen the pinwise nut and insert the tip as far as it will go. The thicker part of the tip should disappear into the pinwise. Hand tighten the nut.

**2. Tip life depends on temperature.** The tips have life expectancies which greatly depend on the operating temperature and power setting. The tip life could be relatively short when operating at maximum temperature (600°C) and maximum power setting (10). All tips will gradually lose their heating efficiency over a period of time.

## Cleaning, Sterilization, and Maintenance

**Sterilization**  
Pluggers and tips must be cleaned and sterilized before every use:

1. Clean with water and mild, non-abrasive, detergent such as dish washing liquid. Dry thoroughly.
2. Double wrap the pluggers and tips using a typical central service wrapping technique and place into autoclave unit avoiding contact with other instruments.

3. Operate sterilization cycle at 250°F for 30 minutes or 270°F for 25 minutes. Follow the instructions supplied with your autoclave unit for specific operating instructions.

**Expected Life of Accessories**  
All accessories are reusable only for a limited number of procedures. This number is highly dependent on how and for what duration the instrument is used during each procedure and, therefore, will vary significantly from user to user. If the device function appears to be erratic, an accessory may have exceeded its useful life and should be replaced.

**Cleaning**  
The System B 1005 enclosure and the handpiece should be wiped down with a cloth soaked in any of the common non-abrasive, mild, cleaning liquids containing alcohol. Do not submerge the device or let liquid enter the enclosure and handpiece.

## Technical Specifications

**Dimension/Weight**  
Casing of unit: Weight: 4.0 lb. (1.81 kg)  
Dimensions: Height: 5.2 inches/132 mm  
Width: 4.6 inches/117 mm  
Depth: 3.7 inches/94 mm

**Classification:** IEC 601-1 Internally Powered Equipment  
**Mode of Operation:** IEC 601-1 Continuous Operation  
**Degree of safety of application in presence of a flammable anesthetic mixture:** Unit unsuitable for use in presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.  
**Conditions for use:** +10°C to +40°C  
30% to 75% relative humidity  
**Storage and transport conditions:** -10°C to +45°C  
10% to 95% relative humidity

**Origin:** SybronEndo  
1332 South Lone Hill Avenue  
Glendora, CA 91740, USA  
**Model:** System B Heatsource 1005  
**Power Supply:** rechargeable sealed lead-acid battery

## Warranty Information

**Returns and Warranty**  
The System B 1005 is designed for lasting durability and predictable results. The unit comes with a one-year warranty. If the unit should fail to operate correctly, please contact your SybronEndo customer service representative at 1-800-346-3636. Outside the United States, please call customer service at 714-516-7979 or contact your local dealer or supplier to coordinate the returns/repairs with SybronEndo.

Warranty repairs can only be performed by SybronEndo or by authorized agencies with original factory parts. Any unauthorized repairs will void the warranty.

1. Use the original packaging when shipping or storing your System B 1005.
2. Many problems can be solved over the phone. Do not hesitate to contact us if you experience difficulties when using the System B.
3. For returns, call us for a Return Good Authorization (RGA) number.
4. Damages incurred during shipping due to improper packaging are not covered by the warranty. If the original box and/or foam packing are not available, please call SybronEndo for packing instructions.
5. Mark all outer packaging with the RGA number, your name, address, and phone number.
6. Shipping costs are not covered by the warranty.

**Disclaimer**  
The responsibility of SybronEndo, as a manufacturer of electrical/medical devices, extends to the technical safety features of the device only if maintenance and repairs are carried out by SybronEndo or by authorized agencies with original factory parts.

For safety reasons, this product should be used with accessories manufactured and sold by SybronEndo. Any use of non-authorized accessories or not following any of the instructions for use is done so at the operator's risk and voids the warranty. SybronEndo does not assume any responsibility for poor results due to operator error or equipment malfunction.

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# The Buchanan “Continuous Wave of Condensation” Technique

English

## Downpacking with the System B HeatSource™

1. Choose the System B Plugger that matches the gutta percha cone fit in the canal. Place a rubber stop on the plugger, and prefit the plugger to its binding point in the canal (usually 5-7 mm short of length). Adjust the stop to the highest reference point and remove the plugger. The pluggers are dead-soft stainless steel and can be bent to follow canal curvature.

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## English

7. Still maintaining apical pressure, push the touch spring again for a full 1 second. Release the switch, pause for a full 1 second, then quickly withdraw the plugger.

*Note: Because the System B Pluggers heat from their tips back, the heat burst in this portion of the procedure causes rapid severance of the instrument and the coronal surplus of gutta percha from the already condensed and set apical mass.*

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English

3. Hold for 5 seconds. Maintaining apical pressure, rotate the System B Plugger to break it loose. Now, with light back-pressure, rotate the plugger again to separate it from the condensed backfill cone.

*Note: The low temperature setting and the momentary heat cycle allows the backfilling gutta percha to be condensed without heating it to a sticky state, which could cause it to pull out.*

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