

THE **APPROACH TO HEMOSTASIS**

- SPECIALLY DESIGNED for the esthetic zone TRANSPARENT GEL stays on the sulcus

 - LEAVES NO RESIDUE and rinses off easily DOES NOT INTERFERE with bonding

ACHIEVE HEMOSTASIS...ESTHETICALLY

This viscous yet spreadable 25% aluminum chloride gel quickly stops minor bleeding and sulcular fluid without the mess of a liquid or paste.



Subgingival preparation and bleeding sulcus.



Rub hemostatic firmly against bleeding tissues. Achieve the most rapid hemostasis with the Metal Dento-Infusor[™] tip. The clear gel allows easy visibility and rinses away quickly.



Press Ultrapak[™] cord into gel in sulcus around tooth preparations. Leave 4–5 minutes.



Remove cord. Firm air/water spray. Air dry. Rub gel into the sulcus again. Leave 1 minute.



Finished restoration 2 weeks post-op. Facilitates great control in the esthetic zone with no residue.

- SPECIALLY DESIGNED for the esthetic zone
- TRANSPARENT GEL stays on the sulcus
- LEAVES NO RESIDUE and rinses off easily
- DOES NOT INTERFERE with bonding*



*Data on file



6407 - ViscoStat Clear Dento-Infusor IndiSpense Syringe Kit

1 x 30 ml IndiSpense syringe 20 x Metal Dento-Infusor tips 20 x 1.2 ml empty syringes



6409 - ViscoStat Clear Dento-Infusor Syringe Kit

4 x 1.2 ml syringes 20 x Metal Dento-Infusor tips



6408 - ViscoStat Clear IndiSpense Syringe 1pk 30 ml syringe



6410 - ViscoStat Clear Syringe 20pk 1.2 ml syringes

